INSTRUCTIONS FOR CONDOM USE

Manufactured to ISO 4074

LTC HealthCare Customer Care Telephone: 1-800-4-A-LTC (4362) or 1-877-326-6897

11100 Linden Road, Linden, NJ 07036
 11100 Linden Road, Linden, NJ 07036
 11100 Linden Road, Linden, NJ 07036

11100 Linden Road, Linden, NJ 07036
 11100 Linden Road, Linden, NJ 07036
 11100 Linden Road, Linden, NJ 07036

UK INSTRUCTIONS FOR CONDOM USE

- Check the date on the condom foil to ensure the condom is within the expiry date. Usually printed on the back of the foil. Tear along one edge of the condom foil making sure you do not damage the condom with finger nails or jewellery. If the condom foil is damaged discard that condom and use a new one from an undamaged pack.
- If you are using a lubricant only use a water based lubricant such as 'DIX' or 'Reps Oen' do not use oil based lubricants such as petroleum jelly as these may damage the condom.
- Hold the condom between your finger and thumb and roll the condom onto an erect penis before contact with your partner. Unroll down the full length of the penis pulling any air from the condom.
- Latex condoms are intended to prevent pregnancy, HIV/AIDS, and other sexually transmitted infections but do not completely eliminate the risks of pregnancy and STIs. Latex condoms are most effective against STIs such as HIV infection (AIDS) and gonorrhoea that are spread by contact with the fluid of the penis. Latex condoms are less effective against STIs such as Human Papillomavirus (HPV) and herpes. These STIs can also be spread by contact with infected skin that is not covered by the condom. Latex condoms reduce the risk of transmitting STIs by preventing contact against the occurrence of the infection. If you believe you have an STI, contact a health care provider. Latex condoms are not a substitute for sexually transmitted infection and pregnancy. For more information on latex condoms or STIs, contact a health care provider or a public health agency.
- If you feel the condom is slipping off, or is tightening excessively then withdraw the penis from your partner and check the condom is in place and not damaged. If the condom has moved then roll it back in place or use another condom if you are concerned about the condom being damaged. If the rubber material is sticky or brittle or obviously damaged, do not use the condom. If the colour is uneven or changed, do not use the condom. Condoms are made of natural rubber latex which may cause allergic reactions including anaphylactic shock if the user allergic to latex.
- After ejaculation withdraw the penis from your partner holding the base of the condom to ensure it does not slip off. When the penis is withdrawn from your partner you can then remove the condom.
- If you use conventional latex condoms have damaged during sex then seek medical assistance such as a Doctor or Sexual Healthcare adviser preferably within 24 hours, but not later than 72 hours. If the condom foil is damaged discard that condom and choose another condom from the pack. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom.
- Use a condom every time. Re-use of a condom may increase the risk of pregnancy, HIV/AIDS, transmission of sexually transmitted infections including HIV. Dispose of the condom by wrapping it in tissue and placing it in a waste bin. Store any unused condoms in a cool dry place away from direct sunlight.

PRECAUTIONS:
 If there are lesions, open sores, or if you have had intercourse on either partner, DO NOT ENGAGE IN SEXUAL PENETRATION AND CONSULT YOUR DOCTOR.

WARNING:
 FOR MAXIMUM PROTECTION IT IS IMPORTANT TO FOLLOW THESE INSTRUCTIONS CAREFULLY.

CAUTION:
 This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

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