

10 pcs

Windi® is a single-use catheter that helps babies get rid of excess gas. Designed to be safe and effective, Windi® cannot harm your baby if used as instructed. Windi® is a Medical Device class 1, registered with the FDA and the Swedish Medical Products Agency.

### How to use Windi®:

1. Always use baby oil or similar products on Windi® to facilitate the insertion. Gently massage the baby's tummy with your hand in a downward motion.

2. Lift the baby's legs towards the stomach. Carefully insert the catheter (as if you were using a thermometer).



3. You will know that the treatment is working when you hear a "whistling" sound and the gas comes out.

4. The treatment is complete when the "whistling" sound ends and your baby is no longer in pain. Dispose of the catheter.



Modelnummer: 05100000 Medical AB, SE 14100 Helsingborg | © 2014 Windi Medical AB. All rights reserved.

Warning! Never leave the catheter in the baby's rectum after the treatment is complete. Always discard Windi® after you have finished the treatment. Do not repeat the treatment more than three times per day.