cresta

BPM 620

AUTOMATISCHE ARM BLOEDDRUKMEETER
TENSIO MÈTRE AU BRAS AUTOMATIQUE
AUTOMATIC ARM BLOODPRESSURE MONITOR
AUTOMATISCHE ARM BLUIDRUCKMESSER

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Shenzhen Pango Electronic Co., Ltd
No.25, 1st Industry Zone, Fenghuang Road,
Xikeng Village, Henggang Town, Longgang District
Shenzhen, Guangdong, China, 518115

Authorised representative: Lotus Global Co. , Ltd
Address : 1 Four Seasons Terrace West Drayton,
Middlesex London, UB7 9GG United Kingdom
Tel : 0044-20-75868010, 0044-20-70961611
Fax : 0044-20-79006187

We thank you for buying our CRESTA electronic upper arm Blood Pressure Monitor BPM 620. It shows your confidence in our brand and our products. Please be assured you have bought a high quality product which you will enjoy using for many years to come.

If you require technical support and/or have questions about the functioning of the product, please contact our After Sales Service

+ 31 900 4000030

Read this manual thoroughly before your first use, even if you are familiar with this type of product. It provides you with tips to make optimal use of this product. Keep the manual in a safe place for future reference, along with the warranty card, purchase receipt and carton.

TABLE OF CONTENTS

INTRODUCTION

NOTES ON SAFETY

ABOUT BLOOD PRESSURE

PRECAUTIONS BEFORE USE

FEATURES OF THE PRODUCT

PARTS IDENTIFICATION

INSERT OR REPLACE BATTERIES

TIME AND SYSTEM SETUP

UNIT CONVERSION mmHg / kPa

WHO BLOOD PRESSURE CLASSIFICATION DISPLAY

ATTACHING THE ARM CUFF

HOW TO TAKE PROPER MEASUREMENTS

HOW TO MEASURE BLOOD PRESSURE

CARE AND MAINTENANCE

SPECIFICATIONS

TROUBLESHOOTING

INTRODUCTION

This Blood pressure monitor uses the oscillometric method of blood pressure measurement. The blood pressure monitor is intended for use by medical professionals or at home to monitor and display the diastolic, the systolic blood pressure and the pulse rate of an adult, with an arm cuff around the left upper arm according to the instruction in "ATTACHING THE ARM CUFF". The expected life time of the product is over 5 years. The product complies with the electromagnetic compatibility requirement of EN60601-1-2, safety standards of EN60601-1 and performance of IEC 80601-2-30 as specified in EEC directive 93/42/EEC.

NOTES ON SAFETY

The warning signs and sample icons shown here are listed for your safe and correct use of the unit, so as to prevent injuries or damages to the device.

The icons and meanings are as follow.

Examples of signs

The icon  indicates prohibitions - what you should not do. Matters involving actual prohibitions are indicated by text or pictures in or near . The left icon refers to "general prohibitions".

The icon  indicates something that is compulsory - what must always be observed. Matters involving actual compulsory actions are indicated by text or pictures in or near . The left icon refers to "general compulsions".

The icon  indicates something can't be disassembled or do not disassemble. Matters involving actual compulsory actions are indicated by text or pictures in or near . The left icon refers to "General prohibitions".

Type BF Applied part

Attention and read before use

**Marking of electrical and electronic
equipment in accordance with Article
11(2) of Directive 2002/96/EC (WEEE)**

Warning

The patient should follow the doctor's instructions and should not perform self-judgment and self-treatment by the measuring result. Self-diagnosis of measured results and treatment are dangerous. The device should not be used to judge illness, first aid and continuously monitor measuring.

This device cannot be used during Patient transport and surgical care. It can be used in households or fixed places only.

Please press the "ON/OFF" button to stop the operation when you feel uncomfortable with the arm, or if the air is inflating abnormally without stop.

Do not let a child below 12 years old and people who can't express one's intention use this appliance. When it is used by children of 12~18 years old, they should be supervised by an adult. This may cause an accident or trouble.

Do not use the unit for another purpose than measuring blood pressure.
This may cause an accident or trouble.

Please do not use a mobile phone around the device. Please do not use the device around the magnetic field.

Do not disassemble, repair, or remodel the main unit or the cuff of the blood pressure monitor. This will cause the unit not to function or to function erroneously.

TIPS FOR OPTIMAL USE

Make sure there is no kink in the connection tube before you start measuring to avoid any injury to the patient.

For any patient, do not measure more than 3 times continuously, it should be at least above 5 minutes of interval rest between any two measurements, otherwise will cause extravasated blood.

Do not measure your blood pressure over 6 times each day.
Do not apply the cuff over a wound as this can cause further injury.

Do measure on the arm which is on the side of a mastectomy, otherwise it could cause injury.

Observe the air pressure value from the LCD display.

When measuring, it should not exceed 299 mmHg, otherwise press the "ON/OFF" button to stop the measurement.

Do not use force to bend the arm cuff or the air tube.

Do not knock or drop the main unit.

Always use the specified accessories in the manual, the use of other parts not approved by the manufacturer may cause faults or injuries.

For service information, parts list etc., please contact our service department

+31 900-400030

ABOUT BLOOD PRESSURE

What is blood pressure?

Blood pressure is the force exerted by the blood against the walls of the arteries. The Systolic pressure occurs when the heart contracts. The Diastolic pressure occurs when the heart expands and re-fills. Blood pressure is measured in millimeters of mercury (mmHg). One's natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while one is still at rest and before eating.

What is hypertension and how is it controlled?

Hypertension, an abnormal high arterial blood pressure, if left unattended, can cause


many health problems including a stroke and a heart attack. Hypertension can be controlled by altering your lifestyle, avoiding stress and with medication under a doctor's supervision. To prevent hypertension or keep it under control: Do not smoke, exercise regularly, reduce salt and fat intake, have regular physical checkups and maintain a proper weight and control this.

Why measure blood pressure at home?

Blood pressure measured at a clinic or doctor's office may cause apprehension and produce an elevated reading, 25 to 30 mmHg higher than that measured at home, Home measurement reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.

WHO blood pressure classification

Standards for a assessment of high blood pressure, without regard to age, have been established by the World Health Organization (WHO), and shown in chart below.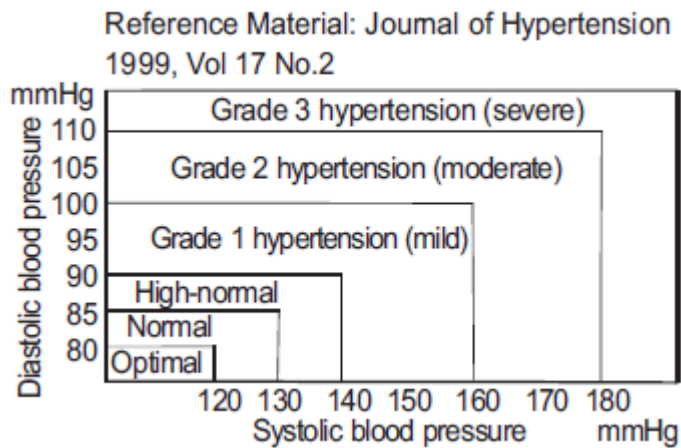

Blood pressure variations

An individual's blood pressure varies greatly on a daily and seasonal basis. It may vary by 30 to 50 mmHg due to various conditions during the day. In hypertensive individuals, variations are even more pronounced. Normally, the blood pressure rises while at work or play and falls to its lowest levels during the sleep. So, do not be overly concerned by the results of one measurement. Take measurements at the same time every day using the procedure described in this manual and know your normal blood pressure. Many readings give a more comprehensive blood pressure history. Be sure to note date and time when recording your blood pressure. Consult your doctor to interpret your blood pressure data.


PRECAUTIONS BEFORE USE

1. If you are taking medication, consult with your doctor to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your doctor.
2. For people with an irregular or unstable peripheral circulation problems due to diabetes, liver disease, hardening of the arteries, etc., there may be fluctuation in blood pressure values measured at the upper arm versus at the wrist.
3. Measurements may be impaired if this device is used near televisions, microwave ovens, X-ray equipment, mobile phone equipment or other devices with strong electrical fields. To prevent such interference, use the monitor at a sufficient distance from such devices or turn them off.
4. Before using, wash your hands.
5. Do not measure on the arm which simultaneously used monitoring ME Equipment, otherwise it could cause loss of function.
6. Consult your doctor if unexpected readings are obtained, also please refer to "Trouble shooting" of the manual.
7. The reading is probably a little lower than measured in the hospital due to the steady mood at home.
8. Cuff pressure range 0-299mmHg

FEATURES OF THE PRODUCT

- Memory can store 90 measurements.
- Large and clear LCD display.
- WHO blood pressure classification display.
- Easy to use, press one button to automatically measure, record the measurement values and measurement time.
- Automatically turns off (within 1 minute) to save power.

PARTS IDENTIFICATION

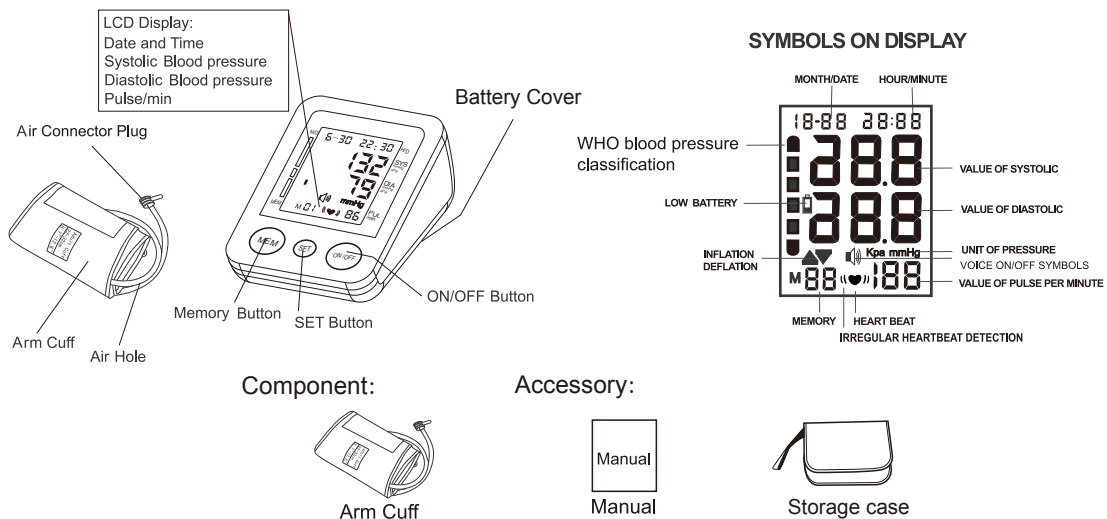INSERT OR REPLACE BATTERIES

- Remove the battery cover.
- Insert new batteries as indicated in the battery compartment, taking care that the polarities (+) and (-) are correct.
- Close the battery cover. Use only IR6, AA batteries.

Dispose of empty batteries to the authorized collecting party subject to the regulation of each individual territory.

CAUTION

Insert the batteries as shown in the battery compartment. If not, the device will not work. When  - LOW BATTERY mark - blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction. LOW BATTERY mark  does not appear when the batteries run out. Please ensure to distinguish the positive polar "+" and the negative polar "-" of the batteries when replacing the batteries. Batteries, which have fluid on the surface or which are modified, cannot be inserted into the product. Battery short circuit must be prevented. Battery life varies with the ambient temperature and may be shortened at low temperatures. The batteries may leak and cause a malfunction. Use the specified batteries only. The batteries provided with the device are for testing the monitor's performance and may have a shorter life. Used batteries may leak and damage the main unit. If you are not going to use the unit for a long period of time (approximately three months or more), remove the batteries. Replace worn batteries with their polarities in the correct direction.

TIME AND OF SYSTEM SETUP

1. Press the "SET" key to turn the device on.
2. Press and hold the "SET" key until the year number displays and flashes on LCD to enter the setting mode.
3. Press the "MEM" key to adjust the year, then press the "SET" key again to save your setting and enter the month setting mode.
4. Press the "MEM" key to adjust the month. Following the same steps to adjust date/hour/minute.

UNIT CONVERSION mmHg/ kPa DISPLAY

The default setting of the monitor is mmHg. Also kPa (kPa) can be set.

Boots continued to press the ON/OFF button exceeding five seconds.

The units will be chosen by the above shows mmHg/kPa after de control. After the normal boot unit values are shown as blood pressure. Also select memory unit value changes.

WHO BLOOD PRESSURE CLASSIFICATION DISPLAY

Diastolic blood pressure reference material: Journal of hypertension 1999. vol 17 No.2

Grade 3 hypertension (severe)


Grade 2 hypertension (moderate)

Grade 1 hypertension (mild)

High-normal

Normal

Optimal

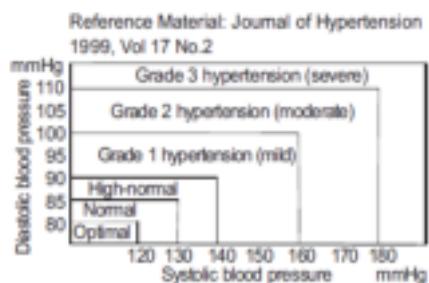ATTACHING THE ARM CUFF

1. Wrap the arm cuff around the upper arm, about 2 - 3 cm above the elbow.

Place the cuff directly to the skin, as clothing may cause a faint pulse, and result in a measurement error.

2. Constriction of the upper arm, caused by rolled up shirtsleeves, may prevent accurate readings.

3. Secure the arm cuff with Velcro Strip in such a way that it lies comfortably and is not too tight. Lay the arm on the table with the hand palm upwards so that the arm cuff is at the same height as the heart. Make sure that the tube is not kinked.

4. Measure your arm circumference for the cuff selection, refer to "Specifications"

How to take proper measurements

For the best accuracy in blood pressure measurement:

PATIENT position in NORMAL USE, including:

- 1) take a comfortable seat
- 2) keep your legs uncrossed.
- 3) place your feet flat on the floor.
- 4) keep your back and arm supported.
- 5) put the middle of the CUFF at the level of your heart.

Remain still and keep quiet during the measurement.

Relax as much as possible and do not talk during the measurement process.

Measure your blood pressure at about the same time every day.

Do not measure right after a physical exercise or a bath. Take a rest for twenty to thirty minutes before taking the measurement.

The readings can be affected under the below conditions:

Within an hour after dinner, after having wine, coffee, tea, sports, bathing, talking, being nervous, being in an unsteady mood, bending forward, moving, room temperature dramatically changing during measuring, in moving vehicles, long time continuous measuring.

HOW TO MEASURE BLOOD PRESSURE

1. Set up the arm cuff to your upper arm as previously described in the section "ATTACHING THE ARM CUFF"
2. Press the "ON/OFF" button, all icons appear two seconds on the display, then it will switch to measurement and will display "0" or the last measurement recorded.
3. Start the measurement, the cuff in the strap will automatically inflate. The mark () will flash on LCD till the measurement is completed. The LCD display will show the measurement results.

READ MEMORY

Press the "MEM" button, to read out the latest measurements. Press the "MEM" button for the latest results (UP) or the "SET" button for the memory (DOWN)

Power Measurement closure or after the end of the state you can press the "MEM"

button to read out the latest measurement of the memory.

DELETE MEMORY

Press the “MEM” button for five seconds, the LCD will display “No”. The memory has been deleted.

CARE AND MAINTENANCE

To keep your digital blood pressure monitor in the best condition and to protect the unit from damage, please follow the directions listed below:

Keep the monitor in the storage case when not in use.

Do not fold the arm cuff too tightly. The fabric fastener could touch the inner surface of the arm cuff and damage it.

Clean the monitor and the cuff with a soft dry cloth.

Do not use any abrasive or volatile cleaners.

CAUTION

* Do not submerge the device or any of the components in water.

* Do not subject the monitor to extreme hot- or cold temperatures, humidity or direct sunlight.

* Store the device and the components in a clean, safe location.

* Do not subject the monitor to strong shocks, such as dropping the unit on the floor.

* Remove the batteries if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

This product is designed for use over an extended period of time.

SPECIFICATIONS

Measuring Method	Oscillometric Measurement
Indication	Digital LCD display
Measuring Range	Pressure : 30~280 mmHg Pulse : 40~199 Be a t/min
Accuracy	Static Pressure : ± 3 mmHg Pulse : $\pm 5\%$
Memory	90 Memories
Power supply	4x 1.5V Batteries (LR6 or AA) use alkaline battery, measure above 200 times
Operating condition:	+5°C ~+40°C. 15%RH-93%RH Atmospheric pressure : 70kPa ~106kPa
Storage condition:	-20°C ~+55°C. 0%RH-93%RH Atmospheric pressure :50kPa ~106kPa
Dimensions	Approx.: 96(W)X130(H)X60(D)mm
Weight:	Approx.: 380g, excluding batteries

Classification	Type BF
Upper arm circumference	(22 ~32) cm
AC adapter	INPUT: 100-240V~ 50/60Hz, 0.2A OUIPUT: 6V ===500mA

- Specifications may be changed without notice in the event of improvements being made.

- 1.Type of protection against electric shock: INTERNALLY POWERED EQUIPMENT
- 2.Degree of protection against electric shock: TYPE BF APPLIED PART
- 3.Mode of operation: CONTINUOUS OPERATION.
- 4.Equipment not suitable for category AP&APG equipment use in presence.

STATEMENT

The system might not meet its performance specifications if stored or used outside the temperature and humidity as mentioned below:

Operating conditions: +5°C~+40°C. 15%RH~93%RH 70kPa ~106kPa

Storage conditions: -20°C~+55°C. 0%RH~93%RH

TROUBLESHOOTING

If you have trouble in using the unit please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed When you push the POWER button or the battery icon flash	No battery installation	Insert batteries
	Battery worn out	Replace new batteries
	The polarities of the batteries placed wrongly	Insert the batteries in the correct polarities
E1: can't normally Increase pressure	Check your arm cuff if any air leakage	Replace arm cuff with new one
E3 inflate pressure too high		Re-measurement or send back dealer for re-calibrate pressure
E2/E4: have moved during the measurement	Hand or body shaking while measuring	keep static and in the correct gesture to measure again
Battery icon on	Battery low power	Replace battery and measure again
The systolic pressure value or diastolic pressure value too high	1.The arm cuff was held lower than your heart	keeping correct position and gesture to measure again
	2.The arm cuff was not attached properly	
	3.You moved your body or spoke during measurement	

The systolic pressure value or diastolic pressure value too low	1.The arm cuff was held higher than your heart	
	2.you moved your body or Spoke during measurement	

Regarding EMC Statement

a) a statement to the effect that this equipment needs to be installed and put into service in accordance with the information provided in the ACCOMPANYING DOCUMENTS; and

b) a statement to the effect that wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie talkies can affect this equipment and should be kept at least a distanced away from the equipment. The distanced is calculated by the MANUFACTURER from the 800 MHZ to 2,5 GHz column of Table 5 or Table 6 of IEC 60601-1-2:2007, as appropriate.

The crossed out "wheeled bin" icon on the product reminds you of your obligation that when you dispose of the appliance, it should be separately collected. Contact your local authority for information concerning the correct disposal of your old appliance.

Shenzhen Pango Electronic Co., Ltd
No.25, 1st Industry Zone, Fenghuang Road,
Xikeng Village, Henggang Town, Longgang District
Shenzhen, Guangdong, China, 518115

Authorised representative: Lotus Global Co. , Ltd
Address : 1 Four Seasons Terrace West Drayton,
Middlesex London, UB7 9GG United Kingdom
Tel : 0044-20-75868010, 0044-20-70961611
Fax : 0044-20-79006187

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