# VERMEIREN

## V300 DL

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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## Contents

<b>Preface</b> .....	<b>2</b>
<b>1 Product description</b> .....	<b>3</b>
1.1 Intended use.....	3
1.2 Technical specifications .....	4
1.3 Components.....	5
1.4 Accessories.....	6
1.5 Location identification plate.....	6
1.6 Explanation of symbols .....	6
1.8 Safety rules .....	7
<b>2 Use</b> .....	<b>7</b>
2.1 Carrying the wheelchair .....	7
2.2 Mounting the rear wheels .....	8
2.3 Unfolding the wheelchair .....	8
2.4 Mounting or removing of stabilisator.....	8
2.5 Mounting or removing of the footrests .....	9
2.6 Operating the brakes.....	9
2.7 Mounting or removing of armrests.....	10
2.8 Adjusting the armrest .....	11
2.9 Adjusting anti-tipping (only mounted to the right side).....	11
2.10 Adjusting height hand grips .....	12
2.11 Transfer in and out the wheelchair .....	12
2.12 Correct position in the wheelchair .....	12
2.13 Riding the wheelchair.....	12
2.14 Moving on slopes .....	13
2.15 Negotiating steps or curbs .....	13
2.16 Fold up the wheelchair .....	15
2.17 Taking off the rear wheels .....	15
2.18 Transport in the car.....	15
2.19 Use of the wheelchair as seat in a motor vehicle .....	16
<b>3 Installation and adjustment</b> .....	<b>18</b>
3.1 Tools .....	18
3.2 Manner of delivery .....	18
3.3 Adjusting the seat height and seat angle .....	18
3.4 Adjusting the seat depth .....	21
3.5 Adjust the brakes .....	22
3.6 Adjusting of the footrests .....	22
3.7 Adjusting the depth arm pad .....	23
3.8 Adjusting the stability and the maneuverability.....	24
<b>4 Maintenance</b> .....	<b>24</b>

## Preface

First of all we want to thank you for putting your trust in us by selecting one of our wheelchairs.

The Vermeiren wheelchairs are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the wheelchair.

The expected lifetime for your wheelchair is strongly influenced by the care and maintenance of the wheelchair.

This manual will help you get acquainted with the operation of your wheelchair.

Following the user instructions and the maintenance instructions are an essential part of the guarantee.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

# 1 Product description

## 1.1 *Intended use*

The wheelchair is intended for people with walking difficulties or no walking abilities.

The wheelchair is designed to transport 1 person.

The wheelchair is suited for indoor and outdoor use.

The user can propel the wheelchair by himself or have the wheelchair pushed by an attendant.

The different types of fittings and accessories, and the modular construction allow full use by persons disabled by:

- paralysis
- loss of limbs (leg amputation)
- limb defects or deformations
- stiff or damaged joints
- heart insufficiencies and poor blood circulation
- balance disturbances
- cachexia (decrease in muscle)
- and also for aged persons.

When providing for individual requirements:

- body size and weight (max. 130 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your wheelchair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.).

The wheelchair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

## 1.2 Technical specifications

Technical terms below are valid for the wheelchair in standard settings. If other footrests / armrests or other accessories are used, the tabulated values will change.

<b>Make</b>	Vermeiren					
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout					
<b>Type</b>	Manual wheelchair					
<b>Model</b>	V300DL					
<b>Maximum occupant mass</b>	130 kg					
Description	Min.			Max.		
Overall length with footrest	1000 mm (minimum length footrest)			1040 mm (maximum length footrest)		
Overall height (depends on seat height - measured for seat height 500 mm)	890 mm (minimum height hand grip)			990 mm (maximum height hand grip)		
Effective seat width	390 mm	420 mm	440 mm	460 mm	480 mm	500 mm
Overall width (depends on the seat width)	600 mm	630 mm	650 mm	670 mm	690 mm	710 mm
Folded length without footrest	810 mm					
Folded width	320 mm					
Folded height	890 mm			990 mm		
Total mass	18,80 kg (measured for seat width 460 mm)					
Mass of heaviest part	11 kg					
Masses of parts that can be dismantled or removed	<b>Footrests:</b> 1,80 kg; <b>Armrests:</b> 2,20 kg; <b>Rear wheels:</b> 3,80 kg					
Static stability downhill	10° (in standard configuration)					
Static stability uphill	7° (in standard configuration)					
Static stability sideways	18° (in standard configuration)					
Obstacle climbing	60 mm					
Seat plane angle	0°			10°		
Effective seat depth	440 mm			520 mm		
Seat surface height at front edge (measured on top of the cross tubes)	440 mm			530 mm		
Thickness seat cushion	50 mm					
Backrest angle	5°					
Backrest height	460 mm					
Distance between footrest and seat	420 mm					
Angle between seat and leg footrest	105°					
Angle between leg footrest and foot plate	80°			100°		
Distance between arm pad and seat (without seat cushion)	195 mm			300 mm		
Front location of arm pad structure	335 mm			455 mm		
Handrim diameter	535 mm					
Horizontal location of axle (deflection)	1,27 mm			61,04 mm		
Minimum turning radius	1500 mm					
Diameter PU Rear wheels	22"			24"		
Tyre pressure, rear (driving) wheels	Max. 3.5 bar					

<b>Make</b>	Vermeiren	
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout	
<b>Type</b>	Manual wheelchair	
<b>Model</b>	V300DL	
<b>Maximum occupant mass</b>	130 kg	
Diameter PU steering wheels	200 mm	
Tyre pressure, steering wheels	Max. 2.5 bar	
Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%

**We reserve the right to introduce technical changes. Measurement tolerance  $\pm 15$  mm / 1,5 kg / °**


*Table 1: Technical specifications*

The wheelchair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

ISO 7176-16: Resistance to ignition of upholstered parts
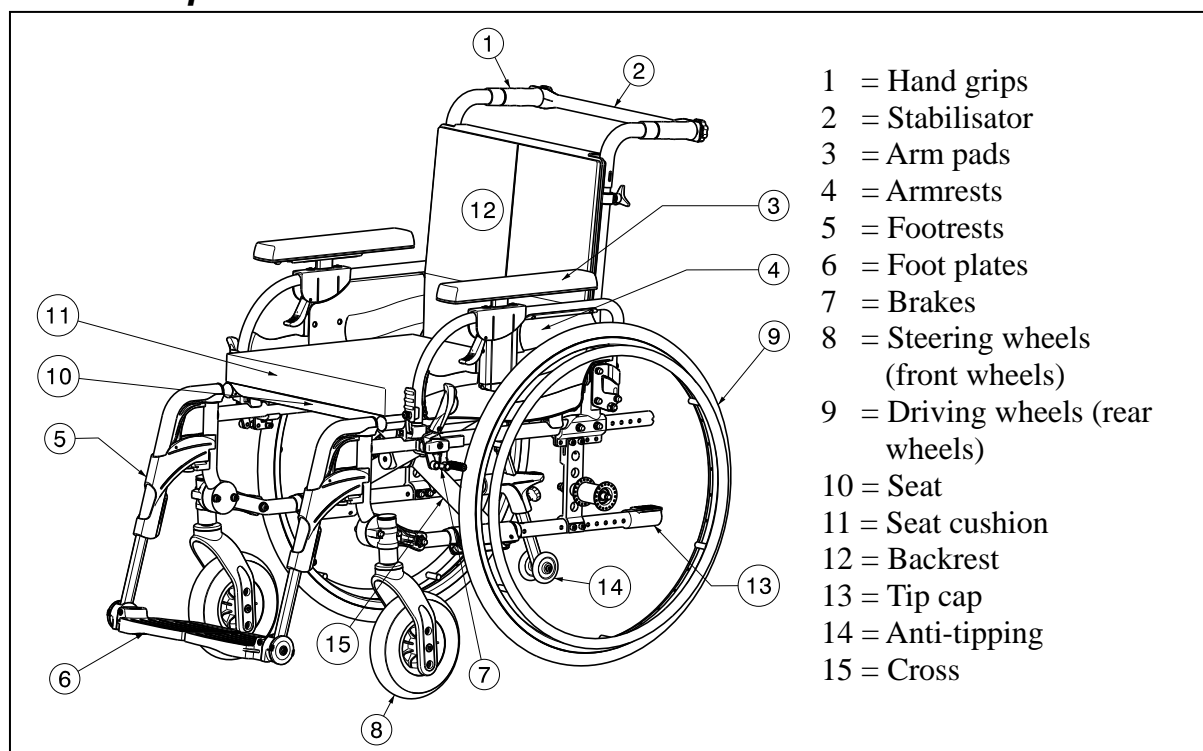
### 1.3 Components


## 1.4 Accessories

The following accessories are available for the V300DL:

- Anterior pelvic belt (B20) for mounting on the tubes of the back
- Tables (B15A, B12)
- Crutch holder
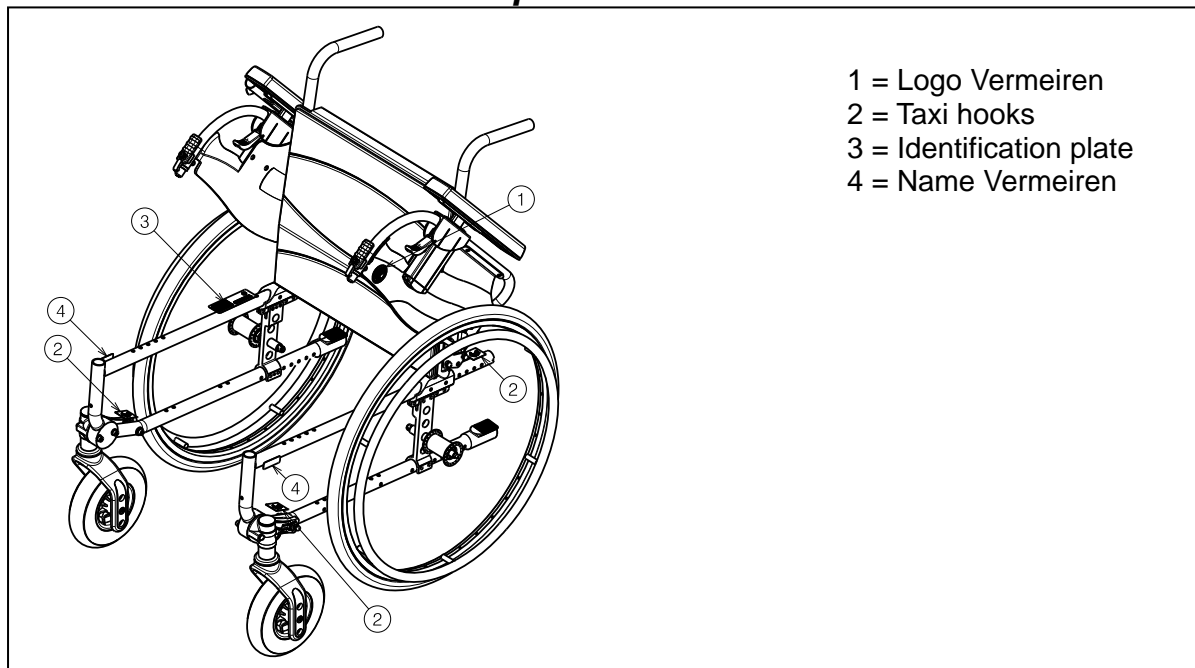
## 1.5 Location identification plate

## 1.6 Explanation of symbols

Maximum mass

Indoor and outdoor use

Maximum Safe Slope

CE conformity

Type designation

## 1.8 Safety rules

- ⚠ To prevent injury and/or damage to your wheelchair, make sure that no objects and/or body parts are caught in the spokes of the driving wheels.
- ⚠ The parking brakes should be applied before getting into and out of the wheelchair.
- ⚠ When getting into and out of the wheelchair, do not stand on the foot plates. These should be folded up beforehand, or the footrest should be swung completely outwards out of the way.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the wheelchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles. Obtain support from an attendant.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the wheelchair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the wheelchair.
- ⚠ Only use your wheelchair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ Stairs may only be negotiated when aided by another person. If furnishings like drive-on, ramps or lifts are available, use them.
- ⚠ When moving on the public highway, you are subject to the rules of the road.
- ⚠ When driving your wheelchair, you should not be under the influence of alcohol or medicine as in the case of driving other vehicles. This also applies to indoor driving.
- ⚠ When travelling outdoors, adapt your driving to weather and traffic conditions.
- ⚠ To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the sides and rear of the wheelchair are clearly visible.
- ⚠ Never exceed the maximum load of 130 kg.

## 2 Use

This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

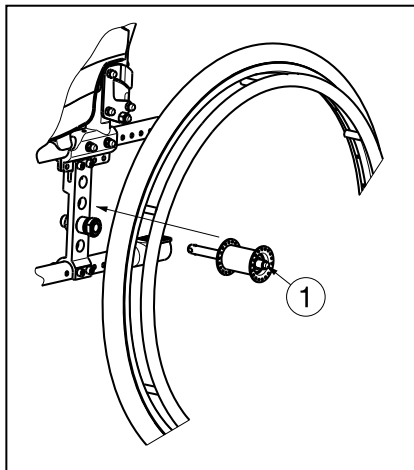
The wheelchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the wheelchair are given in § 3.

### 2.1 Carrying the wheelchair

The best way to carry the wheelchair is to make use of the wheels and roll the wheelchair.

If this is not possible (e.g. when the rear wheels are taken off for transportation in a car), firmly grasp the frame on the front and the hand grips. Do not use the foot or armrests or the wheels to grasp the wheelchair.

## 2.2 Mounting the rear wheels

1. Take the rear wheel and push on button ①.
2. Keep the button pushed in and mount the rear wheels axle till it stops.
3. Release the button.
4. Check that the wheels are secured.

## 2.3 Unfolding the wheelchair

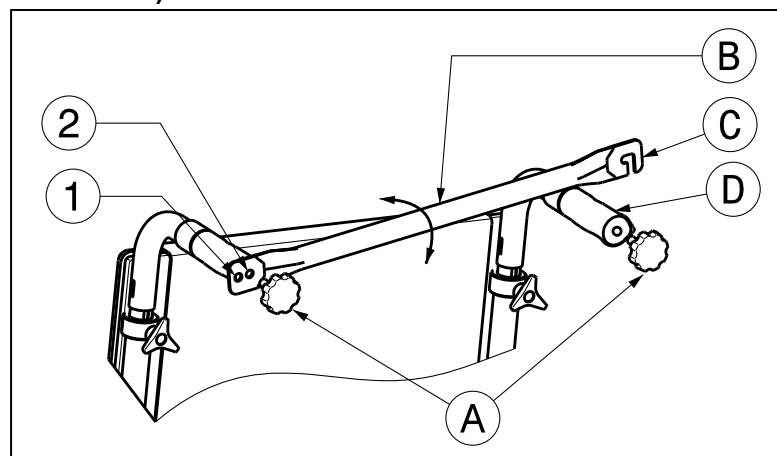
**⚠ CAUTION:** Risk of clamping – Keep fingers away from moving parts of the wheelchair.

1. Position yourself behind the wheelchair.
2. Use the hand grips to open the wheelchair as much as possible.
3. Position yourself at the front of the wheelchair.
4. Push both seating tubes down till they are fixed in their position.
5. Mount the stabilisator to the right hand grip (standing behind the wheelchair).

## 2.4 Mounting or removing of stabilisator

**⚠ WARNING:** Risk of injury - Make sure that star knobs ① are tightened.

**⚠ WARNING:** Risk of injury - Always mount the stabilisator (→ to increase the stability of the backrest).


The mounting of the stabilisator ② is done as follows:

1. Mount the stabilisator ② with the star knob ① in the left hand grip.
2. Swing the stabilisator ② to the right.
3. Mount hook ③ over the right hand grip ④ as shown on the drawing. Use holes ① or ② to adjust the tension of the backrest.

To take off the stabilisator:

1. Loosen the star knob ① on the right hand grip.
2. Remove hook ③ from the right hand grip ④.

## 2.5 Mounting or removing of the footrests

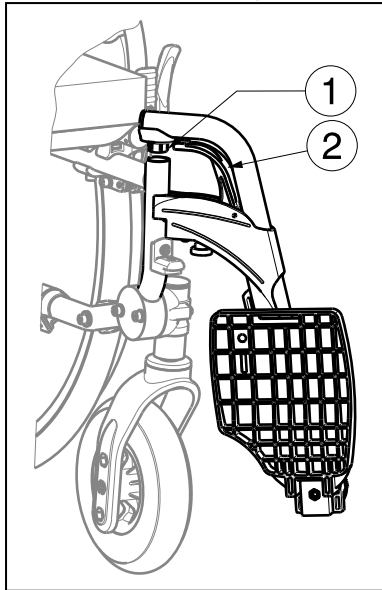The mounting of the footrests is done as follows:



1. Hold the footrest sideways at the outside of the wheelchairs frame and mount the tube hood ① into the frame.
2. Swing the footrest inwards till it clicks in position.
3. Swing the foot plate downwards.

To take off the footrests:

1. Pull lever ②.
2. Swing the footrest to the outside of the wheelchair till it comes loose from the guidance.
3. Pull the footrest from tube hood ①.

## 2.6 Operating the brakes

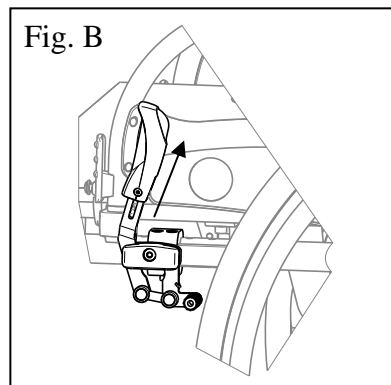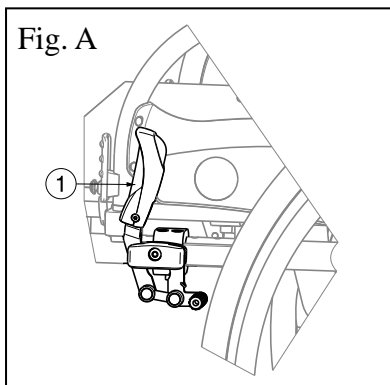- ⚠ **WARNING:** The brakes are not used to slow down the wheelchair during movements – Use the brake only to prevent the wheelchair from unintended movements.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.

To reduce the braking force it is possible to extend the brake lever (Fig B).

To apply the brakes (Fig A):

1. Push the brake levers ① forward till you feel a distinctive click.

- ⚠ **CAUTION:** Risk of unintended movement – Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.


To release the brakes:

1. Release one brake by pulling the lever ① backwards.
2. Hold the hand-rim of the released wheel with your hand.
3. Release the second brake by pulling the lever ① backwards.


## 2.7 Mounting or removing of armrests

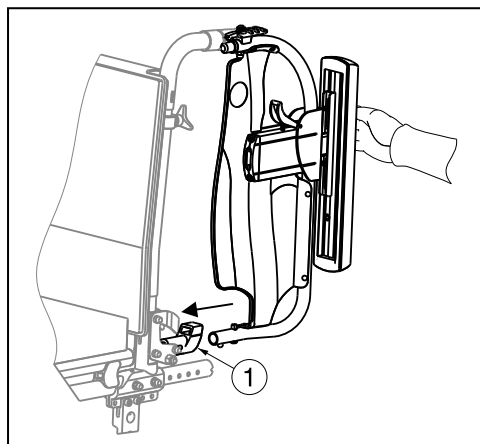The armrests of the wheelchair can be mounted by following instructions.

**⚠ CAUTION: Risk of clamping – Keep fingers, buckles and clothes away from the bottom side of the armrest.**

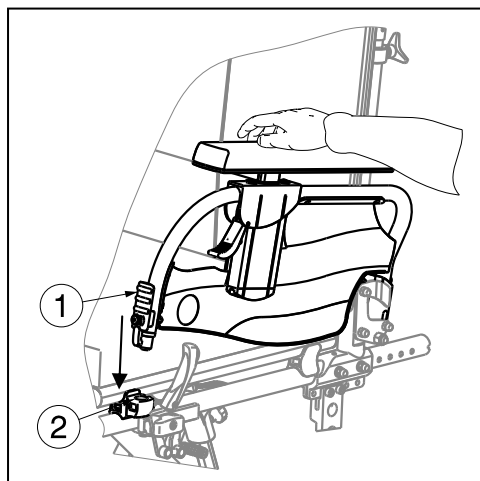
1. Mount the rear tube of the armrest in tube hood ①.
2. Make sure the armrest clicks in the locking mechanism.
3. Fold the armrest forward.

4. Click the front tube of the armrest in tube hood ②.

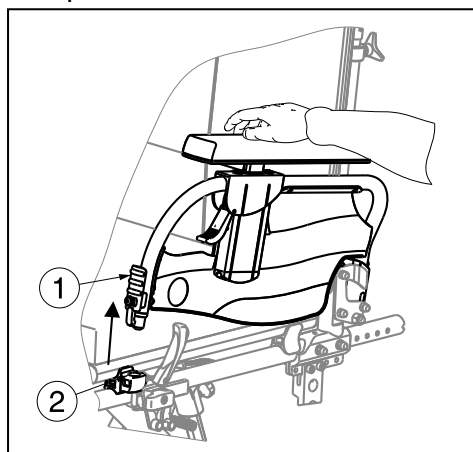
To open and remove the armrest:

1. Press lever ① and pull the front of the armrest upwards.
2. Fold the armrest backwards.

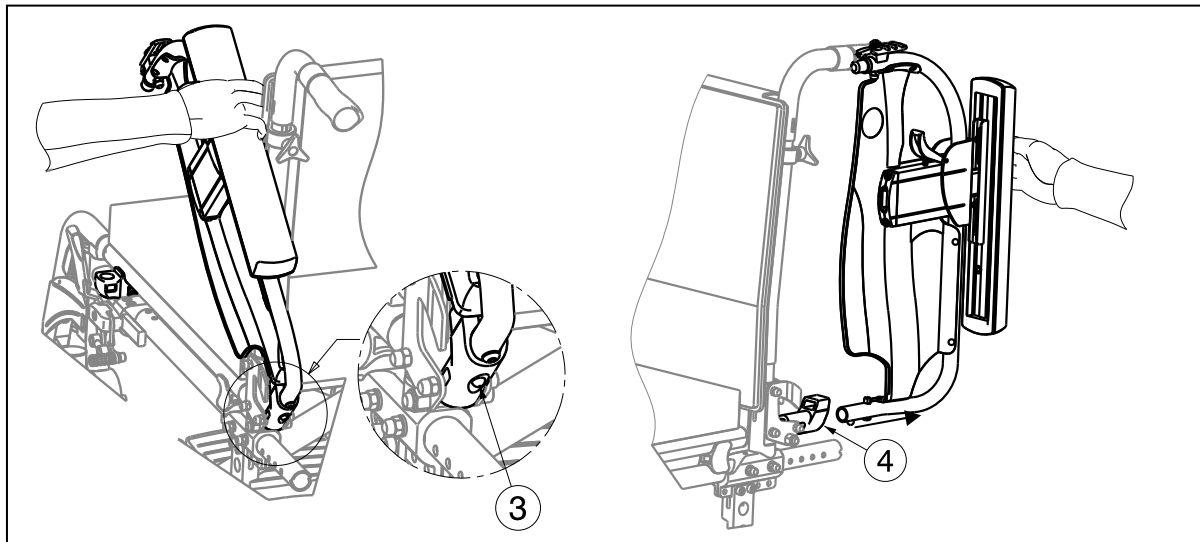
- To remove the armrest, press button ③ and pull the rear of the armrest from tube hood ④.


## 2.8 Adjusting the armrest

The height of the armrest can be adjusted in 8 positions (range 105 mm: steps 15 mm):

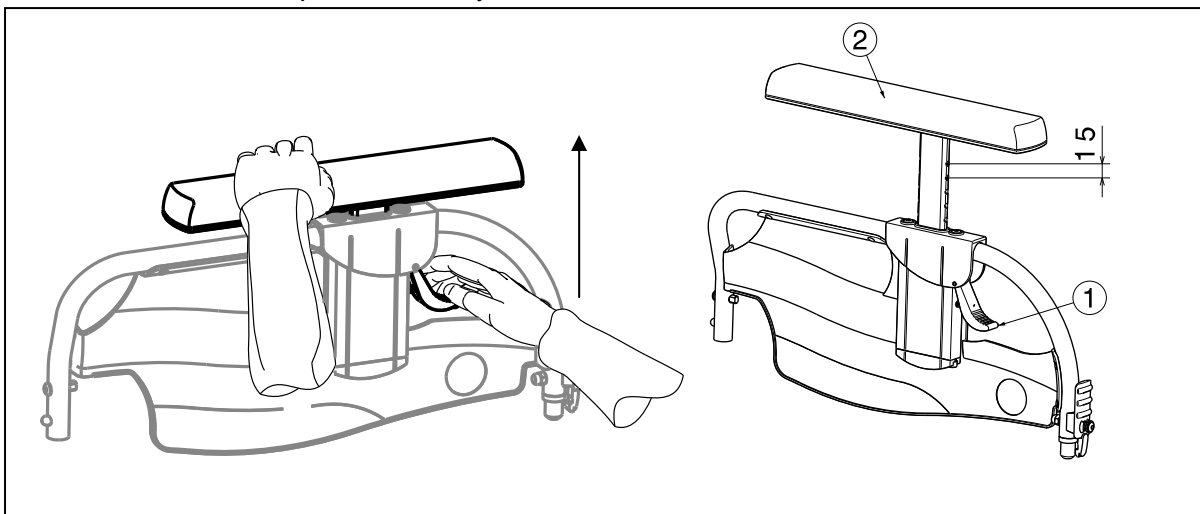- Push the lever ①.
- Adjust the arm pad ② in the desired height.
- Loosen the lever ①.
- Check the arm pad ② is firmly secured.


## 2.9 Adjusting anti-tipping (only mounted to the right side)

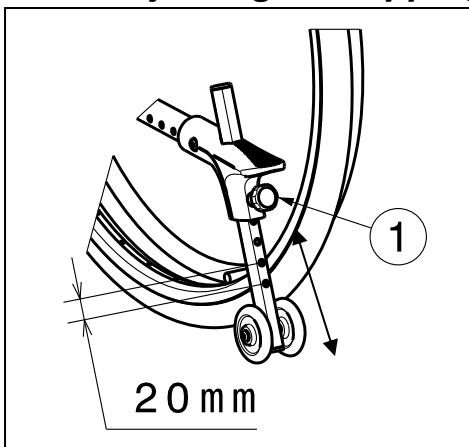Adjust the height of the anti-tipping in 7 positions (range 120 mm: steps 20 mm):

- Loosen the star knob ① a little bit.
- Move the tube of the anti-tipping in the desired height.
- Secure the star knob ①.
- Check the anti-tipping is firmly secured.

## 2.10 Adjusting height hand grips

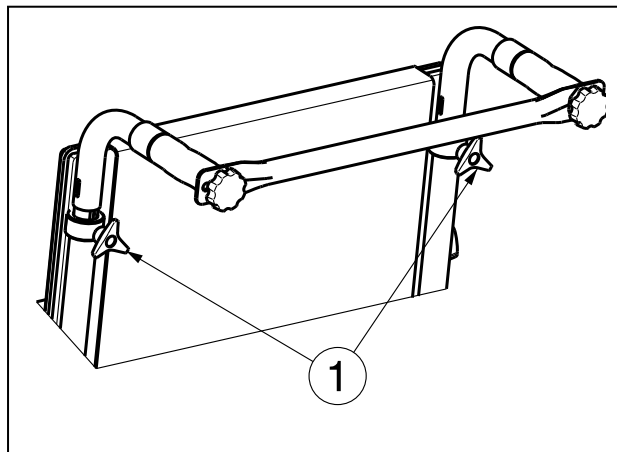
Adjust the height of the hand grips (range 120 mm: steps less):

1. Loosen the star knob ① a little bit.
2. Move the hand grips in the desired height.
3. Secure the star knob ①.
4. Check the hand grips are firmly secured.

## 2.11 Transfer in and out the wheelchair

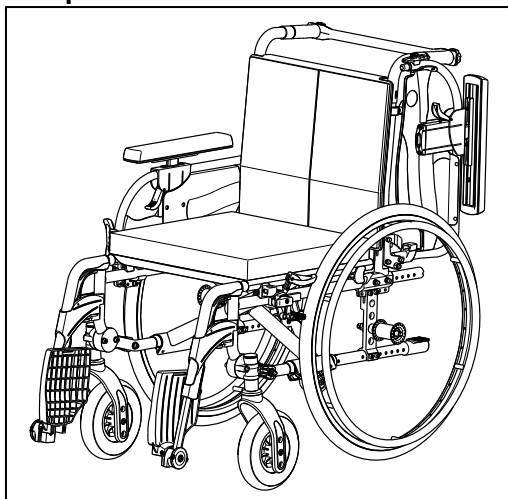- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- ⚠ **CAUTION:** Risk of tipping over of the wheelchair – Do not stand on the foot plates.

1. Position the wheelchair as close as possible to the chair, couch or bed to/from you wish to transfer.
2. Check both brakes from the wheelchair are in the on position.
3. Fold the foot plates upwards to prevent standing on them.
4. If the transfer is on the side of the wheelchair, fold the armrest on that side upwards (see § 2.7) and fold the brake lever down.
5. Transfer to/from the wheelchair.

## 2.12 Correct position in the wheelchair

Some recommendations for a comfortable use of the wheelchair:

- Position your backside as close as possible to the back rest.
- Make sure your upper legs are horizontal – If needed adjust the length of the footrests (see § 3.6).

## 2.13 Riding the wheelchair

- ⚠ **WARNING:** Risk of clamping – Prevent your fingers from being caught by the wheels spokes.
- ⚠ **WARNING:** Risk of clamping – Be careful passing through restricted passages (e.g. doors).
- ⚠ **WARNING:** Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.

1. Release the brakes.
2. Take both hand rims at their highest position.
3. Lean forward and push the hand rims forward until straight arms.
4. Swing your arms loosely back to the hand rims topside and repeat the movement.

## 2.14 Moving on slopes

- ⚠ **WARNING: Control your speed – Moving on slopes as slow as possible.**
- ⚠ **WARNING: Consider the capacities of your attendant – If your attendant does not have enough force to control the wheelchair, put on the brakes.**
- ⚠ **WARNING: Risk of tipping over – Lean forward to move your centre of gravity forward. To improve a better stability.**

1. If available on the wheelchair, wear the safety belt.
2. Do not attempt moving on too high slopes. The maximum slope angles (upwards and downwards) are mentioned in table 1.
3. Ask an attendant to help you moving on the slope.
4. Lean forward to move your centre of gravity forward.

## 2.15 Negotiating steps or curbs

### 2.15.1 Getting down steps or curbs

Riding down low curbs can be done moving forwards. Make sure that the footrests do not touch the ground.

A practiced user can negotiate small steps or curbs by himself:

- ⚠ **WARNING: Risk of tipping over – If you do not have enough experience with your wheelchair, ask assistance of an attendant.**
1. Bring balance on the rear wheels to reduce the pressure on the front wheels.
  2. Negotiate the curbs.

Higher curbs can be taken forward with an attendant:

1. Ask the attendant to tip the wheelchair slightly backwards.
2. Get past the curbs while moving on the rear wheels.
3. Put the wheelchair back on the four wheels.

An experienced user can negotiate higher curbs by himself. This is best done backwards.

1. Turn the wheelchair to have the rear wheels facing the curb.
2. Lean forward to move your centre of gravity forward.
3. Move the wheelchair close to the curbs.
4. Use the hand rim to roll-off wheelchair from the curb in a controlled manner.

## 2.15.2 Moving up steps or curbs

Moving up steps or curbs with attendant as follows:

1. Prevent the footrests from touching the curb.
2. Ask the attendant to tip the wheelchair backwards, just enough to move the front wheels over the curb.
3. Lean backwards to move your centre of gravity above the rear wheel.
4. Place the front wheels on the curb.
5. Roll rear wheels of the wheelchair over the curb.

Higher curbs are negotiated backwards:

1. Turn the wheelchair to have the rear wheels facing the curb.
2. Lean backwards and move your centre of gravity above the rear wheels.
3. Ask the attendant to pull the wheelchair on the curb.
4. Take back the normal position in the wheelchair.

An experienced user can negotiate curbs by himself:

**⚠ WARNING: Risk of tipping over – If you have not enough experience to control the wheelchair, get help from an attendant.**

1. Drive until the curbs.
2. Ensure that the footrest do not touch the curbs.
3. Lean backwards so you are balancing on the rear wheels.

4. Role the front wheels balancing over the curbs.
5. Bend forwards for more stability.
6. Role the rear wheels over the curbs.

## 2.15.3 Taking of stairs

Taking of stairs while you staying in the wheelchair shall be according following rules:

**⚠ WARNING: Risk of tipping over – Taking of stairs shall always with 2 attendants.**

1. Remove the footrests.
2. One attendant tip the wheelchair slightly backwards.
3. The second attendant take the front of the frame.
4. Stay calm, avoid sudden movements and keep your arms inside the wheelchair.
5. Take the steps on the rear wheels of the wheelchair.
6. Mounting the footrests back after taking the stair.

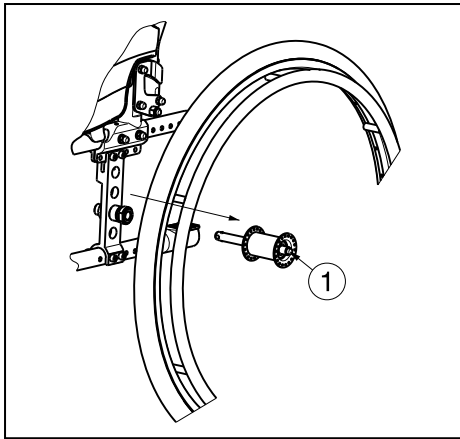
## 2.16 Fold up the wheelchair

**⚠ CAUTION:** Chance of pinching – Do not place your fingers between the components of the wheelchair.

1. Fold the foot plates or remove the footrests (see § 2.5).
2. Swing the stabilisator away from the right hand grip (standing behind the wheelchair - see § 2.4).
3. Take the seat on the front side and backside and pull it up.

## 2.17 Taking off the rear wheels

To facilitate the transport off the wheelchair the rear wheels can be taken off:

1. Make sure the brakes standing in the off position.
2. Take the wheelchair to the side frame where you want to remove the wheel.
3. Press the button ① in the center of the wheel hub.
4. Pull the wheel away from the frame.

## 2.18 Transport in the car

**⚠ WARNING:** Risk of injury – See that the wheelchair is attached properly. So you can avoid injury from the passengers during collision or sudden braking.

**⚠ WARNING:** Risk of injury – Use for attaching the wheelchair and passenger **NEVER** the same seatbelt.

1. Remove footrests and accessories.
2. Store footrests and accessories safely.
3. If possible, fold the wheelchair and remove the wheels.
4. Place the wheelchair in the luggage place.
5. If the wheelchair and the passenger compartment is NOT separated, attach the frame of the wheelchair securely to the vehicle. You can use the available safety belts in the vehicle.

## 2.19 Use of the wheelchair as seat in a motor vehicle

- ⚠ **WARNING:** The wheelchair has passed the crash test of ISO 7176-19: 2008 and, as such, has been designed and tested for use only as forward-facing seat in a motor vehicle.
- ⚠ **WARNING:** The wheelchair's pelvic belt alone is not suited as an occupant restraint belt.

The wheelchair is tested using the four-point strap-tie system and a 3-point occupant-restraint system.

Whenever feasible, use the seat of the vehicle and store the wheelchair in the cargo area.

Steps to secure the wheelchair in a motor vehicle:

1. Check that the vehicle is equipped with a suitable wheelchair tie down and occupant-restraint system, conform ISO 10542.
2. Check that the components of the wheelchair tie down and occupant restraint system are not frayed, contaminated, damaged or broken.
3. If equipped with an adjustable seat and/or back tilt, make sure that the wheelchair user is sitting as upright as possible. If the user's condition prevents this, a risk assessment should be done to evaluate the user's safety during transit.
4. Remove all mounted accessories such as trays and respiratory equipment, and secure them in a safe place.
5. Position the wheelchair facing forward in the travelling direction, centrally between the tie-down rails mounted in the floor of the vehicle.
6. Make sure that the indicated zones around the wheelchair user are clear from rigid vehicle parts.

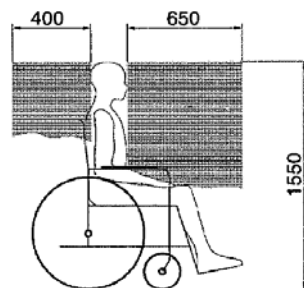
figure 1

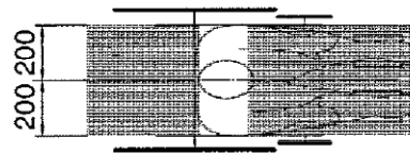
figure 2

7. Mount the front securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 3)  
This place is marked on the wheelchair with a symbol. (figure 4)
8. Roll back the wheelchair until the front straps are tight.
9. Apply the wheelchair brake.
10. Mount the back securement straps according to the instructions of the strap-system manufacturer at the indicated place. (figure 3)
11. This place is marked on the wheelchair with a symbol. (figure 4)


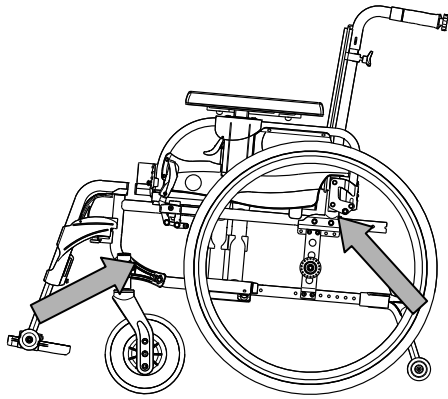
figure 3

Figure 4

Steps to secure the wheelchair user:

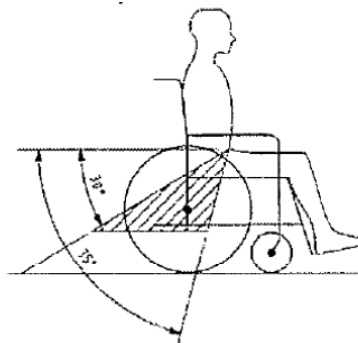
1. Remove both armrests.
2. If present, attach the wheelchair's pelvic belt.
3. Attach the occupant restraint belts according to the instructions of the strap-system manufacturer.
4. Wear the pelvic belt low across the front of the pelvis, so that the angle of the pelvic belt is within the preferred zone of 30° to 75° to the horizontal, similar to that shown below.

5. A steeper (greater) angle within the preferred zone is desirable.
6. Adjust the belt tightly according to the instructions of the strap-system manufacturer, consistent with the user's comfort.
7. Ensure that the restraint belt connects in a straight line to the anchor point in the vehicle and that no bends in the belt are visible, for instance at the axle of the rear wheel.
8. Install the armrests, if desired. make sure that belts are not twisted or held away from the body by wheelchair components such as armrests or wheels.

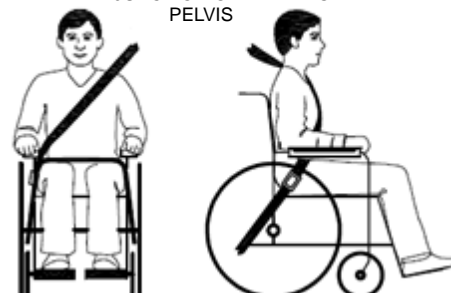PELVIC RESTRAINTS SHOULD MAKE FULL CONTACT ACROSS THE FRONT OF THE BODY NEAR THE JUNCTION OF THE THIGH AND PELVIS

### 3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren V300DL has been designed to be adjust with a minimum of replacements parts. There is no need for extra stock of spare parts.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

- ⚠ **WARNING: Risk of unsafe settings - Use only the settings described in this manual.**
- ⚠ **WARNING: Variation of allowed adjustments can still change the stability of your wheelchair (tilt back or sideways).**

#### 3.1 Tools

To set up the wheelchair the following tools are needed.

- Wrench set n° 10 to n° 22
- Allen keyset n° 4 to n° 6

#### 3.2 Manner of delivery

The Vermeiren V300DL shall be delivered with:

- 1 frame with armrests, rear and front wheels (standard delivery seat height: 500 mm without seat cushion, seat angle: 5°)
- Seat cushion
- Anti-tipping (only right side)
- 1 pair footrests
- Tools
- Manual
- Accessories

#### 3.3 Adjusting the seat height and seat angle

The Vermeiren V300DL is adjustable in 4 seat heights and 5 seat angles (0°-2,5°-5°-7,5°-10°) by changing the position of the wheels for each height and angle there is a different setting of the front and rear wheels.

The summary of the different seat heights by a standard seat angle 5° is tabulated below.


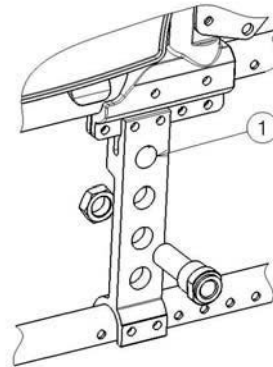
Seat height (without seat cushion)	Rear wheels		Front wheels
	Hole	Hole	Bushing ® relative to castor stem housing ©
440 mm	①	①	Above
470 mm	②	①	Under
500 mm (Standard)	③	②	Under
530 mm	④	③	Under

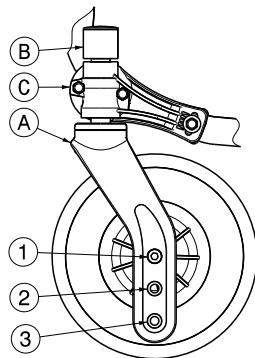Table 2: Seat heights by seat angle 5°



Front wheel



Axle plate rear wheels

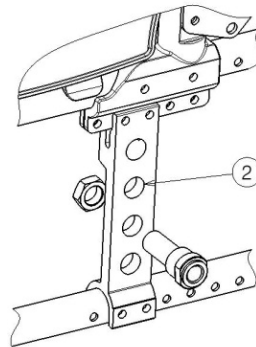
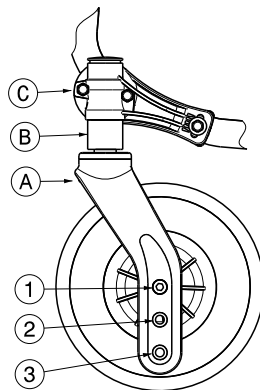
Seat height 440 mm

Seat height 470 mm

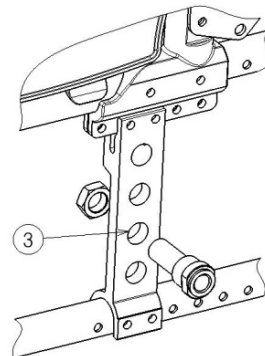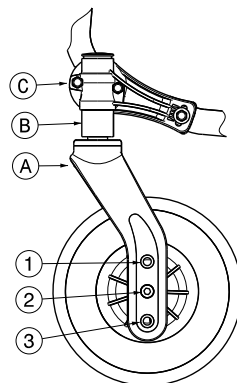Seat height 500 mm (standard)

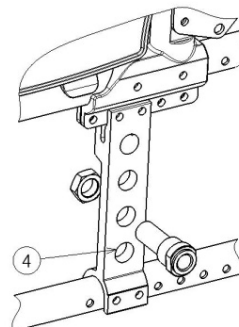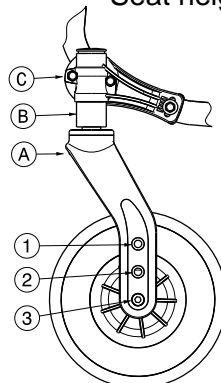Seat height 530 mm

Change the seat height according the following steps:

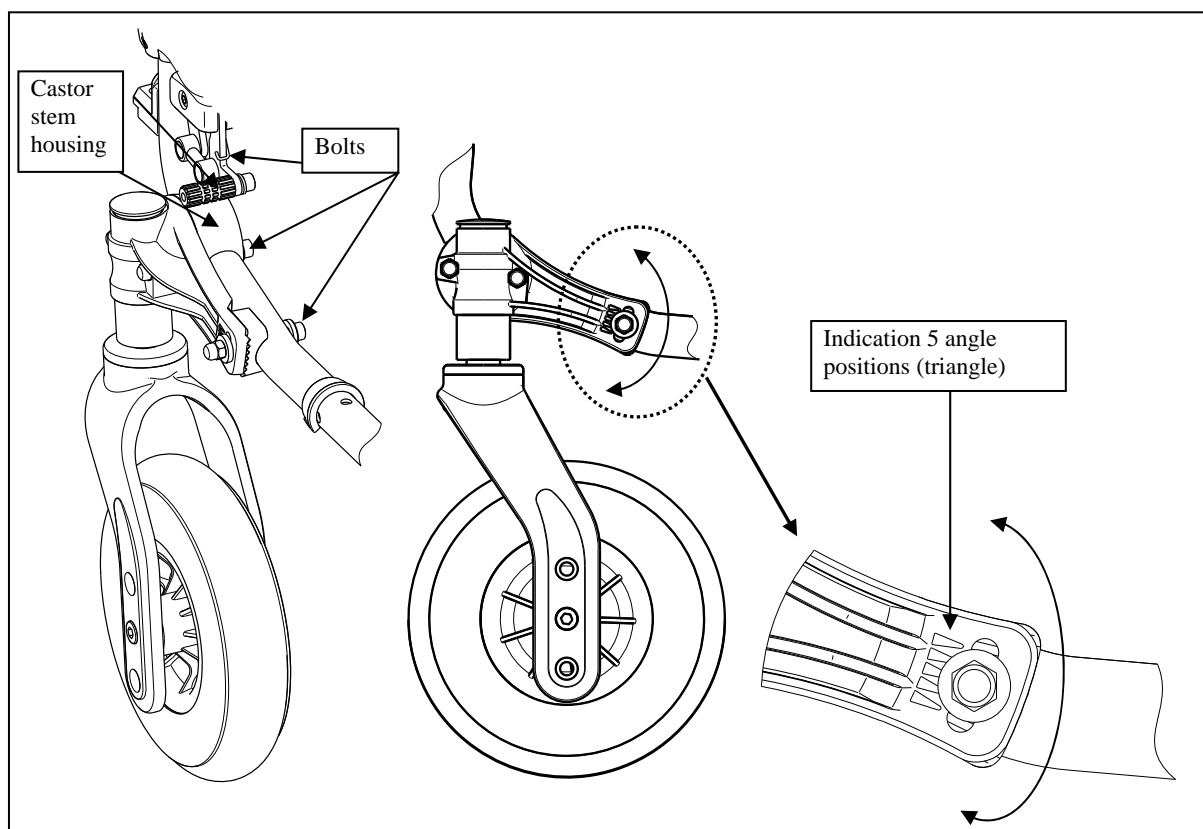
1. Remove the rear wheels (see § 2.17).
2. Screw the axle bushings of the rear wheels from the axle plate.
3. Assemble the axle bushings in the right hole of the axle plate, see Table 2 and previous figure.
4. Check if the axle bushings are fastened well.
5. Screw the swivel axles Ⓐ of the front wheels loose.
6. Place the front wheels in the right hole of the front fork (table 2).
7. Place the bushings according table 2.
8. Install the swivel axles Ⓐ of the front wheels and tighten them. Check the tension in the swivel axle Ⓐ so it turn smoothly, but there is no space for motion.
9. Mount the rear wheels.
10. If installed properly the swivel axles of the front wheels and the ground must be perpendicular. Check this.
11. Adjust the brakes according § 3.5.

For other seat angle and seat height combinations, the above procedure can be used to adjust the front fork and axle plate in the right combination.

If you do not find the right combination you can always contact the company Vermeiren.

Check that the swivel axles are perpendicular to the ground. If necessary use the procedure below to adjust the castor stem housing:

1. Loosen the 3 bolts using a size 5 Allen key.
2. Adjust the castor stem housing to the desired angle.
3. Tighten the 3 bolts.


### 3.4 Adjusting the seat depth

The Vermeiren V300DL is adjustable in 5 seat depths by changing the cross and the backrest connection ①.


Seat depth	Cross	Back rest connection ①
460 mm	Hole 1 and 3	Position 1
440 mm (Standard)	Hole 2 and 4	Position 1
500 mm	Hole 1 and 3	Position 2
480 mm	Hole 2 and 4	Position 2
520 mm	Hole 1 and 3	Position 3

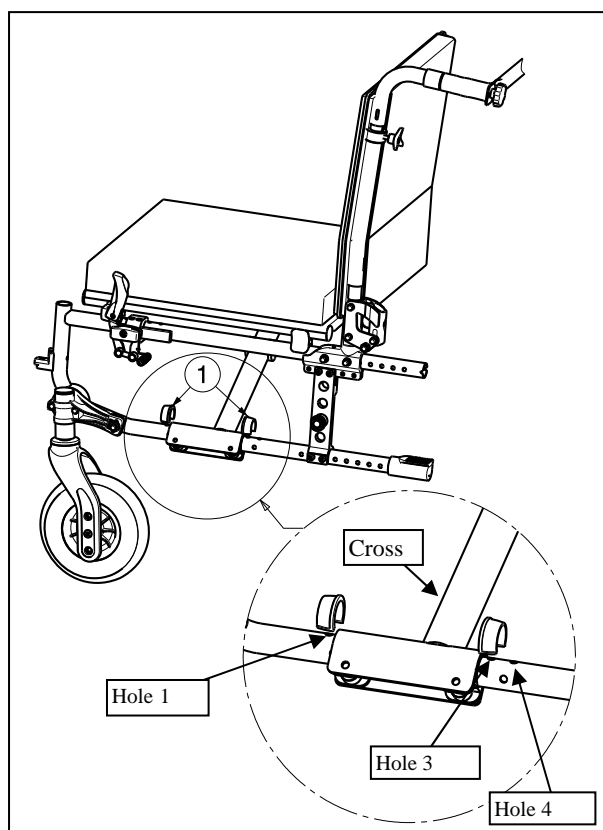Table 3: Seat depth

The seat depth can be changed by the cross according the following steps:

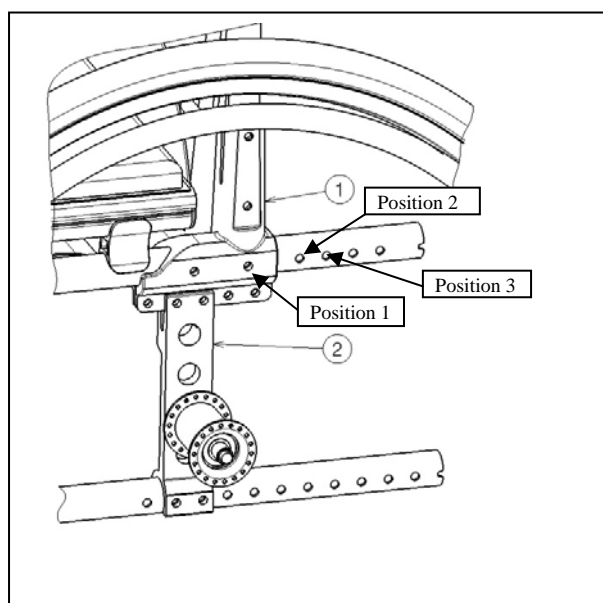1. Remove both clips ①.
2. Move the cross in the desired position (see table 3).
3. Replace both clips ①.


The seat depth can be changed by the back rest connection ① according the following steps:

1. Remove the bolts.
2. Move the back rest connection ① in the desired position (see table 3).
3. Retighten the bolts.
4. Check the seat height and seat angle.


### 3.5 Adjust the brakes

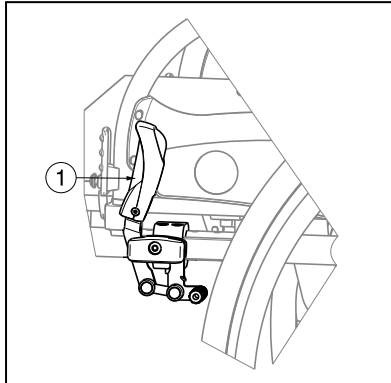**⚠ WARNING: Risk of injury – Brakes may only be adjusted by your specialist dealer.**

Adjust the brakes according following rules:

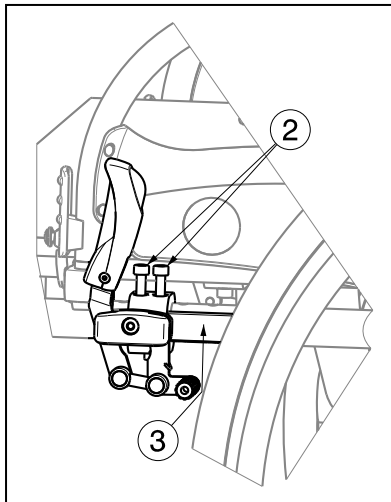1. Install the wheels according § 2.2.
2. Disconnect the brakes by pulling lever ① backwards.


3. Loosen the bolts ② so the brake mechanism can slide over the guide ③.
4. Pull the brake mechanism over the guide ③ to the desired position.
5. Retighten the bolts ②.
6. Check working of the brakes.
7. If necessary repeat the above steps until the brakes are adjusted well.

### 3.6 Adjusting of the footrests
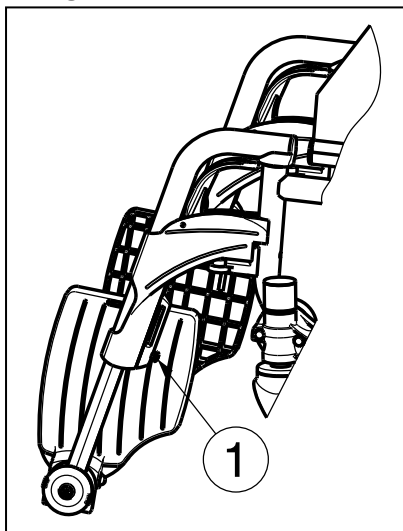
#### 3.6.1 Length of the footrests

**⚠ CAUTION: Risk of damage – Avoid that the footrests make any contact with the ground. Keep a minimum distance from 60 mm above the ground.**

Adjust the length of the footrests as follow:


1. Remove the screw ① (on the back of the foot rests).
2. Adjust the length of the footrest to a comfortable length.
3. Tighten the screw ① properly.

### 3.6.2 Adjusting the foot plates

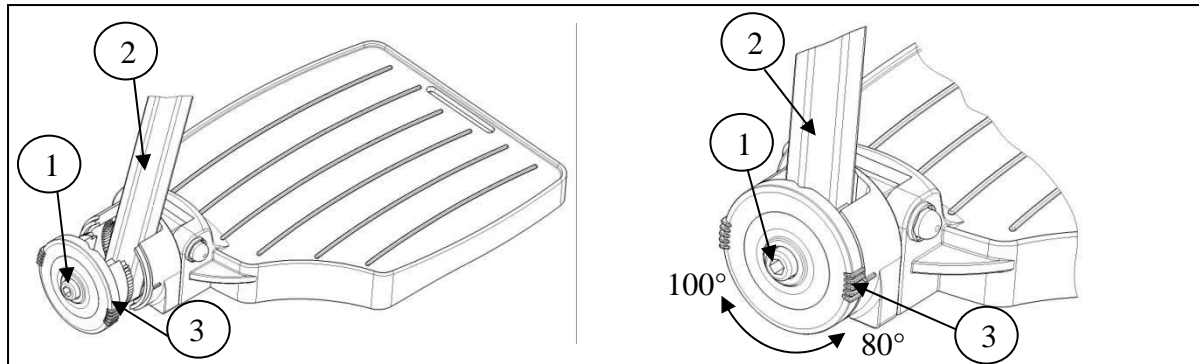The foot plates can be used in 2 positions. Standard they are adjusted with the base plate in the back position.

The foot plates can be adjusted to the front by switching the left and right foot plate.


### 3.6.3 Adjusting angle foot plates

Adjust the angle of the foot plates as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (80°-85°-90°-95°-100°) is according the dashes ③.
3. Retighten the bolt ①.

### 3.7 Adjusting the depth arm pad

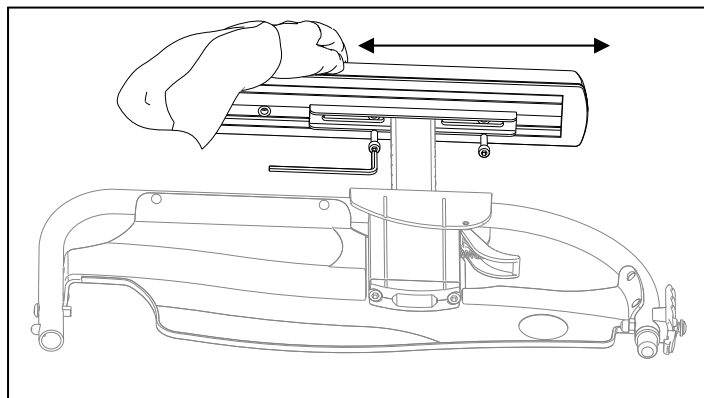
Adjust the depth of the arm pad as follow (range 120 mm: step less):

1. Remove or loosen both screws under the arm pad.
2. Move the arm pad to the front or rear in the desired depth with the grooves (42,5 mm) or mount the arm pad in the other holes (3 positions steps from 42,5 mm).
3. Remount both screws.

### 3.8 *Adjusting the stability and the maneuverability*

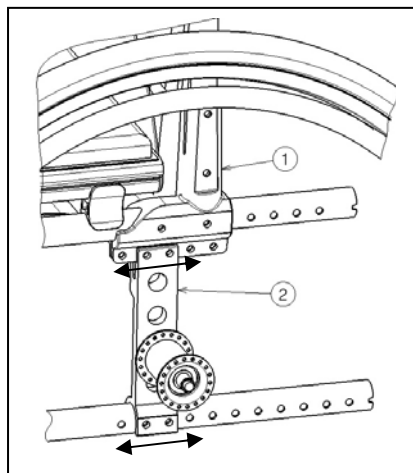The stability and maneuverability can be adjusted by moving the axle plate of the rear wheels ②.

Adjust the stability and maneuverability according following steps:

1. Loosen the bolts of axle plate.
2. Move the axle plate ② in the desired position.
3. Retighten the bolts.

## 4 Maintenance

For the maintenance manual of the wheelchairs refer to the Vermeiren website:  
[www.vermeiren.com](http://www.vermeiren.com).

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

Vermeiren GROUP  
Vermeirenplein 1 / 15  
B-2920 Kalmthout  
BE

website: [www.vermeiren.com](http://www.vermeiren.com)

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### **All rights reserved, including translation.**