



INSTRUCTIONS MANUAL EN


IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.
 User: if you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage to the product.
 Distributor: this leaflet must be given to the user of the product.

PRODUCT DESTINATION

This toilet armrests are technical products intended for professional use.

PRODUCT DESCRIPTION

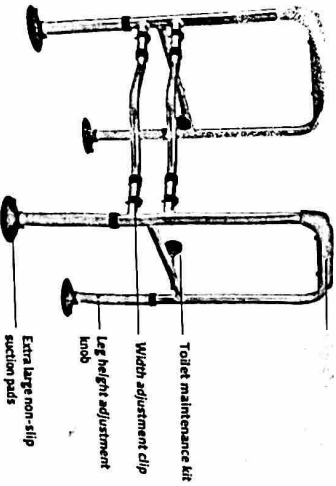


PRECAUTIONS FOR USE

Failure to observe the precautions for use may result in the risk of serious injury.


- Do not exceed the maximum weight limit of 125 kg.
- Use only accessories and parts supplied by the manufacturer.
- Visually inspect the general condition of the chassis.
- Make sure that the toilet armrest is correctly extended and that all legs are adjusted to the same height.
- Make sure that all parts are securely fastened and in good working order.
- Do not make any adjustments to the toilet armrest while it is in use.
- Do not use the toilet armrest on stairs or escalators.
- Make sure that you put your hands on each handle so that the toilet armrest can be in a parallel force position.
- Only use the toilet armrest as a step support.
- Do not use the toilet armrest to walk backwards.
- Always lean on the toilet armrest. Serious injury to the user and/or damage to the walking frame and/or wheels can result from improper use.
- In the event of damaged, broken or worn parts, contact the distributor.

ASSEMBLY



1. Remove all items from the packaging and make sure all components are present (see photo below).

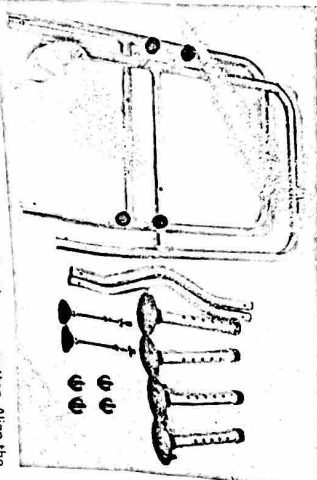


2. Insert the feet into the frame by pressing the pins down. Align the pin and the hole until the pin comes out completely. Adjust the height of the legs, making sure that all 4 legs are at the same height.

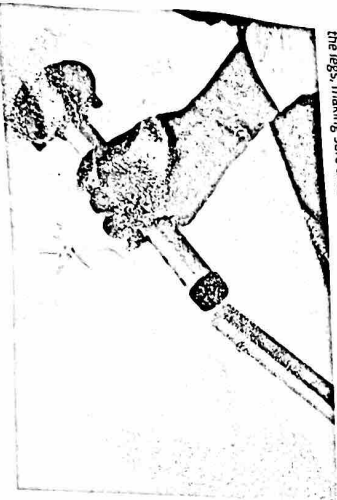

3. Assemble the side frames using the two tubes. Fasten the tubes with the side frames using the clips. You can adjust the width of the product to your convenience.

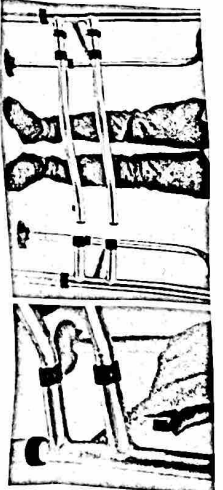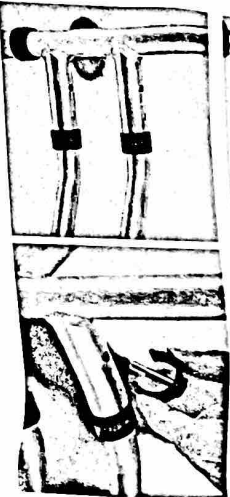

4. Place the Bermuda Toilet Armrest over the toilet. Insert the toilet support kit on each side. Adjust and secure the kit with the bolts. Make sure the kit is tight around the toilet.

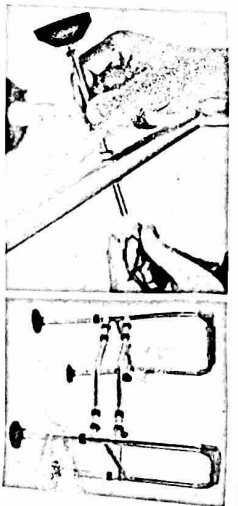

USE

To use the armrest, place your forearms on the forearm rests and your hands on the handles. Adjust the height if necessary. Move forward slowly.

CARE AND MAINTENANCE

Failure to care for and maintain your rollator can result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry cloth to avoid oxidation.
- Make sure there is no accumulation of dirt or debris around the tips.
- Do not use cleaning products containing phenol or chlorine as this could damage aluminium and polyamide materials.
- Periodically check the wear and strength of all screws, nuts and other components for wear and tear.
- In case of malfunction, do not repair the product yourself. Consult an authorised distributor.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the walking frame, remember to hand over all the necessary technical documents to the new user. The product must first be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away, please contact your specialist distributor. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

- Failure to comply with the storage conditions may lead to deterioration of the product and thus the risk of serious injury.
- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing and storing your product:
 - In a dry and temperate place
 - Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air)
 - Store all removed parts together in the same place to locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL FEATURES

Dimensions	Width 54,5/59,5 (between armrests 45/50,5) x depth 44,5 x height 69/82 cm in 6 positions
Height adjustable	From 73 to 90 cm in 8 positions
Maximum load	125 kg

OPERATING TIME

The expected life of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the distributor from whom you purchased the product to apply the warranty.