INVENTUM

maakt 't moment

BLOOD PRESSURE MONITOR

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BDA435

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Shenzhen AOJ Medical Technology Co., Ltd.

Room 301&4F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiaweiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, CHINA

EC REP Share Info GmbH

Address: Heerdter Lohweg 83, 40549 Düsseldorf, GERMANY

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English

1	safety instructions	page 26
2	appliance description	page 30
3	things to know about blood pressure	page 31
4	before first use	page 32
5	taking a measurement	page 34
6	memory function	page 37
7	cleaning & maintenance	page 38
8	common Q & A on blood pressure	page 39
9	troubleshooting	page 40
10	specifications	page 42
11	appendix 1 EMC Information	page 44
	<i>General terms and conditions of service and warranty</i>	page 50

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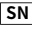
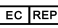
1 safety instructions

The warnings and illustrations shown in the manual are intended to enable you to use the product safely and correctly, thus preventing harm to you and others. The description of the symbols can be found below:

Legend, mark and meaning	
	Warning information, refer to the attached document
	BF-type anti –shock degree for the application part
	Comply with local regulations
	Consult the instructions for use
	Keep dry
	Low voltage prompt
	Keep out of the sun
	Vertical upward
IP21	The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects.
RoHS	RoHS mark

	CE mark
	Manufacturer
	Date of manufacture
	Serial number
	Lot number
	EU authorized representative

ATTENTION

- **Please read these instructions before operating the appliance and retain for future use.**
- No maintenance or servicing when using.
- The patient is an intended operator, and all the functions can be safely used.
- Do not leave the small parts where children can reach them. Children may swallow them. If a child accidentally swallows them, please contact a doctor immediately.
- Maintenance should be done by the operator as suggested in the user manual.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When the ambient temperature is less than 5°C, please take the device to a place where the ambient temperature is between 5°C~40°C for at least 1 hour; When the ambient temperature is higher than 40°C, please take the device to a place where the ambient temperature is between 5°C~40°C for at least 2 hours.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT kink the air tube during use, otherwise, the cuff pressure may continuously increase which can prevent blood flow and result in harmful injury.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.
- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.
- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.
- Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.
- Consult with your physician before using this monitor on an arm where intravascular access or therapy, or an arteriovenous (A-V) shunt, is present because of temporary interference to blood flow which could result in injury.
- Consult with your physician before using this monitor if you have had a mastectomy.

- Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders, as cuff inflation can cause bruising.
- DO NOT take measurements more often than necessary because bruising, due to blood flow interference, may occur.
- ONLY inflate the arm cuff when it is applied on your upper arm.
- Remove the arm cuff if it does not start deflating during a measurement.
- DO NOT use this monitor for any purpose other than measuring blood pressure.
- DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.
- DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor.
- DO NOT use this monitor in a moving vehicle, such as in a car.
- DO NOT drop or subject this monitor to strong shocks or vibrations.
- DO NOT use this monitor in places with high or low humidity or high or low temperatures.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.


Dispose of the monitor when its service life is reached. Follow local regulations regarding the disposal of such product.


- When the performance changes (such as: inaccurate measurement or abnormal display), please stop using it immediately and contact the after-sales service personnel in time.

2 appliance description

1. Display
2. Memory button
3. Start/ Stop button
4. User selection/ Date/ Time setting button
5. Air tube connection
6. Cuff
7. Air plug
8. Air tube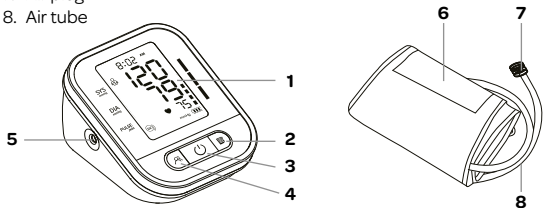

1. Date and Time
2. User Icon
3. Inflation Icon
4. Deflation Icon
5. Memory Icon
6. Detection for Cuff Wrapping Icon
7. Memory Number
8. Indication for Wrong Operation
9. Heartbeat Icon
10. Average Value
11. Irregular Heartbeat Icon
12. Unit of Pressure
13. Low Battery
14. Pulse Rate
15. Diastolic Blood Pressure
16. Blood Pressure Classification
17. Systolic Blood Pressure

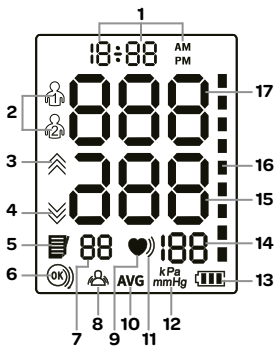3

things to know about blood pressure

What are systolic pressure and diastolic pressure?

When ventricles contract and pump blood out of the heart, blood pressure reaches its maximum value, the highest pressure in the cycle is known as systolic pressure. When the heart relaxes between heartbeats, the lowest blood pressure is diastolic pressure.

CAUTION:

Only a physician can tell your normal blood pressure range. Please contact a physician if your measuring result falls out of the range. Please note that only a physician can tell whether your blood pressure value has reached a dangerous point.

Category	Systolic (mmHg)	Diastolic (mmHg)
Optimal	≤ 120	≤ 60
Normal	≤ 130	≤ 85
High normal	130 - 139	85 - 89
Grade I hypertension	140 - 159	90 - 99
Grade II hypertension	≥ 160	≥ 100

Irregular heartbeat detector

An irregular heartbeat is detected when a heartbeat rhythm varies while the device is measuring systolic pressure and diastolic pressure. During each measurement, the blood pressure monitor will keep a record of all the pulse intervals and calculate the average value of them. If there are two or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 25\%$, or there are four or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 15\%$, then the irregular heartbeat symbol will appear on the display with the measurement result.

**CAUTION:**


The appearance of the Irregular Heartbeat icon indicates that a pulse irregularity consistent with an irregular heartbeat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol often appears, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

4 before the first use

Before you use the appliance for the first time, please do as follows: carefully unpack the appliance and remove all the packaging material. Keep the material (plastic bags and cardboard) out of reach of children. Check the appliance after unpacking for any damage caused by the transportation.

Install the batteries

1. Open the battery cover.
2. Install the batteries by matching the correct polarity, as shown. Always use the correct battery type (4 x AAA batteries).
3. Replace the cover.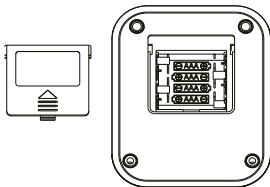

Replace the batteries whenever the below happens:

- the symbol  appears in the display
- the display dims
- the display does not light up

**CAUTION:**


- Remove batteries if the device is not likely to be used for some time.
- The old batteries are harmful to the environment; do not dispose with other daily trash.

- Remove the old batteries from the device and follow your local recycling guidelines.
- Do not dispose of batteries in fire. Batteries may explode or leak.
- Do not use new and used batteries together.
- Do not use different types of batteries together.

FUNCTION SETTING

User mode
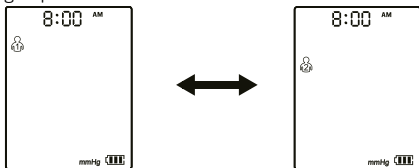
In standby mode, press the  button to enter the user group selection interface. Then press the  button again to switch and select user groups.


Date and time setting

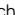
It is important to set the date and the time before using your blood pressure monitor, so that a time stamp can be assigned to each record that is stored in the memory (year: 23 –99, time format: 12 H).


In standby mode, press the  button for about 3 seconds to enter the date setting, and “year” will flash. Press the  button to adjust to the desired year, and then press the  button to confirm the selection.

When the “year” is set, it will automatically enter the month setting. At this time, the “month” icon will flash. You can switch to the desired value by pressing the  button. Follow the same step to set the date, hour and minute.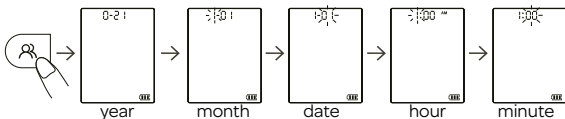

Unit display setting

In standby mode, keep pressing the  button for about 10 seconds to enter the unit selection, and press  to switch mmHg/kPa. Press  to confirm the selection. The default unit is mmHg.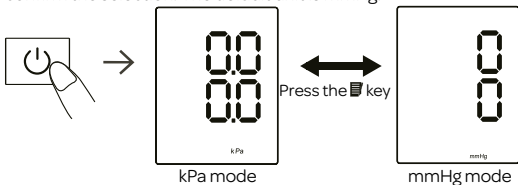


5 taking a measurement

Preparation before taking a measurement


- Remove the clothing on the arm.
- Always measure in the same arm (generally the left arm).
- Remain still and keep quiet during measurement.
- Relax as much as possible and do not talk during measurement.
- Measure your blood pressure at about the same time every day.
- Do not measure right after physical exercise or a bath. Rest for 20 to 30 minutes before taking the measurement.
- Readings under the conditions listed below may affect results: Within an hour after dinner, after having wine, coffee, tea, sports; talking, being nervous, being in unsteady mood, bending forward, moving, room temperature dramatically changing during measuring; inside a moving vehicle, repeated and continuous measuring.

Applying the arm cuff

1. Connect the arm cuff to your monitor by inserting the air plug into the air tube connection securely.
2. Place your hand through the cuff loop. Pull the cuff until it reaches your upper left arm.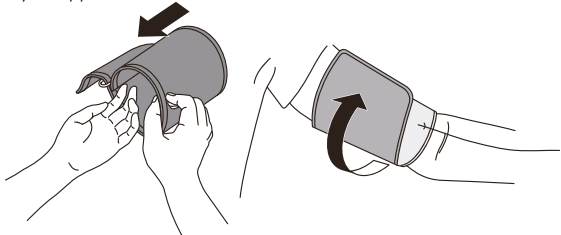

Note: the bottom edge of the arm cuff should be 2 to 3 cm (0.79-1.18 inches) above the inside elbow. The air tube is on the inside of your arm and aligned with your middle finger.

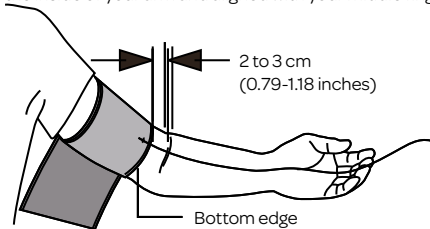
- Make sure that the air tube is positioned on the inside of your arm and wrap the cuff securely, so it can not move around your arm.

Note: repeated measurement will result in blood congestion in the arm, which will affect the measurement result. How to avoid blood congestion and ensure the repeated measurement is accurate? You can raise the left hand and hold the fist several times, or take off the cuff and rest for at least 2-3 minutes before taking the measurement.

Sitting correctly

To take a measurement, you need to be relaxed and comfortably seated in a room with a comfortable temperature.


- Place your arm on the table.
- Sit in a comfortable chair with your back and arm supported.
- Keep your feet flat and your legs uncrossed.
- The arm cuff should be placed on your arm at the same level as your heart, with the arm resting comfortably on a table.

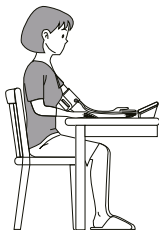Taking a measurement

Start the measurement after putting on the cuff:

- Press the  button and the monitor will begin to inflate. Make sure you do not move or talk during the measurement.

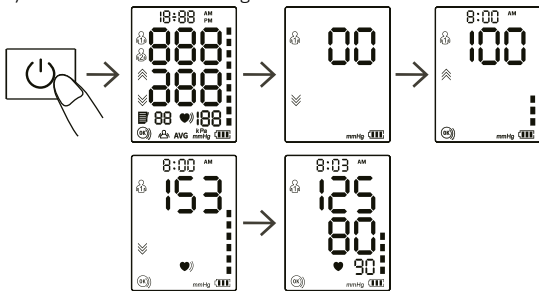Note: if you feel uncomfortable during the measurement, press the  button immediately to stop measure. When the air pressure is filled to a certain value, the value on the display will slowly drop at a certain speed, and the heartbeat symbol will flash. After the measurement is completed, the systolic pressure, diastolic pressure, and pulse measurements will be displayed on the screen.

Note: consult your doctor if unexpected readings are obtained.

Note: please use the original cuff that has passed the clinical trial! Any replacement of original component with the component not provided by the manufacturer might result in measurement error.

Self check for the cuff strap

The  icon is always displayed on the screen when the cuff is worn correctly. When the cuff is worn too loosely, the  icon will always flash to remind you. If the  icon is flashing all the time, please press the  button to stop the measurement.

Keep still indication

The  icon flashes when the body moves during the measurement, which may cause incorrect measurement results. If so, take another measurement.

6 memory function

This blood pressure monitor stores 99 sets of memories of two users. After each measurement, the monitor will automatically store the measurement data. When the memory is full, the old measurement data will be overwritten with the new one.

- In the standby mode, press the  button once and the device will display the average value of the blood pressure measured for the first three times.
- Press the  button again, the first memory will be displayed.
- Press the  button again and the rest of the memories will be displayed one by one.

Delete the memory

Keep pressing the  button for three seconds to delete the measurements stored on the device for user 1 or 2, depending on the selected user. When the memory is empty, the display shows .

To keep your blood pressure monitor in the best condition and protect the unit from damage, follow the directions listed below:

- Remove the batteries before cleaning.
- The body of the device can be cleaned with a clean, soft cloth. If necessary, you can use a slightly damp cloth.
- Do not fold the cuff to tightly.
- Do not bend or crease the air tube excessively.

**CAUTION**

- Do not use any abrasive or volatile cleaners.
- Never immerse the device in water or any other liquid to protect against electrical hazards and damage.
- Store the device and the components in a clean, safe location.
- Do not subject the monitor to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach, vibrations or shocks.
- Remove the batteries if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

Why is the blood pressure value obtained at home lower than that obtained at the hospital?

- The blood pressure difference between home and hospital measurements is about 20 mmHg - 30 mmHg (2.7 kPa - 4.0 kPa). This is because individuals tend to be more relaxed at home than at the hospital.
- In addition, when the device is placed at a position over the heart, the blood pressure value tends to be much lower than it actually is. Ensure the device is positioned right at the heart level.

Why is the blood pressure value obtained at home higher than that obtained at the hospital?

- The antihypertensive drug might have lost its efficacy. Kindly adhere to your doctor's instructions.
- The cuff might not be in the correct position. If the cuff is not placed right, no arterial pressure value will be obtained, and the blood pressure value might be much higher than it is. Therefore, properly position the cuff.
- The cuff is not tight enough. If the cuff is loose, the compression force might fail to transmit to the artery, causing the blood pressure value to be much higher than it is. Therefore, re-adjust and tighten the cuff further.
- The patient is not sitting correctly during the measurement. Slouching, tilting, bending, and sitting cross-legged are not encouraged while taking blood pressure measurements due to increased abdominal pressure or the arm position being below the heart. Kindly take readings in the correct posture.

When can I obtain better measurements?

- Measurements are best taken in the mornings right after you urinate or when your mind and body are stable. We recommend taking readings at the same time of the day, every time.

Anomaly	Inspection items	Countermeasures
There is no display after the power switch is pressed	Whether the battery power is insufficient	Replace the battery.
	Whether positive and negative poles of the battery are installed reversely	Install in the positive and negative pole direction of battery
No pressurizing	Whether the air tube plug is inserted tightly	Insert the air tube plug firmly into the connection
	Whether the air tube is broken or leaked?	Purchase a new cuff
Unable to measure due to display error	Whether the arm is moved when pressurization	Keep your arm and body still static
	Whether you talk during measurement	Keep quiet while measuring the blood pressure
Air leakage of cuff	Whether the cuff is twined too loose	Please tighten the cuff
	Whether the air bag of cuff is ruptured	Replace with new cuff
	If the blood pressure still cannot be measured normally by the above methods, please contact the service department. Please do not disassemble it by yourself!	

Abnormal phenomena and handling

When the measurement is abnormal, the following symbols will appear. Please use the correct method for measurement.

LCD display information	Remark
Er U	The inflation can not reach 30mmHg in 12 seconds
Er H	The inflation reaches 295mmHg, it deflates automatically after 20 ms.
Er 1	The pulse rate is not detected correctly.
Er 2	Too much disturbance (movement, talk or magnetic disturbance during a measurement).
Er 3	The test result is abnormal.
Er 23	SYS value reading is lower than 57mmHg.
Er 24	SYS value reading is higher than 255mmHg.
Er 25	DIA value reading is lower than 25mmHg.
Er 26	DIA value reading is higher than 195mmHg.

10 specifications

Product name	Arm Blood Pressure Monitor	
Display mode	LCD display 50 mm x 65 mm	
Measuring method	Oscillometric measurement	
Measuring part	Upper arm	
Measuring range	Pressure value	SYS: 57-255 mmHg; DIA: 25-195 mmHg.
	Pulse rate	40-199 bpm
Accuracy	Pressure value	±3 mmHg
	Pulse rate	±5%
Low battery	4.2V±0.1V: low battery; <4.0V±0.1V: item will be turned off	
Auto power-off	1 minute without operation	
Power source	4 x AAA DC6V	
Anti Electronic Shock Degree	Type BF	
Operation mode	Continuous operation	
Protection against harmful ingress of water or particular matter	IP21	
Weight	About 230g (without batteries)	
Dimensions	118mm x 98mm x 61mm (l x w x h)	
Cuff size	22-42 cm (8.66-16.53 inches)	
Service life	5 years	

Protection against electric shock	Internally powered ME equipment (when using only batteries)		
Operating environment	Condition		beyond the designated temperature and humidity range, it will not be used properly
	Humidity condition	15%-90%RH	
	Atmospheric condition	70kPa-106kPa	
Transportation and storage environment	Avoid strong impact, direct impact, exposure or rain during transportation. The packaged Blood Pressure Monitor shall be stored indoors at the temperature of -20°C-55°C and the relative humidity of 10%-93%, atmospheric Condition: 70kPa-106kPa. without corrosive gas and with good ventilation.		

11 appendix 1 EMC Information

Guidance and manufacturer's declaration - Electromagnetic emission

The arm blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the arm blood pressure monitor should assure that it is used in such an environment.

Emissions	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The arm blood pressure monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The arm blood pressure monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC61000-3-3	N/A	

Guidance and manufacturer's declaration - Electromagnetic immunity

The arm blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of arm blood pressure monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8kV contact ±2 kV, ±4kV, ±8 kV, ±15kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency Magnetic field IEC 61000-4-8	30A/m, 50/60Hz	30A/m, 50/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz

NOTE: UT is the a.c. mains voltage prior to application of the test level

Guidance and manufacturer's declaration - electromagnetic Immunity

The arm blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the arm blood pressure monitor should assure that it is used in such an environment.

Radiated RF IEC6100-0-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Max. Power (W)	Distance (m)	IEC 60601-1-2 Test level (V/m)	Compliance level (V/m)
	385	380 - 390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27	27
	450	430 - 470	GMRS 460, FRS 460	FM \pm 5 kHz deviation 1 kHz sine	2	0,3	28	28
	710	704 - 787	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0,3	9	9
	745							
	780							
	810	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0,3	28	28
	870							
	930							

	1720	1	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modu- lation 217 Hz	2	0,3	28	28
	1845	70						
	1970	0 - 1 99 0						
	2450	2 40 0 - 2 57 0	Blue- tooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modu- lation 217 Hz	2	0,3	28	28
	5240	5	WLAN 802.11 a/n	Pulse modu- lation 217 Hz	0,2	0,3	9	9
	5500	10						
	5785	0 - 5 80 0						

Guidance and manufacturer's declaration - electromagnetic Immunity

Radiated RF IEC61000-4-39 (Test specifica- tions for ENCLO- SURE PORT IMMUNITY to proximity magnetic fields)	Test Frequency	Modulation	IEC 60601-1-2 Test level (A/m)	Compliance level (A/m)
	30 kHz	CW	8	8
	134.2 kHz	Pulse modulation 2.1 kHz	65	65
	13.56 kHz	Pulse modulation 50 kHz	7.5	7.5

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general terms and conditions of service and warranty

5-year Inventum warranty!

1. Customers enjoy a 5-year full manufacturer's warranty on all Inventum products.
2. Within this period, a defective product or part, which makes normal functioning of the product or part impossible, will always be repaired free of charge or, if repair is not possible and/or does not provide a solution, replaced (by the same or equivalent product). The assessment of this lies with Inventum.
3. In order to make a claim under the 5-year Inventum warranty you must, within two months after discovering the fault or defect, either return the product to the shop where you bought it, or contact the Inventum customer service department by using the form at www.inventum.eu/service.
4. The warranty period starts from the date stated on the product's original sales receipt.
5. To invoke the warranty, you must always send a copy of the original proof of purchase with the service request and present the original proof of purchase to the service technician.
6. The 5-year warranty applies only to normal domestic use of Inventum products within the Netherlands.

Warranty exclusions

1. The 5-year Inventum guarantee does not apply in case of: normal wear and tear, consumables, improper use, insufficient or incorrect maintenance, incorrect and/or inexperienced assembly and installation of the appliance, repairs by unauthorised third parties, business or commercial use, missing serial number, damage caused by an external cause. This enumeration is not exhaustive.
2. B-choice products and products that have been built-in in a showroom are not covered by the 5-year Inventum warranty. All optical damage such as dents, scratches and slight traces of use are excluded from any warranty.

Reporting malfunctions on Inventum products

1. Faults or defects on Inventum products can be reported by using the service request form on the website www.inventum.eu/service or through the seller of the product.
2. A copy of the original proof of purchase must be included with the request.
3. Inventum's service department determines whether the product will be repaired or replaced, based on the request and the sales receipt.

Important to know

- Products sent without prior notification will not be accepted. Further to the notification you will be informed by Inventum whether a warranty applies and whether you must return the product.
- The replacement or repair of a defective product, or a part thereof, does not lead to an extension of the original warranty period.
- Products eligible for replacement or repair must be emptied and clean.

For a detailed explanation of the warranty conditions and items excluded from them, please visit Inventum's website www.inventum.eu/algemene-voorwaarden

These general warranty and service conditions apply to Inventum brand products purchased after 27 April 2022 and intended for use within the Netherlands.

For Inventum products used outside the Netherlands, but within a EU Member State, the warranty rules of that country apply. Inventum provides warranty and service by a white goods technician only in the Netherlands.

As a consumer, you are entitled by law to a properly working product.

Does the product break down sooner than you might expect?

This means that the product is not in conformity. In that case, the law offers you, the buyer, with a number of legal remedies for your protection. This legal protection is not affected by the above commercial warranty given by Inventum on Inventum products used within the Netherlands.

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Postbus 5023
6802 EA Arnhem

www.inventum.eu

 [instagram.com/inventum1908](https://www.instagram.com/inventum1908)

 [facebook.com/inventum1908](https://www.facebook.com/inventum1908)

 [youtube.com/inventum1908](https://www.youtube.com/inventum1908)

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FSC
www.fsc.org

MIX

**Packaging | Supporting
responsible forestry**

FSC™ C121626

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**Inventum Huishoudelijke Apparaten B.V., Meander 901, 6825 MH
Apparaten B.V.**