

# AREBOS

## Bath stool

AR-HE-BH60004



Please follow all security measures in this user's manual to ensure a secure use.



**Please read and save these instructions. Read through this user's manual carefully before using this product. Protect yourself and others by observing all safety information, warnings and cautions. Failure to comply with instructions could result in personal injury and/or damage to product or property. Please retain instructions for future reference.**

## **Safety instructions**

### **Explanation of the symbols**



By means of a CE marking, it can be recognized that a product complies with the legal requirements of European legal standards and therefore may be traded within the European Community.

### **Safety Instructions**

- Read the instructions completely before first use. Unpack the product and check that all components are included in the package and that the product may be damaged in transit. If this is the case, please contact the service address below.
- There is a special risk of slipping when sitting down and getting up in damp bathtubs or shower rooms, especially if bath products or detergents are used.
- After each assembly, make sure that the stability of your bath stool is given.
- The risk of tipping can be increased by balancing body movement.
- If you can not keep yourself sufficient, ask an assistant for help.
- Do not use high pressure cleaners or harsh, corrosive chemicals for cleaning.

### **Warning! Risk of injury!**

- Incorrect use of the bath stool can cause injury.  
- Never stand on the shower stool!

### **Warning! Risk of damage!**

- Incorrect use of the bath stool can cause damage to the product.  
- Do not use the shower stool if the product is damaged or if the screw connection is loose.

### **The shower stool should not be used by persons with**

- cognitive disorders
- Strong balance disorders
- Seat inability

## **Scope of delivery**

- Seating surface
- 4 chair legs (telescopic)
- 4 rubber feet
- Screws

## Assembly



### Height adjustment of the chair legs

- By adjusting the height of the chair legs, the seat height of the person can be individually adjusted. You have to press the spring button on the chair leg to the telescopic tube. Then push the tube into the desired position.



- **Warning!** After adjustment , make sure the spring button clicks into one of the existing holes.  
**Risk of falling!**
- Check that all chair legs are set at the same height.

## Technical specifications

Material	Aluminium/HDPE
Colour	Silver/white
Seat width	19.7 in (ca. 50 cm)
Seat depth	11.8 in (ca. 30 cm)
Seat height	Adjustable up to 20.47 in (52 cm)
Weight	4.4 lbs (2 kg)
Max. capacity	220.5 lbs (100 kg)

## Cleaning and care

- We recommend cleaning the bath stool after each use. You can wipe the bathroom stool with a damp cloth; if it is soiled you can use mild detergent. To disinfect the product, you can use a standard surface cleaner.

### Maintenance

- The bath stool is designed maintenance-free. If the shower stool is defective, contact the manufacturer to replace the product.

## Disposal instruction

### Disposal of the packaging

- Please make reference to the guidelines and standards for appropriate disposal of the packaging valid in your region. In part, the package may consist of plastic bags - watch this respect, with special care to ensure that this is not out of the reach of children. There is a risk of suffocation!

### Disposal of waste equipment

- Equipment must be disposed of in accordance with the rules and regulations of the local waste disposal.

**Our customer service number: +49 (0) 931-45232700**

## EU Declaration of Conformity

We,

Canbolat Vertriebs GmbH, Gneisenaustraße 10-11, 97074 Würzburg, Germany,

Hereby declare that the product named below, seen its design and construction as well as according to our sales, has been complied with the relevant and basic health and safety EU-requirements.

Name of the product: AREBOS Bath stool

Model Nr.: AR-HE-BH60004

Art. Nr.: 4260199756560

If the product has any modification not allowed by us, this declaration loses its validity.

Tested acc. to:

93/42/EEC OF 14 JUNE 1993 CONCERNING MEDICAL DEVICES;

INCLUDING, AT 21 MARCH 2010, THE AMENDMENTS BY COUNCIL DIRECTIVE 2007/47/EEC

EN ISO 11199-2-2005

EN 980:2008

EN 1041:2008

EN/ISO 14971:2007

ISO-7173; ISO-11199-1; ISO-11334; ISO-7176-8

Date/Manufacturer Signature/Location:

Würzburg, 13.02.2019



Identification of the signatory:

Korhan Canbolat, head of the company

Authorised representative for the technical documentation:

Korhan Canbolat

**Office address:**

Canbolat Vertriebs GmbH

Gneisenaustraße 10-11

D-97074 Würzburg

Return address can be found in the imprint: <https://www.arebos.de/impressum/>

VAT identification number: DE 263752326

Court of the Commercial Register is Würzburg, HRB 10082

WEEE Reg.-No. DE 61617071