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The BLX-10 enables you to listen to the heartbeat, hiccups and kicks of your unborn baby. Thanks to the dual output, you and your partner can both enjoy the sounds together. You can record the sounds using the included recording cord which enables the sounds to either be saved onto a tape recorder, or copied onto a computer. The recording can then be shared over the Internet with your family and friends.

After the birth of your baby, you can record your own heartbeat and play this familiar sound for your baby to hear.

The BLX-10 operates without vibration or radiation so is absolutely safe for you and your baby. You can use the BLX-10 as often as you want.

DECLARATION OF CONFORMITY

The Alecto BLX-10 complies with the essential requirements and provisions as defined by the European Directive 1999/5/EC.

The declaration of conformity is available on the website WWW.ALECTO.INFO

**ENVIRONMENT**

If this product has run its course, do not throw it into the normal household garbage. Rather bring the product to a collection point for the recycling of electrical and electronic equipment.

Dispose of the battery pack in an environmentally-friendly manner according to your local regulations.

INSTALLATION

Battery:

1. Slide the battery cover located in the front of the baby amplifier in a downward position.

2. Place a 9 volt battery (not included) into the battery compartment as indicated.


3. Slide the battery cover back onto the Baby Audio Amplifier.

Headphones:

Connect the supplied headphones into either one of the two audio outlets on the BLX-10. You can attach a second set of headphones (not included) into the other power outlet so you can listen together with your partner.

Recording cord: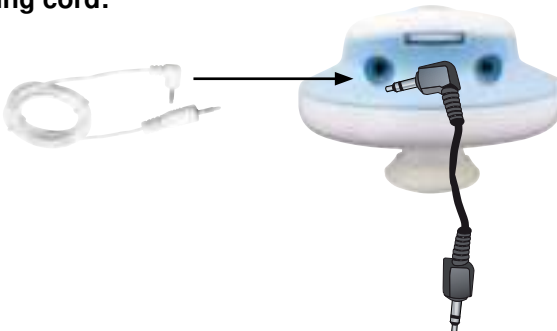

You can record the sounds of your baby by connecting the recording cord into one of the audio outlets on the BLX-10. Connect the other side of the cord to a recording device (tape recorder, computer, etc.). You can connect headphones to the other audio outlet on the BLX-10 so you can hear what you are recording.

Please only use the angled plug of the recording cord.

HOW TO USE

1. Switch On:

Switch the BLX-10 by turning the on/ off / volume control to the right.

2. On / off Light Indicator:

After the click, the on / off light indicator will come on to show that the BLX-10 is now switched on.

3. Sound Sensor:

Turn on the sound sensor by pressing on the blue button. If you release the button, the sound sensor will go off again.

4. Volume:

Turn the volume control to the right to increase the volume and to the left to decrease the volume.

5. Switching off:

Switch off the BLX-10 by turning the on/ off / volume control to the left; after the click, the on/off light indicator will go off to show that the unit is now switched off.

GENERAL TIPS

From what month?

From around the 5th month you are able to hear the heart beat of your unborn baby. This can be different however for each individual.

What you hear:

The heart of an unborn baby's beats between 120 and 160 times per minute, and sounds similar to a galloping horse. Hiccup noises sound like two rapid drumbeats and the baby kicks sound like dull claps.

Best location:

In the early stages of pregnancy a good location to hear the heartbeat is just above the pubic area in the lower abdomen. At a later stage the heartbeat will move higher up towards the middle of the abdomen.

Tip: Take the BLX-10 with you to the midwife and together find the most appropriate place to look for the heartbeat.

Best time to listen:

About 3 to 4 hours after a meal would be the most optimum time to listen.

Do not use gel:

You do not need to use gel with the BLX-10 as it works with a sound sensor.

DISCLAIMER

The BLX-10 is not a medical device and should never replace a doctor or midwife. For questions or concerns about your pregnancy, you should always consult your own doctor or midwife.

CLEANING / MAINTENANCE

- The BLX-10 is not waterproof. You can clean the outside of the product with a damp cloth. Do not use abrasive or harsh cleaners.
- Keep the BLX 10 out of the reach of children and in a safe place.
- Do not place the BLX-10 in direct sunlight.
- It is advisable to remove the battery if the unit is not being used over a long period (>1 month).

WARRANTY CARD

The Alecto BLX-10 has a warranty of 24 months from date of purchase. We guarantee during this period the free repair of defects caused from the material and workmanship. Some things are kept for assessment of the importer.

WHAT TO DO:

If you notice a defect, first consult the manual. If the manual does not give an answer, you can consult your dealer or the the customer service office at www.alecto.info.

TERMINATION OF WARRENTY:

For improper use, incorrect connection, the use of non-original parts or accessories, negligence and damage caused by moisture, fire, flood, lightning and natural disasters. For unauthorized modifications and / or repairs by third parties, incorrect transporting of the equipment without appropriate packaging and if the device is not accompanied by the warranty certificate and purchase receipt. Any further liability, especially for any consequential damages, is excluded.

Service

Help

Alecto[®]

Aziëlaan 12

's-Hertogenbosch

WWW.HESDO-SERVICE.NL

INFO@HESDO-SERVICE.NL

SERVICE@ALECTO.DE

NL +31 (0) 73 6411 355

FR +32 (0) 3 238 5666

DE +49 (0) 180 503 0085