



VR166S
LINTON MOBILE COMMODE & FOOTRESTS



This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for choosing to purchase the Linton Mobile Commode/Footrests from Aidapt. If used in accordance with user instructions they should give you many years of continuous service.

NB. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.

VR166S SPLIT DOWN COMMODE/WITH FOOTRESTS
VR167 FOOTRESTS (PAIR)

Weight Limit: 190kg (30 stone)


Do not exceed the weight limit stated- doing so could put the user at risk.

INTENDED USE

The Linton is a commode or for use over a toilet and may be used to transport persons over short distances. **For Indoor Use Only.**

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage, if you see any damage or suspect a fault, do not use your product as it may compromise the safety of the user, but contact your supplier for support.
- Before beginning assembly, ensure you have all necessary parts (see fig. 1):
 - A pair of side frames with castors attached (A)
 - 1 crossbar with 2 e-clips attached (B)
 - A pair of armrests (C)
 - A pair of footrests (D)
 - Plastic aperture seat with cross piece attached (E)
 - Grey seat cover (F)
 - Back rest (G)
 - Two plastic allen screws (H)
 - Bucket (I)

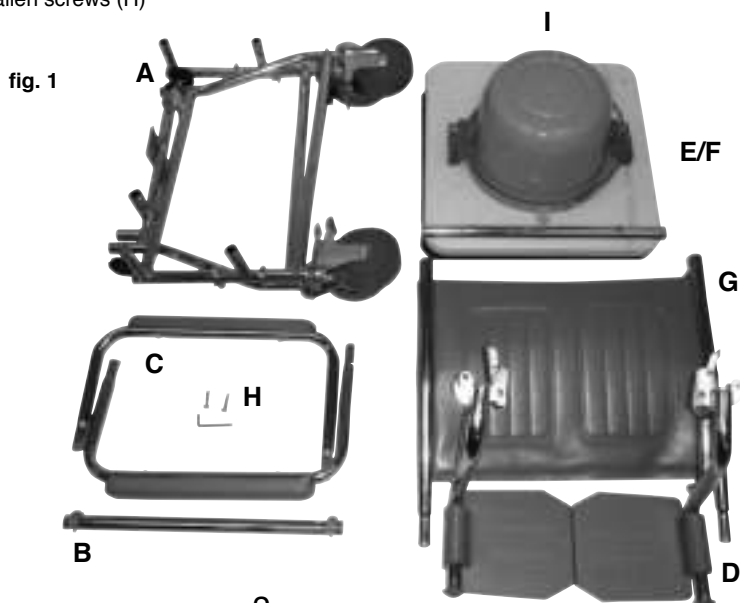






ASSEMBLY

Assembling the frame

1. Take one of the side frames (with castors fitted) (A) and attach crossbar (B) to the lower spigot using the provided e-clip (see fig.2) Make sure the e-clip is secure
2. Take the plastic aperture seat (E) and attach one end of the attached crosspiece onto the upper spigot (see fig. 3) making sure that the locator pin aligns with the hole in the crosspiece (you should hear a click)
3. Attach the second side frame (A) in the same way as explained in steps 1-2.
4. Secure the front spigots of both side frames (A) to the aperture seat (E) using the allen screws and allen key (H) (see fig. 4 and 5)
5. Locate the bucket (I) in the runners on the underside of the aperture seat (E) (see fig. 6)
6. Turn the completed base frame over (see fig. 7)
7. Locate the lower tubes of the back rest (G) into the back spigots of the base frame (see fig. 8)
8. Tighten the knobs on the back of the frame to make sure the back rest is held in place securely (see fig. 9)
9. Slot the armrests into the 2 tubes on either side of the frame (see fig. 10)

Attaching the footrests

1. On the front legs of the commode frame (approx. 150mm above the castors) you will see 2 retaining lugs on each side.
2. Take the footrest and align the holes in the inverted 'U' section with the lugs and lower them into place (see fig. 11)

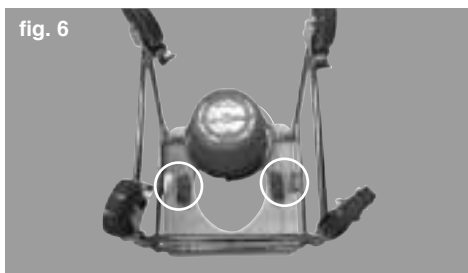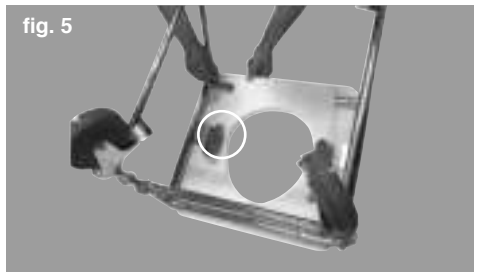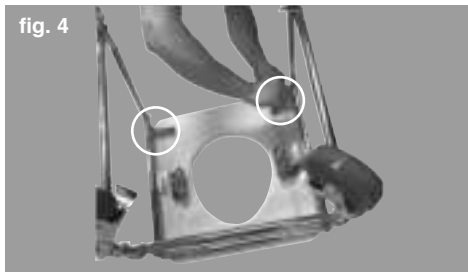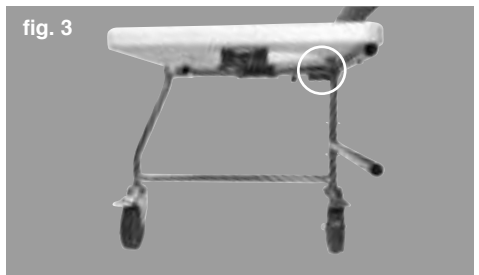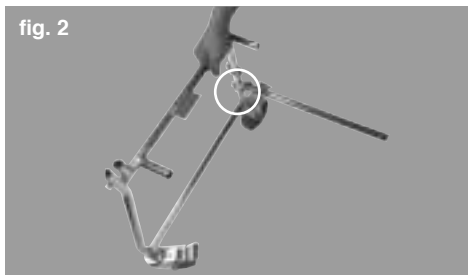



The commode is now ready to use. When the commode is not in use the grey seat cover can be placed over the aperture seat (see fig. 12)

PLEASE NOTE

A lap strap (VA121S) can be fitted to the chair, which, if required, can help to secure the user in the chair.

WARNING

Footrests are only for use to support the feet e.g. when moving the user over short distances. They should not be used to stand on while getting in or out of the commode as this may cause a tipping hazard. If the user whilst sitting on the chair has their feet on the footrests then they must remain upright as to lean forward with their feet on the footrests will cause a tipping hazard. Please note the footrests can be easily folded out of the way.

USE OF THE COMMODE

When moving always lift the commode by placing your hands on the tubular metal frame beneath the blow moulded seat.

- Do not allow children or unauthorized persons to play with or use the commode.
- Do not rock back and forth or sideways when using the commode as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the commode outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury. It is not intended as a wheelchair.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.
- When operating the brake mechanism, do not stamp on the lever as this may cause damage to the mechanism and possible failure.
- Not for use in the shower. Dry immediately if the Linton becomes wet.

CLEANING

A useful guide to decontamination is MHRA leaflet DB2003(06).

Do not use abrasive cleaners as this could damage the surface of the equipment. For cleaning, wipe with a clean cloth soaked in detergent solution and wring out. Depending on the detergent manufacturer's instructions the solution may be heated.

Wipe all surfaces of the equipment and the cloth should be re-immersed periodically and the detergent changed frequently. The detergent should be rinsed or wiped off using a clean cloth immersed in clean water and wrung out.

To disinfect make sure the equipment has been cleaned and dried and then wipe over with a clean cloth and a compatible liquid disinfectant. After disinfecting clean with clean water and dry.

When using detergent or disinfectants then please refer to manufacturers' instructions and if unsure then consult your local Infection Control Specialist for advice.

This equipment can be cleaned/disinfected by moist heat at 80°C for a minimum of 1 minute or autoclaved at 80°C for at least 1 minute. After cleaning or disinfecting dry and treat all joints with a water dispersing agent, e.g. WD40.

COMPOSITION

The frame is made from chromium-plated mild steel. You must not, under any circumstances, use any abrasive cleaners or cleaners which attack chromium plating.

COMMODO BUCKET

To remove the commode bucket, simply slide the bucket out from the chrome runners at the rear of the commode. Cover the top of the bucket to avoid splashing, carefully dispose of the contents into a proper sluice or WC.

Never use an open drain, sink or basin. This could cause serious health risk.

Clean out the residue with a suitable disinfectant and refit the commode after use.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes.

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St. Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptions@aidapt.co.uk • sales@aidapt.co.uk