# VERMEIREN

## Quadri-Light

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

### **Instructions to the specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: B, 2014-10

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2014

**EN Contents**

1.	Technical details	2
2.	Components	2
3.	Explanation of symbols	2
4.	Check after delivery	2
5.	Intended use	3
6.	Assembly (Fig. 1)	3
7.	Dismantling (Fig. 1)	3
8.	Handgrips (Fig. 2)	4
9.	Brake system	4
10.	Basket	4
11.	Mounting / Removing crutch, walking stick	5
12.	Safety instructions	5
13.	Care	5
14.	Disinfection	5
15.	Repairs / Service / Inspection	5
16.	Shipping and storage	6
17.	Warranty	6
18.	Disposal	6
19.	Declaration of conformity	6

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### 4-wheels rollator Quadri-light

#### 1. Technical details

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

<b>Make</b>	Vermeiren		
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout		
<b>Type</b>	Rollator		
<b>Model</b>	Quadri-light		
Width	650 mm		
Width (folded)	290 mm		
Width (transport)	290 mm		
Length (unfolded)	700 mm		
Length (folded)	700 mm		
Length (transport)	400 mm		
Maximum height	930 mm		
Minimum height	805 mm		
Height (folded)	805 mm		
Height (transport)	670 mm		
Seat height	610 mm		
Seat width	450 mm		
Seat depth	200 mm		
Weight	<b>7,20 kg</b>	with basket	7,25 kg
Wheels	Front: 8" (Ø 200 mm x 40 mm) Rear: 7" (Ø 178 mm x 40 mm)		
Brakes	Multifunctional brake		
Max. inclination forwards	15°		
Max. inclination backwards	15°		
Max. inclination sideways	10°		
Maximum user weight	max. <b>130 kg</b>		
Maximum basket weight	max. 5 kg		
Turning width	1310 mm		
Storage temperature	+5°C - +41°C		
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °			

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.


Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

#### 2. Components

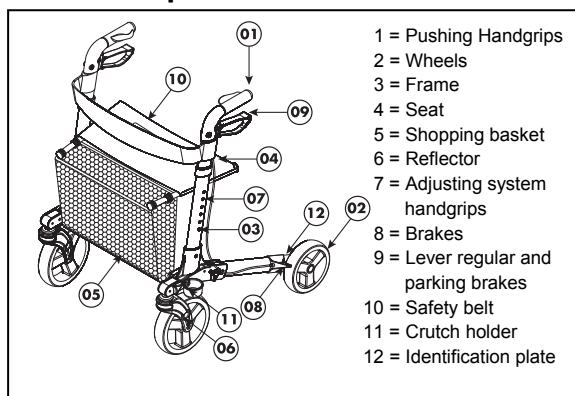#### 3. Explanation of symbols

Maximum mass

Indoor and outdoor use

CE conformity

Read the instruction manual before use

Observe the safety instructions

Type designation

#### 4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 4 wheels (mounted)
- height adjustable pushing handgrips with brakes
- shopping basket
- safety belt
- crutch holders
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

## 5. Intended use

**⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.**

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator can be used indoor and outdoor. The rollator should only be used on surfaces where all four wheels are touching the ground. Not to be used on rocky terrain or steep hills (see technical details: max. inclination) so the rollator can become unstable.

The product is not meant for transport of people and/or goods. The seat can only be used when the parking brake is locked.


## 6. Assembly (Fig. 1)

- ⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.**
- ⚠ CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.**

Steps 1-9 shall be used for first time use or for transportation of the rollator.

1. **Remove packaging.**
2. **Loosen the velcro straps.**
3. **Place the rollator on his side.**
4. **Move the rear wheel downwards (③) until the holes lie over each other.**
5. **Mount the star button ②.**
6. **Move the front wheel downwards until the holes lie over each other.**
7. **Mount the star button ⑤.**
8. **Place the rollator on his other side.**
9. **Repeat steps 4 - 7 for the other side.**
10. **Unfolding rollator**
  - Unfold the rollator by pushing on the handgrips.
  - Push on the tubes of the seat ④ to unfold the rollator further. Keep pushing until the locking mechanism is secured.
  - Check that the rollator is firmly secured and cannot fold anymore.
11. **Adjust the pushing handgrips to the desired height (see further).**

Fig. 1 – Assembly / dismantling rollator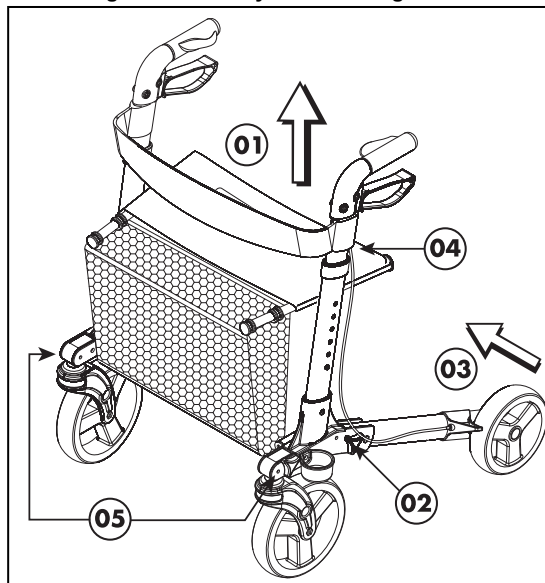

## 7. Dismantling (Fig. 1)


- ⚠ CAUTION: Chance of pinching - Do not place fingers between the components of the rollator.**
- ⚠ CAUTION: Risk of clamping - Keep fingers away from moving parts of the rollator.**

1. **Adjust the pushing handgrips to the minimum position (see further).**
2. **Folding rollator**
  - Pull lightly on the handgrip of the seat ① until the locking mechanism came loose and the rollator can be folded (do these when you stand straight).
  - Fold the rollator by pushing on the handgrips or pulling on the tubes of the seat ②.
3. **For transportation**
  - Remove the star button ⑤.
  - Fold the front wheel upwards.
  - Remove the star button ②.
  - Fold the rear frame / wheel upwards ③.
  - Fasten the rear frame / wheel with the velcro straps to the frame of the handgrips.
  - Repeat the steps above for the other side.

## 8. Handgrips (Fig. 2)

- ⚠ **CAUTION: Risk of falling - Do not exceed the maximum adjustment, your rollator will not be stable anymore.**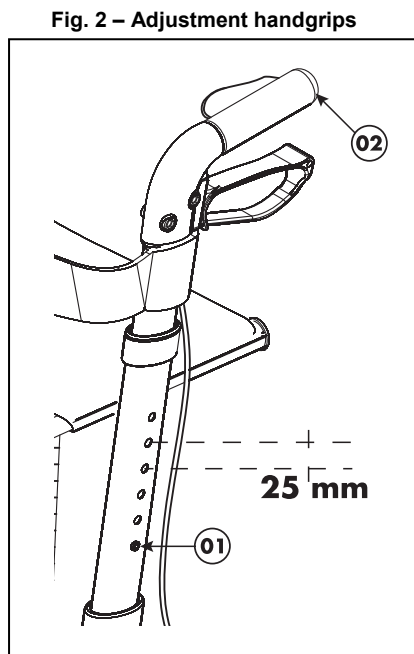


The handgrips can be adjusted in height by following steps:

- Press on the spring pin ①.
- Move the handgrip ② until the desired height (6 different positions: steps from 25 mm). Check that the brake cable cannot stick around the crutch holder.
- Check that the spring pin ① is clicked back and firmly fixated in a hole.
- Adjust the other handgrip ② with the same instructions.
- Check that both handgrips ② are placed in the same position. When standing straight, the handgrips should be as high as your wrists. Leave your arms hanging loose. Elbows should be flexed at 20° to 30°.

## 9. Brake system


- ⚠ **WARNING: Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, mud, ...) - Check the condition of the wheels before each use.**
- ⚠ **WARNING: The brakes can wear - Check the operation of the brakes before each use.**

The rollator has been equipped with a multifunctional brake system. The brake levers underneath the handgrips are regular brakes as well as parking brakes.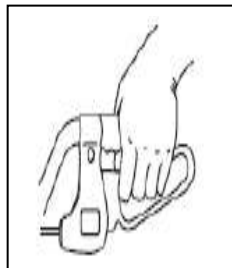

For the usage of a short brake (regular brakes), pull the brake lever upwards (toward the pushing handgrip).

For the usage of a long brake (parking brake), push the brake lever downwards till you hear a locking sound of the brake. The wheels (use both parking brakes left and right simultaneously) are now blocked until you pull the parking brakes upwards again.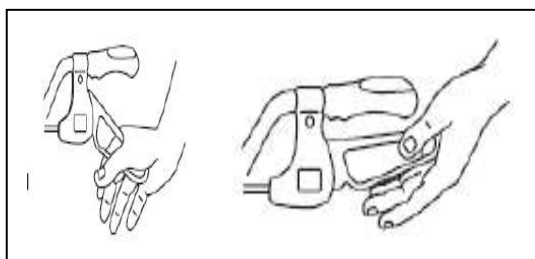


## 10. Basket

- ⚠ **WARNING: Risk of falling, tipping over - Do not overload the shopping basket.**

The shopping basket ③ is meant to be mounted on the front side. Mount the shopping basket ③ with the two basket attachments ① to the tubes of the rollator seat frame ②.

The maximum carrying weight from the shopping basket is 5 kg.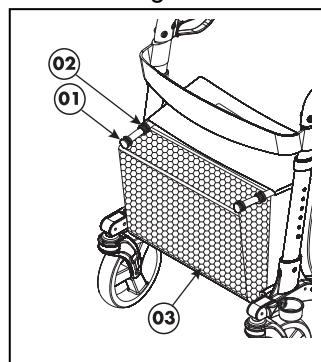


## 11. Mounting / Removing crutch, walking stick

**⚠ WARNING: Risk of injury - Check that the crutch or walking stick is firmly mounted.**

On the left side of your rollator you can find a crutch holder ② and velcro strap ①.

Place the crutch / walking stick in the crutch holder ② and fasten the crutch / walking stick with the velcro strap ①.

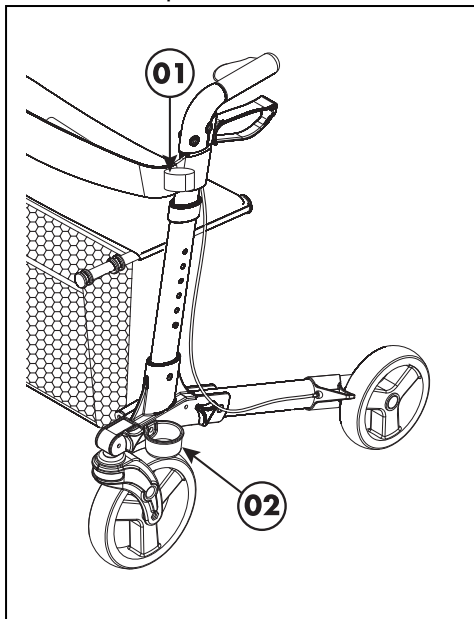## 12. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the rollator.
- Ensure the brakes or the rollator are always in parking-mode before using your rollator. When you are standing comfortable and stable, you may unlock the parking brakes.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check that the pushing handgrips are firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handgrips.
- Do not hold the pushing handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, lock the parking brakes of your rollator and have a seat on the bench.

- **The maximum users weight for your rollator is 130 kg** and can only be used on a flat underground. Do not use your rollator on a staircase or bumps higher than 40 mm.
- When using the rollator outdoors, find yourself flat and stable areas to use your rollator. Avoid curbs and/or deep holes. Find a place where the curbs are bevelled.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Avoid bumpy or sandy roads. Otherwise you might lose control over the stability of your rollator.
- Avoid roads with a very steep inclination.
- Do not use your rollator to transport people or items.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

## 13. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your rollator is preserved in a perfectly functional condition.

## 14. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and plastic surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: [www.rki.de](http://www.rki.de)). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

## 15. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your

specialist dealer. He would gladly assist you further in regard to service and repairs.

**Before you resume use of the rollator, it should be serviced according to the following criteria:**

- Completeness
- Foldingmechanism
- Check the wheels (stability, tracking, fixing screws)
- Frames (deformation, stability, connections)
- Pushing handgrip (safe locking from height adjustment system, good support from the handgrips)
- Brakes
- Seat, safety belt and shopping basket on wear or damage
- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

**Every 8 weeks:**

- Lubricating (moving parts shall be greased)

**Every 6 months or for every new user, it should be serviced according to the following criteria:**

- General review
- Disinfection

## 16. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Do not put too heavy parts on the rollator, not clamping between something, ...)

## 17. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The terms of the guarantee may differ from country to country. Consult your specialist dealer for the guarantee period for warranty claims.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

## 18. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

## 19. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERBODEN N.V.

Address :

Vernieuwenplein 115

2000 Kortrijk

Belgium

declare under his sole responsibility that the CE marked devices :

Productname: Rollator

Productgroep (EMSD): Walker, Rollator (IMEN 27951)

FAMPP (FAMI) registration

number: BE1CA21112-0295-26-011

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Type: 25", 27", 27GEN, 29", 29GD, 29GC, Steator, 201 F

Delta, 44gH, ECO-Light, Quadri-Light

have been classified as class I, according to annex II of MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below including the latest modifications - and with the national law, that organizes this directives :

Medical device directive MDD 93/42/EEC 2007

and is in conformity with the relevant European harmonized standards:

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**CONTRACTUAL WARRANTY**

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

**APPLICATION CONDITIONS**

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

**EXCEPTIONS**

- This warranty is not valid in case of:
- damage due to incorrect usage of the wheelchair,
  - damage during transport,
  - involvement in an accident,
  - a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
  - normal wear of the wheelchair,
  - non-return of the warranty card

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N.V. VERMEIREN N.V.  
Vermeirenplein 1/15  
B-2920 Kalmarhout  
BELGIUM

## SERVICE

The rollator was serviced:  
Le rollator roulant contrôlé:  
De rollator is gecontroleerd:

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---

## Notes

A series of horizontal dotted lines for writing notes.

## Belgium

### N.V. Vermeiren N.V.

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

### Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

### Vermeiren Italia

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.reatime.it](http://www.reatime.it)  
e-mail: [info@reatime.it](mailto:info@reatime.it)

## Poland

### Vermeiren Polska Sp. z o.o

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

### Vermeiren ČR S.R.O.

Nadrazni 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 133 277  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

### L. Vermeiren Ges. mbH

Winetzhammerstraße 10  
A-4030 Linz  
Tel: +43(0)732 37 13 66  
Fax: +43(0)732 37 13 69  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

### Vermeiren Suisse S.A.

Hühnerhubelstraße 59  
CH-3123 Belp  
Tel: +41(0)31 818 40 95  
Fax: +41(0)31 818 40 98  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

### Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)