BRAUN

iCheck® 7

BPW 4500

Braun iCheck® 7

BPW 4500

Wrist blood pressure monitor

GB	English.....	2
DE	Deutsch.....	12
ES	Español.....	22
FR	Français.....	32
IT	Italiano.....	42
NL	Nederlands.....	52
PT	Português.....	62

www.hot-europe.com

This product is manufactured under licence to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks is under licence.

App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Android™, Google Play and the Google Play logo are trademarks of Google LLC.
See at: <https://developer.android.com/legal.html>

iCheck® is a registered trademark of Helen of Troy Limited.

 Kaz Europe Sàrl
Place Chauderon 18
CH-1003 Lausanne
Switzerland

© 2018, All rights reserved.

BPW4500WE P/N A000510R1 30APR18

Product description

Printed in China

Intended use of the Braun iCheck® 7

Braun iCheck® 7 is a blood pressure monitor which has been developed for fast, accurate blood pressure measurements from the wrist. This blood pressure monitor operates as stand-alone device and automatically records your most recent readings. The recorded readings can be downloaded later on your Smartphone for advanced trend interpretation.

What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. It may also vary over a short period of time. Therefore readings from successive measurements can fluctuate.

This device will display your blood pressure reading referring to two values: the systolic and the diastolic values. The systolic blood pressure (the upper number) indicates how much pressure your blood is exerting against your artery walls when the heart beats. The diastolic blood pressure (the lower number) indicates how much pressure your blood is exerting against your artery walls while the heart is resting between beats. The pulse value is also displayed after each measurement.

The ESH (European Society of Hypertension) and WHO (World Health Organisation) have set up the following standard blood pressure values when measured with a resting pulse at home:

Blood pressure (mmHg)	Too low	Normal values	Grade 1 mild hypertension	Grade 2 moderate hypertension	Grade 3 severe hypertension
SYS = systole (upper value)	below 100	up to 134	135-159	160-179	≥180
DIA = diastole (lower value)	below 60	up to 84	85-99	100-109	≥110

For every measurement displayed on the screen, the associated colour (from green to red) will indicate the corresponding blood pressure level. You can use this classification daily to help you understand your blood pressure reading. If you are really concerned by the classification, you should consult your doctor as soon as possible.

	SYS (mmHg)	and	DIA (mmHg)
Green >	< 135		< 85
Yellow >	135-159	or	85-99
Orange >	160-179	or	100-109
Red >	180+	or	110+

Safety information and important precautions

To ensure accurate measuring results, read the complete instructions for use carefully.

People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in the extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves. Deviations in blood pressure values may occur in such cases.

If your blood pressure reading is in the severe hypertension range with values of 180 mmHg systolic and above, and/or 110 mmHg diastolic and above, and you find you have one or two readings in this range – please see your doctor straight away.

If you are under medical treatment or taking any medication, please consult your doctor first.

The use of this blood pressure monitor is not intended as a substitute for a consultation with your doctor.

Key rules for accurate blood pressure measurement

Always take readings at the same time of day, under the same conditions.

Do not take a measurement within 30 minutes after smoking, drinking coffee or tea or any form of exertion. These factors will influence the measurement.

Always measure on the same wrist. We recommend using left wrist.

Take off your wrist watch and jewellery before placing the monitor on the measuring arm.

Remove tight clothing from your arm.


If you want to take multiple measurements, wait approximately 3 minutes before repeating a measurement.

Product description (see Page 1)

- A** On/Off button (also used for Bluetooth® pairing, see page 2)
- B** Graphic display for measurements and indications
- C** Instant read colour indicator
- D** Battery compartment cover
- E** Cuff

Inserting/replacing batteries

- Remove the battery compartment cover by sliding it downwards. Insert the two batteries supplied with this product, with polarity as indicated in the battery compartment. When replacing batteries, use alkaline batteries exclusively, type AAA/LR03, 1.5V.
- New batteries will provide approximately 300 measurements.

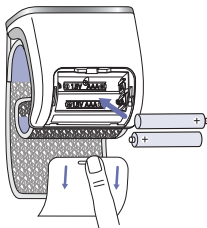The basics

The On/Off button is located to the right of the display. This is a round button with the power symbol. Simply tap on this touch-sensitive button to turn your iCheck® 7 On and Off.

When turning On, you will see a welcome screen first. When this message disappears the device will be ready to use.

The graphic display shows numerical values for measurements, indications for user, guidance system and error notifications, if any.

At the conclusion of the measurement the device will show a coloured heart. Use the colour codes shown on Page 6 find out what the colour means.

Taking a blood pressure measurement

1. Sit down, relax, keep still, especially the arm being used for the measurement, do not move or speak. For greatest accuracy, we recommend using the left wrist for the measurement.
2. Fit the monitor on your left wrist, roughly 1 (one) cm below the palm line (see Fig. 1 for correct placement). **DO NOT ATTACH LIKE A WRIST WATCH.** The device should be on the inner side of your wrist. Secure the cuff with the Velcro fastener, so that it is tight but comfortable. There should not be a space between the cuff and the wrist.
3. Press the On/Off button  to start a measurement. The graphic display will show the welcome screen followed by the 'Easy Positioning System'. This is shown as a moving dot within a white circle.

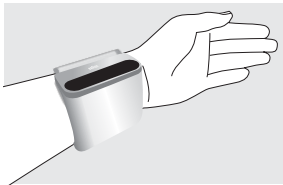
Fig. 1

- The **'Easy Positioning System'** helps to ensure the arm is in the correct position. Simply move your arm up and down until the white dot stays inside the circle (see Fig. 2). This means that the iCheck® 7 is now level with your heart.
- The measurement starts automatically once the correct position has been obtained. Do not move or speak while the measurement is in progress.

Note: if it is necessary to interrupt a measurement, press the On/Off button  at any time. The device immediately cancels the measurement, lowers cuff pressure and turns itself off.

- During the measurement, keep your legs uncrossed and feet flat on the floor.

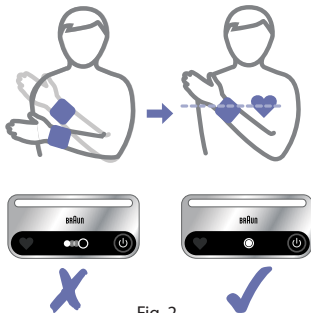
Fig. 2

Understanding your reading

Once the measurement is complete, the instant read 'heart' symbol will illuminate. The 'meaning' of the colour can be found in the table on Page 6. The graphic display will show the numerical values of your blood pressure readings.


SYS: shows the systolic blood pressure in mmHg

DIA: shows the diastolic blood pressure in mmHg

PUL: shows the pulse - heart beats per minute

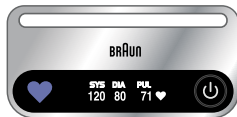 This symbol at the right-hand side of pulse reading indicates a regular heartbeat.

 This symbol at the right-hand side of pulse reading indicates that pulse irregularities were detected during the measurement. This irregular heartbeat can be caused simply by talking, moving or shaking during a measurement. In this case, the result may deviate from your normal basal blood pressure – repeat the measurement. Normally this is not cause for concern. However if the symbol appears frequently, and these guidelines are strictly followed, you should seek medical advice.

The **instant read colour indicator** follows the ESH and WHO recommendations for interpreting readings and is described in the table below.

	Green: normal blood pressure	Your blood pressure reading is normal, and values are below 135 mmHg systolic and below 85 mmHg diastolic.
	Yellow: mild hypertension	Your blood pressure reading is in the mild hypertension range with values between 135-159 mmHg systolic and/or 85-99 mmHg diastolic.
	Orange: moderate hypertension	Your blood pressure reading is in the moderate hypertension range with values between 160-179 mmHg systolic and/or 100-109 mmHg diastolic.
	Red: severe hypertension	Your blood pressure reading is in the severe hypertension range with values of 180 mmHg systolic and above and/or 110 mmHg diastolic and above. If you get readings in this range, consult your doctor straight away.

Blood pressure readings will be automatically saved in the device memory. Results are displayed for approx. 60 seconds, then device will automatically shut-off. You can also shut off your device with On/Off button.

Note: If your iCheck® 7 device is switched on within 60 seconds of powering off, the welcome screen is bypassed.

Memory functions

Your iCheck® 7 will store up to 100 readings in the internal memory. Readings are stored automatically when the measurement is complete. When the 100 readings are reached, the new readings automatically replace the oldest ones. We suggest you regularly transfer your stored readings onto the Braun Healthy Heart app on your Smartphone in order to follow the evolution of your blood pressure easily over a long period of time.

The memory is non-volatile. Stored readings will not be lost when the batteries are replaced.

Connection to Smartphone and Braun Healthy Heart app

Your iCheck® 7 can be connected by Bluetooth® to your smartphone (iOS and Android™) to download stored readings automatically. When readings are downloaded to your Smartphone, they are deleted from your iCheck® 7 memory.

Make sure the 'Braun Healthy Heart' app has been downloaded onto your Smartphone and the app is open.

Switch on the Bluetooth function on your smartphone. If you are using the app for the first time, it will guide you in pairing your iCheck® 7. If you have already used the app before, go to the "Connect device" function available in the app menu and follow the instructions.

During Bluetooth pairing, iCheck® 7 will show a flashing Bluetooth icon  on the display. Once pairing is complete, iCheck® 7 will show your name on the display.

Battery life indicator

Batteries are low

When the batteries are approximately 80% discharged, the low battery symbol will show at power-up for about 6 seconds. The iCheck® 7 device will continue to measure reliably, but new batteries should be obtained.

Batteries are empty —replacements required

When the batteries are fully discharged, the empty battery symbol will show at power-up for about 6 seconds and then device will automatically shut off. You cannot take any readings and must replace the batteries.

Storage and cleaning

- Do not place/leave the item in direct sunlight, at high temperatures or in damp and dusty places.
- Do not store at extremely low (less than -20 °C) or high (more than 55 °C) temperatures.
- Use a wet cloth or one impregnated with a mild cleansing agent to clean the case and then use a dry cloth to wipe it dry.
- Do not use any strong cleansers to clean it.

- When the unit is not going to be used for a long time, remove the batteries. (Batteries may leak or cause damage).
- Do not modify the device. NEVER open the device! This will void the manufacturer's warranty.

Calibration

This device was calibrated at the time of manufacture. If used according to the instructions for use, periodic recalibration is not required.

Warranty

Please read all instructions before attempting to use this device. Please retain the receipt as proof/date of purchase.

The receipt must be presented when making any claim within the relevant warranty period. Any claim under the warranty will not be valid without a proof of purchase.

Your device is warrantied for two years (2 years) from date of purchase.

This warranty covers defects in materials or workmanship that occur under normal use, defective devices meeting these criteria will be replaced free of charge.

The warranty DOES NOT cover defects or damage resulting from abuse or failure to follow the user instructions.

The warranty becomes void if the device is opened, tampered with, or used with non Braun branded parts or accessories, or if repairs are undertaken by unauthorised persons.

Accessories and consumables are excluded from any warranty.

For support requests, please visit www.hot-europe.com/support or find service contact information at the end of this owner's manual.

This warranty is applicable to Europe, Middle East and Africa only.

The LOT and SN of your device are printed on the rating label located inside the battery compartment.

UK Only: This does not affect your consumer statutory rights.

Troubleshooting

Problem	Reason	Solution
	Low battery warning	Be ready to replace batteries with new ones soon.
	Empty battery warning	Insert new batteries or check that batteries are correctly inserted.
Device does not turn on, screen is black	Problem with batteries	Change batteries for new ones and make sure to insert them correctly. If problem persists, contact consumer support
"ERROR X"	Cannot connect to Smartphone	Stay away from radio interference such as microwave ovens, WiFi routers, laptop computers, etc. Make sure Bluetooth is enabled on your Smartphone before starting the pairing sequence. Follow the instructions for Bluetooth pairing in the Braun APP and re-initiate the Bluetooth pairing sequence using the Braun APP.
"ERROR 0"	Measuring error The pulse signals on the cuff are too weak	Reposition the cuff and repeat the measurement.
"ERROR 1"	Pumping error The cuff is not fastened properly	Reposition the cuff, close it tighter then repeat the measurement.
"ERROR 2"	Out of Range Results Error	Relax for a while and repeat the measurement. Make sure the cuff is correctly positioned on your arm.
"ERROR 3"	Pump time error The cuff is not fastened properly	Re-fit the cuff, and take a measurement again.
"ERROR 4"	User could not find the correct wrist position (angle) in less than 60 seconds	Position the wrist at heart level as indicated by the 'Easy Positioning System' .
"ERROR 254"	Device internal error	Remove and then reinsert the batteries. If the problem persists, contact consumer support.
"ERROR 255"	Calibration error	Remove and then reinsert the batteries. If the problem persists, contact consumer support.
"ERROR 256"	An error occurred during the Bluetooth Firmware update	Device will automatically restart using the original Bluetooth firmware. Re-initiate the firmware download process or follow the on-screen instruction from Braun APP.

Specifications

Model	BPW4500
Measurement method	Oscillometric
Display	OLED display
Blood pressure measurement range	40 ~ 255mmHg
Pulse measurement range	40 ~ 199 beats/minute
Calibration accuracy	Blood pressure: +/- 3 mmHg Pulse rate: +/- 4% of reading
Inflation/Deflation	Automatic
Sets of memory	Up to 100 readings
Cuff size	Fits wrist circumference: 12.5 - 21 cm (4.9 - 8.3 inch) min/max
Operating temperature	+10 °C ~ +40 °C, less than 85 % R.H. non-condensing
Storage temperature	-20 °C ~ +55 °C, less than 85 % R.H. non-condensing
Unit weight	Approximately 125g (excluding batteries)
Power source	Alkaline battery: 2 x AAA (LR3) 1.5V
Auto power off	Approx. 60 seconds
Service life	5 years
Wireless communication	Bluetooth® Smart Frequency Range: 2.4GHz ISM Band (2400 - 2483.5 MHz) Modulation: GFSK Effective radiated power: <20dBm

Important

Read the operating instructions.

If the device is not used within the specified temperature, humidity and atmospheric pressure ranges, the technical accuracy of the measurement cannot be guaranteed.

Classification:

Internally powered equipment

Type BF applied parts

IP22

Protected against solid foreign objects of 12.5 mm diameter and larger. Protected against water drops falling vertically when the device is tilted up to 15°.

Not suitable for use in the presence of flammable anaesthetic mixtures with air, oxygen or nitrous oxide.

Continuous operation with short-time loading.

Operating temperature

Storage temperature

Storage humidity

Subject to change without notice.

This device conforms to the following standards:

EN 60601-1:2006/A1:2013 - General requirements for basic safety and essential performance.

EN 60601-1-2:2015 - Medical Electrical Equipment - Electromagnetic compatibility - Requirements and tests.

EN 60601-1-11:2010 - Medical Electrical Equipment - Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment.

This product conforms to the provisions of the EC Directive 93/42/EEC (Medical Device Directive and superseding regulations).

This product carries the CE mark and is manufactured in conformity with the RoHS Directive 2011/65/EU.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC.

For a detailed description of EMC requirements please contact an authorised local Service Centre (see guarantee) or visit www.hot-europe.com/support.

Portable and mobile RF communications equipment can affect medical electrical equipment.

Hereby, Kaz Europe Sàrl, declares that the radio equipment type BPW4500 is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following internet address www.hot-europe.com/support.

This product contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in household waste, but take it to an appropriate local collection centre provided in your country.

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