FERTILILLY

A FERTIL CONCEPT

- A. FERTILILLY CONCEPTION GEL
- B. HOW TO USE THE FERTILILLY CONCEPTION GEL
- C. PRECAUTIONARY WARNINGS
- D. INGREDIENTS

A. FERTILILLY CONCEPTION GEL

The FERTILILLY Conception Gel is a personal lubricant that can be used while trying to conceive. The Conception Gel is clinically proven to be friendly to sperm. The FERTILILLY Conception Gel is water-based, has a pH-friendly to sperm survival and elicits the properties of the fertile fluid. The FERTILILLY Conception Gel can be used in combination with the FERTILILLY Conception Cup to increase your chances of conception.

B. HOW TO USE THE FERTILILLY CONCEPTION GEL

Step 1
Remove the seal from the tube before first use.

Step 2
Apply the lubricant to the outside of the vagina or to the shaft of the penis prior to intercourse.

Dispense a small amount of product onto your fingers and/or the palm of your hand and apply to genital area. Very moist to achieve desired lubrication.

Step 3
After each use, firmly close the tube. Do not leave the tube open between uses.

C. PRECAUTIONARY WARNINGS

If you have sensitive skin, try applying a thin gel on the inside of your elbow to test if you are sensitive to its ingredients. The FERTILILLY

Conception Gel can be used for 3 months after opening. Do not use the FERTILILLY Conception Gel if you have a yeast or bacterial infection or if you have recently had infections or wounds on the vaginal area. Resumes use once the infection has completely cleared. If you experience any adverse effects, discontinue use and consult your physician. Keep out of the reach of children.

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Step 3
The use of the gel should be discontinued if you experience any adverse effects. Consult your physician if you experience any adverse effects.

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Use the gel for 3 months after opening. Do not use the gel if you have a yeast or bacterial infection or if you have recently had infections or wounds on the vaginal area. Resumes use once the infection has completely cleared. If you experience any adverse effects, discontinue use and consult your physician. Keep out of the reach of children.

Step 1
Remove the seal from the tube before first use.

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