MEDICAL DEVICES INSIDE THE KIT / MEDICINISCHE GERÄTE INNERHALB DES KITS / DISPOSITIFS MÉDICAUX À L'INTÉRIEUR DU KIT / DISPOSITIVO MEDICI INTERNI AL KIT / PRODUCTOS SANITARIOS DENTRO DEL KIT / DISPOSITIVOS MÉDICOS DENTRO DO KIT / MEDISCHE HULPMIDDELEN IN DE KIT


STERILE **R** Sterile swab / Steriler Tupfer / Tampon stérile / Tampone sterile / Hisopo estéril / Cotonete estéril / Steriel wattenstofje

CE See the label of the swab included in the kit / Siehe das Etikett des im Kit enthaltenen Tupfers / Voir l'étiquette du tampon inclus dans le kit / Consultare l'etichetta del tampono incluso nel kit / Veja la etiqueta del hisopo incluida en el kit / Veja o rótulo do cotonete incluído no kit / Zie het etiket van het wattenstofje in de kit.

STERILE **EO** Sterile tongue depressor / Steriler Zungenspatel / Abaisse-langue stérile / Abbassalingua sterile / Depressor de lingua estéril / Abaixador de lingua estéril / Steriele tongspatel

CE See the label of the tongue depressor included in the kit / Siehe das Etikett des im Kit enthaltenen Zungenspatels / Voir l'étiquette de l'abaisse-langue inclus dans le kit / Consultare l'etichetta dell'abbassalingua incluso nel kit / Veja la etiqueta del depressor de lingua incluído en el kit / Veja a etiqueta do abaixador de lingua incluído no kit / Zie het etiket van de tongspatel in de kit

REF **880060** **PRIMA** Lab SA **EC REP** Qarad EC-REP BV
Via Antonio Monti 7 Pas 257
CH-6828 Balerna, Switzerland 2440 Geel, Belgium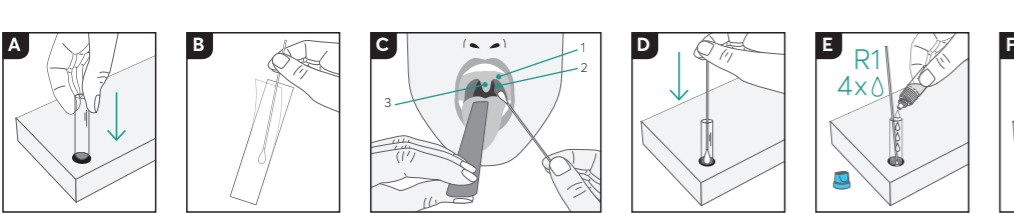

EN **Strep A Self-Test**
For Self-testing - Instructions for Use

[STREPTOCOCCUS A]

Group A β-hemolytic Streptococcus (also known as *Streptococcus pyogenes*) is a Gram-positive bacterium that causes a variety of pathological conditions and complications, including pharyngitis (throat infections), skin infections (such as impetigo) and erysipelas, acute glomerulonephritis, sepsis and rheumatic heart disease. Pharyngitis is commonly caused by viral infections and less often by bacterial infections, which require treatment with antibiotics. Strep A Self-Test is an immunological test designed to specifically identify Streptococcus A bacteria in pharyngeal infections. It helps to understand quickly if a sore throat is caused by Streptococcus A or by other microorganisms that do not require treatment with antibiotics.

[PRINCIPLE OF THE TEST]

Strep A Self-Test is an immunochromatographic test that detects the presence of Streptococcus A specific antigens, using gold-conjugated antibodies integrated into the reactive strip.

[F.A.Q. – QUESTIONS AND ANSWERS]

How does the Strep A Self-Test work? Group A β-hemolytic Streptococcus could be involved in infections of the pharynx. For appropriate antibiotic treatment to be administered, the bacterium must be identified first. Strep A Self-Test can detect Streptococcus A antigens through the specific antibodies it contains.

When can the test be used? Strep A Self-Test can be carried out if there are symptoms such as pain when swallowing, sore throat, red and swollen tonsils (sometimes with white spots or pus), small red patches on the back of the palate (soft or hard), swollen lymph nodes, fever, headache, nausea or vomiting, especially in children.

Can the result be incorrect? The result is correct if the instructions are followed carefully. However, the result may be incorrect if Strep A Self-Test is wet before performing the test or if the sample from the pharynx is not collected and prepared correctly (carefully follow the instructions for collecting the sample from the pharynx –FIG. C). Incorrect results may also be obtained if the test is carried out after treatment with antibiotics or at the beginning of the infection (the concentration of bacteria is below the level of detection).

How do I interpret the test if the colour and intensity of the test and control bands are different? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the Test band, even if it is weak.

What is the purpose of the control band? The appearance of this band means the test has worked correctly.

What should I do if the result is positive? If the result is positive, Streptococcus A has been detected in the sample from the pharynx and you must consult a physician. Please be aware the Strep A Self-Test cannot distinguish between live and dead bacteria. Patients who have recently recovered from Streptococcus A or similar infections may test positive because of inactive bacteria that are still present in the body.

What should I do if the result is negative? A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test. If symptoms persist, seek medical advice.

Is the result reliable if read after 10 minutes? No. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.

How accurate is the Strep A Self-Test? The device specifically identifies Group A β-hemolytic Streptococcus when its concentration in the extracted sample is equal to or higher than 10⁴ CFU/mL (cut-off value). In some cases, samples with concentrations below this value (down to 5 x 10³ CFU/mL) can yield a positive result. The Test is very accurate. Assessment reports show an analytical specificity of 99,00% (CI 95%: 97,63 – 100%) and a sensitivity of 91,89% (CI 95%: 78,70 – 97,20%) with an overall concordance rate of 97,90% (CI 95%: 96,08 – 99,72%). Although this test is reliable, it may yield false positive and negative results.


[PRECAUTIONS]

- Read these instructions for use carefully before performing the test. The Test is reliable if the instructions are followed correctly (reaction times, how the sample is collected and the extraction instructions shown).
- Keep the Test out of the reach of children.
- Not suitable for use on children under the age of 2 years.
- Do not use the Test after the expiry date or if the package has been damaged.
- Store the Test components at a temperature between +4 °C and +30 °C. Do not freeze.
- Only use individual tests once.
- In vitro diagnostic device for individual use.
- The test is for external use only. Do not swallow solutions 1 and 2 and the other items in the box! If swallowed, seek medical advice immediately. Show the remaining parts of the box, the instructions for use and the package.
- If liquids come into contact with the eyes, skin or other mucosae, rinse immediately and thoroughly with water and contact a physician. Show the vials' labels.
- After using, dispose of all components according to local laws. Liquid wastes should not be disposed of down the drain.
- Do not open the sealed elements (test case, swab and tongue depressor) until just before performing the test. Take care to open as marked.
- Do not switch the caps of the reagent vials.
- All kit components are single-use, with the exception of the test package (which can be re-used as test tube holder) instructions for use and solution 1 and 2 which must be stored, closed with their caps, at a temperature between +4 °C and +30 °C.


[CONTENT]

1 hermetically sealed aluminium pouch containing 1 STREP A TEST cassette, 1 desiccant bag; 1 sterile swab; 1 sterile wooden tongue depressor; 1 empty plastic test tube for sampling with dropper; 1 instructions for use leaflet; 1 dropper vial (Blue cap) containing enough STREP A TEST – R1 solution to perform 1 Test. R1 solution contains 20.7% Sodium Nitrite; 1 dropper vial (Green cap) containing enough STREP A TEST – R2 solution to perform 1 Test. R2 solution

newfoundland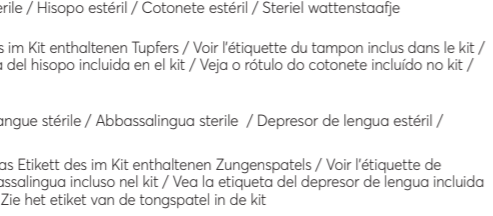

REF **880060** **PRIMA** Lab SA **EC REP** Qarad EC-REP BV
Via Antonio Monti 7 Pas 257
CH-6828 Balerna, Switzerland 2440 Geel, Belgium

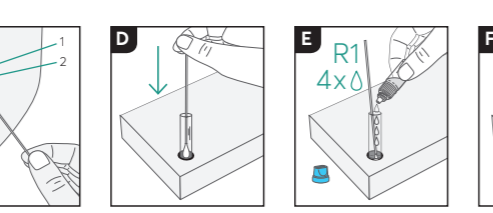EN **Strep A Self-Test**
For Self-testing - Instructions for Use

[STREPTOCOCCUS A]

Group A β-hemolytic Streptococcus (also known as *Streptococcus pyogenes*) is a Gram-positive bacterium that causes a variety of pathological conditions and complications, including pharyngitis (throat infections), skin infections (such as impetigo) and erysipelas, acute glomerulonephritis, sepsis and rheumatic heart disease. Pharyngitis is commonly caused by viral infections and less often by bacterial infections, which require treatment with antibiotics. Strep A Self-Test is an immunological test designed to specifically identify Streptococcus A bacteria in pharyngeal infections. It helps to understand quickly if a sore throat is caused by Streptococcus A or by other microorganisms that do not require treatment with antibiotics.

When can the test be used? Strep A Self-Test can be carried out if there are symptoms such as pain when swallowing, sore throat, red and swollen tonsils (sometimes with white spots or pus), small red patches on the back of the palate (soft or hard), swollen lymph nodes, fever, headache, nausea or vomiting, especially in children.

Can the result be incorrect? The result is correct if the instructions are followed carefully. However, the result may be incorrect if Strep A Self-Test is wet before performing the test or if the sample from the pharynx is not collected and prepared correctly (carefully follow the instructions for collecting the sample from the pharynx –FIG. C). Incorrect results may also be obtained if the test is carried out after treatment with antibiotics or at the beginning of the infection (the concentration of bacteria is below the level of detection).

How do I interpret the test if the colour and intensity of the test and control bands are different? The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is positive regardless of the intensity of the colour of the Test band, even if it is weak.

What is the purpose of the control band? The appearance of this band means the test has worked correctly.

What should I do if the result is positive? If the result is positive, Streptococcus A has been detected in the sample from the pharynx and you must consult a physician. Please be aware the Strep A Self-Test cannot distinguish between live and dead bacteria. Patients who have recently recovered from Streptococcus A or similar infections may test positive because of inactive bacteria that are still present in the body.

What should I do if the result is negative? A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test. If symptoms persist, seek medical advice.

Is the result reliable if read after 10 minutes? No. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.

How accurate is the Strep A Self-Test? The device specifically identifies Group A β-hemolytic Streptococcus when its concentration in the extracted sample is equal to or higher than 10⁴ CFU/mL (cut-off value). In some cases, samples with concentrations below this value (down to 5 x 10³ CFU/mL) can yield a positive result. The Test is very accurate. Assessment reports show an analytical specificity of 99,00% (CI 95%: 97,63 – 100%) and a sensitivity of 91,89% (CI 95%: 78,70 – 97,20%) with an overall concordance rate of 97,90% (CI 95%: 96,08 – 99,72%). Although this test is reliable, it may yield false positive and negative results.

[RESULTS INTERPRETATION]

Read the result after 5 minutes. Do not read the result after 10 minutes.

Positive result - Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line.

This result means that the test has detected the presence of Streptococcus A specific antigens in the sample. You should consult a physician.

Negative Result - A coloured band appears only under the C (control) sign.

This result is negative and means that no Streptococcus A antigens have been detected in the sample from the pharynx or that their amount is too low to be detected.

Invalid result - No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new sample.

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