Drive Medical GmbH & Co. KG \* Leutkircher Str. 44 \* D-88316 Isny / Allgäu  
Tel. 0049 (0) 7562-9724-0 \* Fax 0049 (0)7562-9724-25  
mail info@drivemedical.de \* www.drivemedical.de

Drive Medical Ltd. \* Ainley's Industrial Estate; Elland \* GB West Yorkshire HX5 9JP  
Phone +44 1422-314488 \* Fax +44 1422-314489  
www.drivemedical.co.uk

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*Grab bars stainless steel*

www.drivemedical.de  
www.drivemedical.co.uk

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## Indication

This Grab bar, is particularly suitable as holding element for the Bath-/WC -area. It is equipped with 2 fastening flanges.

**Attention!** The grab bar is not suitable for hanging up shower benches.

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## Safety- and assembling instructions

With the attachment of the product, the cultivation height of 850 mm (floor – upper edge pipe) is to be considered. Due to different wall- soil – and ceiling conditions, are to be selected individually certified fixings, depending upon wall construction.

The provided standard attachment contains plastic pegs as well as stainless-steel screws Ø 6 x 60. These are suitable for the attachment at the following wall types:

Cement, natural stone, solid brick and line sandstone.

With other wall types, a specialized company of the attachment technology must be consulted. The assembly has to come for safety reasons via a specialist of the building services engineering. For the assembly no guarantee is taken over. For the security of the user, examined max. load not to be exceeded.

**The operating instructions as well as the safety and assembling instructions are to be considered absolutely!**

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## Product Contents

Grab Bar, Mounting material ( 6 screws, 6 pegs)

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## Cleaning / Maintenance

The employment depends on the spatial conditions. The high-grade steel has to be cleaned and disinfected in regular intervals for saving the longevity. For the cleaning and disinfection, commercial cleaning products without scrubbing effect are to be used.

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## Warranty

We issue a warranty of 24 months from date of purchase.

Subject to technical alterations.