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How to use?

Instructions

- Fill the Peri Bottle with (lukewarm) water or chamomile tea (when it comes to personal hygiene, water is the most effective and gentlest method) and screw the cap tightly and extend the mouthpiece.
- Sit on the toilet (or anywhere else) and place the bottle between the legs (not internally!) with the extended mouthpiece in the desired direction.
- Gently squeeze the bottle to create a gentle and focused stream of water.
- Repeat the process as needed for comfortable and effective cleaning.

When to use?

Suitable for cleaning the anus, female vulva and menstrual nursing after daily toilet; pregnant women to clean their genitals after using the toilet and to clean their wounds after childbirth and surgery; Cleaning hemorrhoids patients after toilet can effectively improve hemorrhoids; The elderly, motorists, students living in schools, white-collar workers and other sedentary people clean their parts after using the toilet. Can effectively reduce the genitals and vulva anal bacteria growth and infection, also can save paper. The use concerns the entire perineum.

What can be put in the Peri Bottle?

Both (lukewarm) water and chamomile tea are possible to use. The water temperature should not exceed 40°C.

Washing/cleaning

For cleaning, a water temperature of approximately 38°C is recommended for rinsing. By washing it by hand. Too high a temperature can burn you or deform the bottle. It is also possible to wash the peri bottle in the dishwasher, but this is not recommended.

klantenservice@purelygoods.nl

After childbirth

During pregnancy

Baby's

With hemorrhoids

Joint pain in the elderly

Period