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EN: Instructions for use for nasal rinse

1. Wash your hands well before using the bottle. Since you will be cleaning your sinuses, there should be no bacteria on your hands. Then fill the clean bottle with 300 ml of lukewarm water and add 2.25 grams of salt. You can pre-mix it to dissolve it as best you can.
2. Always check whether the temperature is right. It is important to ensure that the water is not too hot, as this can further irritate your nose hairs.
3. Then stand in front of a sink and keep your face slightly bent forward. Look down and keep your head slightly bowed. Make sure your chin is brought to your chest. This ensures that the water cannot run back into the throat. If water does get into your throat, you don't have to worry about this, because a saline solution is not harmful.
4. Hold the bottle upside down and press the nozzle against your nostril. Make sure the nozzle closes the nostril tightly to allow all the water to flow through the cavities. Then press the white button at the bottom of the bottle. By pressing the button, the water flows through the nasal passages. Keep in mind that this may feel a little awkward at first.
5. Make sure that you do not hold your breath while rinsing, but that you continue to breathe through your open mouth. First, rinse the nostril that is least clogged. If both nostrils are blocked, you can choose one of the two.
6. If you have rinsed one nostril, it is useful to blow your nose before continuing with the other nostril. If you have a bad cold, you can repeat the process again.

Safe use of the nasal rinse

Although there are no known side effects of rinsing with a saline solution, it is not wise to rinse the nose in the following situations:

- Your nostril is completely closed
- You suffer from inflamed nostrils
- You get internal nosebleeds
- Your nose is broken or bruised
- You have had an ear operation (contact your doctor)

Always use boiled or distilled water.

This nasal rinse is not suitable for children under 8 years old.

Storing the product

The product should be stored in a cool, dry, well-ventilated place. Avoid direct sunlight and keep it away from liquid or gas. This product is good for two years from the date you first used it.

Product specifications:

- **Materiaal:** BPA-free plastic
- **Content:** 300 ml
- **Color:** Blue

Package Contents:

- 1x Nasal shower
- 2x Attachments
- 1x Nasal Spray
- 30 Nasal salt rinse bags

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Manufacturer:
Ningbo Albert Novosino Co., Ltd.
No.1 Xinheng 3 Road, Ciceng,
Jiangbei District, Ningbo 315036, China

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Humiss International B.V.
Joop Geesinkweg 701, 1114 AB