# EIS

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Am Lenkwerk 3, 33609 Bielefeld

E-Mail: [amazon@aaa-marketing.de](mailto:amazon@aaa-marketing.de)

## EN - Instructions for use about your lubricant

# EIS

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**IMPORTANT! Please read these instructions carefully prior to using your product for the first time, and ensure you have fully understood the warning and safety information. Keep these instructions handy, as they contain important information on the safe use of the toy, as well as providing necessary technical and legal information which you might need at a later date.**

**Use:** Simply drip the lubricant onto the desired area of the body, or onto a condom that has been rolled onto your erect penis and distribute evenly with your fingers. For hygiene reasons, we recommend that you wash your hands thoroughly prior to application. Ensure that you apply the lubricant carefully to ensure that the condom is not damaged, e.g. due to sharp fingernails. The lubricant is water-based. This means that it is easily rinsed off with water, leaving behind no residue or sticky feeling on the skin. It can be used with all sex toys, regardless of material composition.

**More information about the lubricant:** This oil- and fat-free lubricant is a medical product suitable for use on the skin, in intimate areas, and with all types of condom and sex toy. It meets the strict requirements of the European Directive 93/42 EEC on Medical Devices and the German Medical Devices Act. This is a high-quality product thanks to the select raw ingredients, monitored manufacturing processes and comprehensive testing. It has been tested for compatibility with the skin and mucous membranes, and is verifiably condom-safe.

**Other information:** Lubricants reduce friction and simultaneously improve gliding capacity, which is why we recommend their use during intercourse and when using toys. This enables you to avoid pain and minor injuries in your sensitive intimate areas. It also improves safety when using condoms thanks to the reduced friction.

**Contraception and protection against infection:** The lubricant is not a form of contraception. It does not contain spermicides and offers no protection against infection.

**When trying to conceive:** Gynaecologists recommend that you should use lubricants sparingly or abstain from using them completely when trying to conceive as they may hamper conception.

**For information on ingredients that may cause intolerance or an allergic reaction,** please refer to the 'Ingredients (INCI)' section of the product label. If you find that you are unable to tolerate the product, please remove the product as described above, and consult a doctor if necessary.

Once opened, use within 3 months. Observe the best before date on the bottle.

## **Specifications**

Contents (in ml): 300

Gross weight (in g): 330

Ingredients (INCI): glycerin, water, hydroxyethylcellulose, sodium lactate, sodium benzoate, allantoin, lactic acid, salicylic acid, menthol, aroma

In the event of faults or operating difficulties, please contact [amazon@aaa-marketing.de](mailto:amazon@aaa-marketing.de). Our customer support team will be happy to assist you.

**CE 0197**

## **Limitation of liability**

Use at your own risk! Triple A Import GmbH and/or their retailers are liable for damages caused by intentional or grossly negligent behaviour of Triple A Import GmbH, a representative or vicarious agent, in accordance with legal directives. With the exception of injury to life, limb or health, and breach of essential contractual obligations (cardinal obligations), Triple A Import GmbH and/or their retailers are only liable for damages which can be attributed to intentional or grossly negligent behaviour. This also applies to indirect damages, in particular lost profits.

Moreover, liability with regard to consumers due to an assumed guarantee, except in the case of intentional or grossly negligent behaviour, or where there is an injury to life, limb or health and a breach of essential contractual obligations

# EIS

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(cardinal obligations) is limited to the typical, foreseeable damages on conclusion of the contract, and also to the level of the average damages typical for this contract. This also applies to indirect damages, in particular lost profits. With regard to companies, except in the case of injury to life, limb and health or intentional or grossly negligent behaviour by Triple A Import GmbH, and/or its retailers liability is limited to the typical, foreseeable damages on conclusion of the contract, and also to the level of the average damages typical for this contract. This also applies to indirect damages, in particular lost profits.

Claims for liability due to product liability legislation remain unaffected.

No liability will be accepted for damages due to:

- Non-compliance with this manual
- Improper use
- Unauthorised alterations
- Technical modifications
- Use of unapproved spare parts
- Use of unapproved accessories

Triple A Import GmbH  
Am Lenkwerk 3, 33609 Bielefeld  
E-Mail: amazon@aaa-marketing.de

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