freedom&comfortable
safe&environmental

Aneer
Menstrual cup

The product is made of imported medical silica gel, which is without irritation and allergenic characters, and is widely used in health care products, such as baby pacifiers, catheter etc. This product does not contain silicone and allergic substances, meaning there is no possible smell and seal situation while using, and daily activities are not restricted during its valid 15 years.

Instructions
It apply to women who has not produced or less than 20 years of age
1. apply to women who has produced or older than 20 years of age making with boiling water totally for 2 minutes while first using.

Put in: Firstly, wash your hands and had the sterilized menstrual cup, and then hold the bottom of the menstrual cup and push into the vagina gently until the cervix. Secondly, rotating the menstrual cup to ensure that it is fully open and close bonding with the vagina. And last adjust the handle to the proper length.

Take out: Hold with your fingers and press the menstrual cup body slightly, squinting or sitting on the toilet, drag gently the menstrual cup after the air is in.

Replacing and cleaning properly according to your situation while period. And this product can be reused after cleaning and disinfection. Use with caution for its usual experiences.

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This product was made of silicone medical imported that is without irritate or absorbent, widely applied in the medical or subjects of the health care such as baby pacifier, catheter etc. This product is without silicone substances allergenic. During its use, there is no smell and seal situation, because they restrictions on the activities during. Its period of validity is 15 years.

Instructions
It apply to women who never had birth or younger than 20 years
1. apply to women who have had birth or older than 20 years
Before the first use, immerse in the boiling water 2 minutes.

Caution: Wash the hands and keep the cup menstrual sterilized. Please to push bottom and gently insertion into the cervix until the full of the cup. Then the cup can be open and close completely directly and contact directly with the vagina. It can be realize the length to the large space.

Take out: squat or sit on the toilet. Pushing downwards gently, holding the cup and the bottom of the cup and it will enter and take the cup smoothly.
Replacement and keep the cup according to your own situation. This product can be used repeatedly after being washed and disinfection.
Use with caution for its usual experiences.

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