Livingful Premium Irrigation Set 2L

EN

The system can be used:

1. To deliver an enema for bowel irrigation
2. For female hygiene when used as a vaginal douche

Product description: the product is composed of a storage bag for the liquid to be used and a tube set with a tap, tubing and 2 different tips. It can be cleaned and re-used.

Instructions for use: Please consult your doctor before using the product. It is particularly important that doctors recommendations are taken specially if using any other rinsing than lukewarm water.

Using the irrigation set: Push the screw adaptor into the tubing until it is secure and screw onto the male threaded screw outlet in the bag. Push the tap into the other end of the tubing. You can then either put the enema tip or the vaginal douche tip onto the tap as required. Close the tap thoroughly, hang the bag using the eyelet and fill with the fluid desired. The higher the bag is positioned the greater the pressure of the fluid when it's discharged from the outlet.

Disinfecting and maintenance of the irrigation set: before you use the irrigation set for the first time and each time thereafter you should dismantle all the components and clean with warm soapy water. If required you can briefly immerse the product in boiling water.

Warning: the irrigation set should not be used in the microwave or cleaned with alcohol, solvents, acid-based cleaning fluids. Keep out of children.

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