


KETONE TEST STRIPS

Reagent Strips For Urinalysis Anti-VC Interference ability

TESTING AND READING TIME

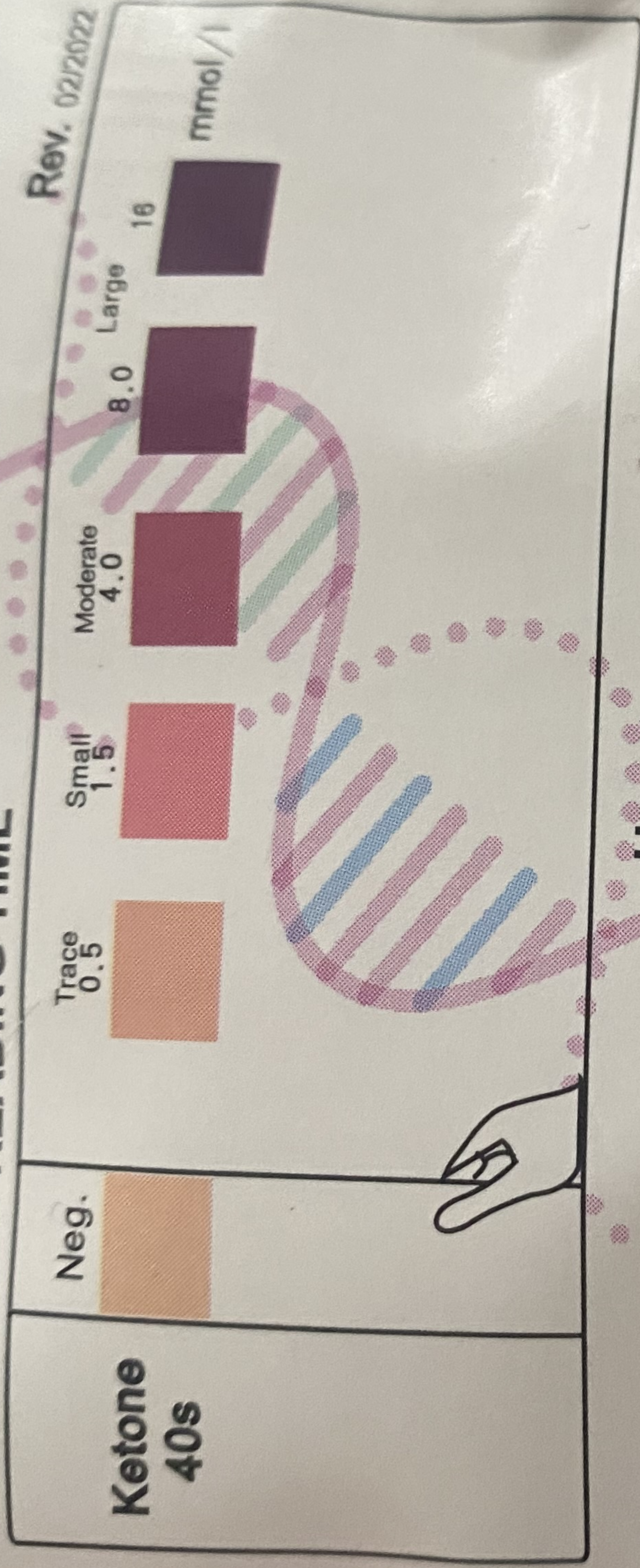

How to use

1. Take out one strip. Dip the pad in the urine cup or single stream of urine.
2. Wait exactly 40 seconds and compare to the color chart on the bag.



Important

Do not remove desiccants.
Keep away from light and moisture.
Close promptly after taking out strips.
Don't touch test areas of reagent strips.
Use within the expiration. Once opened, they are best if used within 60-90 days.

LOT 2022021001
Q2-10-2022
02-09-2024

100 strips

