TOPCOM®



EAR & FOREHEAD THERMOMETER 201

USER GUIDE / HANDLEIDING / MANUEL D'UTILISATEUR
GEBRAUCHSANWEISUNG / MANUAL DE USUARIO / BRUKSANVISNING
BRUGERVEJLEDNING / ANVÄNDARHANDBOK / KÄYTTÖOHJE
MANUALE D'USO / MANUAL DO UTILIZADOR
UŽIVATELSKÁ PŘÍRUČKA / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / HASZNÁLATI ÚTMUTATÓ
INSTRUKCJA OBSŁUGI / MANUAL DE INSTALARE SI UTILIZARE
UŽIVATEĽSKÝ MANUÁL
V1.1 - 06/11

CE 0123

- UK** The features described in this manual are published with reservation to modifications.
- NL** De in deze handleiding beschreven mogelijkheden worden gepubliceerd onder voorbehoud van wijzigingen.
- F** Les possibilités décrites dans ce manuel sont publiées sous réserve de modifications.
- D** Die in dieser Bedienungsanleitung umschriebenen Möglichkeiten, werden vorbehaltlich Änderungen publiziert.
- ES** Las características descritas en este manual pueden ser objeto de futuras modificaciones.
- S** Funktionerna i denna bruksanvisning publiceras med reservation för ändringar.
- DK** Vi forbeholder os retten til ændringer af de specifikationer, der er beskrevet i denne brugsanvisning.
- N** Funksjoner beskrevet i denne manualen kan endres uten nærmere informasjon.
- SF** Tässä ohjekirjassa julkaissut tiedot voivat muuttua ilman ennakkoilmoitusta.
- IT** Le caratteristiche descritte nel presente manuale vengono pubblicate con riserva di modifica.
- PT** As características descritas neste manual são publicadas sob reserva de modificação.
- CZ** Možnost úpravy funkcií popsaných v této příručce vyhrazena.
- GR** Οι λειτουργίες που περιγράφονται στο παρόν εγχειρίδιο, δημοσιεύονται με επιφύλαξη τυχόν τροποποιήσεων.
- HU** A jelen használati útmutatóban leírt adatokra a gyártó fenntartja a változtatás jogát.
- PL** Właściwości opisane w niniejszej instrukcji obsługi są publikowane z zastrzeżeniem prawa wprowadzenia zmian.
- RO** Descrierile din acest manual sunt publicate sub rezerva modificarilor.
- SK** Vlastnosti popísané v tejto príručke sú publikované s vyhradeným právom na zmenu.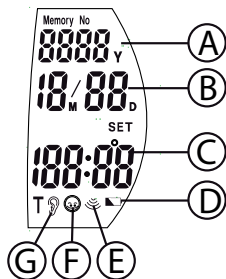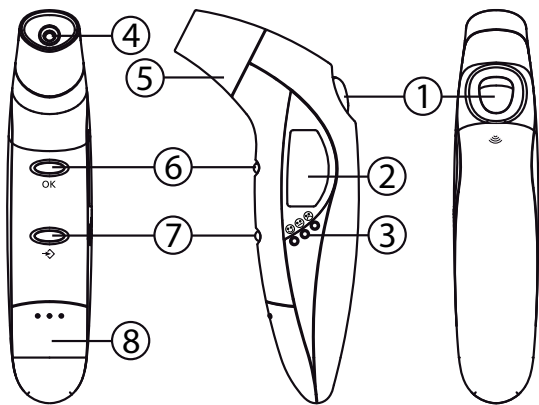

1 INTRODUCTION

The Ear & Forehead thermometer uses infrared technology to measure infrared energy emitted from the forehead or eardrum and surrounding tissue, and then converts it into temperature value.

2 ABOUT BODY TEMPERATURE

Normal human body temperature fluctuates throughout the day; generally, lowest in the morning, highest in the evening. Different parts of human body also have different temperatures. Therefore, the standards of measuring oral, axillary, rectal and ear temperatures are different. The human body temperature is controlled and adjusted by the hypothalamus in the brain. As the veins flow through the hypothalamus, they also flow through the ear-drum in the mid ear. Thus, measuring the ear temperature (the temperature of the ear drum membrane) can best reflect the body temperature.

Normal body temperature is a range, not a fixed value.

When using an IR ear thermometer to take temperature, the normal body temperature range is 35.8 °C - 38.0 °C. Also, people's normal temperature ranges tend to decrease with their age.

However, the normal range is also different from person to person. To determine your and your family members' normal temperature ranges is very important. It can be done easily by using our Infrared Thermometer. Please practice to take temperatures on yourself and your healthy family members to determine their normal temperature ranges.

3 SAFETY NOTES

WARNING

- Please read carefully through the following information concerning safety and proper use. Make yourself familiar with all the functions of the equipment. Be careful to keep these advice notes and if necessary pass them on to a third party.
- This thermometer is intended for household use only. The use of this thermometer is not intended as a substitute for physician's consultation.
- Don't touch or blow the infrared sensor. A polluted infrared sensor may cause inaccuracy.

EAR & FOREHEAD THERMOMETER 201

- Clean a polluted infrared sensor with a soft wet cloth in a gentle manner. Cleaning with toilet paper or paper towel may scratch the infrared sensor, causing inaccuracy.
- Keep the unit out of children's reach. A self-measurement forced by children may cause ear damage. For accidental swallowing of a battery, please contact a doctor at once.
- Measurement must be suspended until the ear is warmed up after outdoor activities to avoid lower results.
- After a shower or exercise, the body temperature will rise. Please take a rest before taking temperature.
- For proper measurement, condition the thermometer for 30 minutes in measurement site till room temperature.
- Earwax or obstructions will affect the accuracy of the thermometer.

NOTE

- Stop using the product in occurrence of any pain. It is likely to damage the ear hole.
- It is not recommended to use the product on users suffering ear disease including otitis external and lymphantitis. Affected part is likely to deteriorate.
- Don't use the product in a wet ear hole after swimming or bathing. It is possible to damage the ear hole.

PROHIBITED

- Don't throw batteries into fire. Batteries may explode in fire

4 CARE AND MAINTENANCE

- Do not shock, vibrate or drop the thermometer.
- Do not allow any liquid in the thermometer.
- Do not disassemble, repair or remodel the thermometer.
- Keep out of reach of children.
- Clean the main body of the thermometer with a soft and dry cloth.
- Do not use benzene, gasoline, thinner or any other cleaner containing abrasive agents.

- The tip of the probe is the most delicate part of the thermometer, accurate readings from the thermometer can be obtained only if the tip of the probe is clean and intact.
- To clean the tip of the probe, hold the thermometer downwards so that no liquid can enter. Use a cotton swab moistened with alcohol to wipe gently and carefully on the surface of the tip of the probe. Allow an hour drying time before operating the thermometer again.

5 BUTTONS

See folded cover page


- | | |
|--------------------------------|-------------------------------------|
| 1. START - button | 5. Forehead probe |
| 2. LCD display | 6. OK - button |
| 3. Temperature LED indication | 7. SET - button |
| 4. Infrared thermometer sensor | 8. Battery compartment (1 x CR2032) |

6 DISPLAY

- | | |
|-----------------------------|------------------------|
| A. Year | E. Measuring mode icon |
| B. Date | F. Forehead mode icon |
| C. Room- / Body temperature | G. Ear mode icon |
| D. Low battery icon | |

7 BATTERY INSTALLATION

1. Remove the battery cover .
2. When the batteries are pre-installed. Pull out the insulator.
3. If it is not pre-installed, insert a CR2032 - 3V - Lithium battery with the positive pole (+) facing upwards.
4. Reposition the battery cover.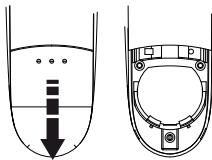

Replace the battery when  **D** is displayed.
Take out the battery if you do not use it for a long time.
You have to dispose of the batteries in an environment friendly manner according to your country regulations.

8 HOW TO USE

- It is recommended that you take three temperatures under the following circumstances.
 - Infants in the first 3 months of life.
 - Children under three years old who have a condition such as compromised immune system and for whom the presence or absence of fever is critical.
 - When you are first learning to use the thermometer until you are comfortable with the technique and are gaining consistent readings
- Wait about 30 seconds before measuring the same person to avoid excessive cooling of the skin.
- If the results are different, use the highest reading.

- Clean the probe cover to ensure accurate measurement.
- As with any type of thermometer, slight temperature variations (+/- 0.2 - 0.3°C) can happen, if consecutive measurements are taken.

8.1 EAR TEMPERATURE

- Pull off the Forehead probe **5** from the unit.
- Press the **START** - button **1** to turn the power on. The display will show all symbols.

Before inserting the probe:

For children < 1 year: Pull the ear down and back.

For children > 1 year to adult: Pull the ear up and back.

- While holding the ear, insert the probe to seal the ear canal.
- Press the **START** - button **1** once. You will hear a short beep. The ear mode icon **6** is displayed.
- When you hear a double beep, the measurement is finished.
- Take out the thermometer from the ear canal.
- The result is displayed.

8.2 FOREHEAD TEMPERATURE

- Put on the forehead probe **5**.
- Gently position the probe onto the center of the forehead.
- Press the **START** - button **1** once. You will hear a short beep. The forehead mode icon **F** is displayed.
- When you hear a double beep, the measurement is finished.
- The result is displayed.

At completion of measurement, the unit automatically returns to time mode after 30 seconds.

The display will automatically turn off completely, 3 minutes after time mode is activated.

8.3 TEMPERATURE LED INDICATION

As soon as the result is displayed, the relevant LED (3) will light.

- Green Light - 35.6 °C to 37.2 °C (96 °F to 99 °F)
- Orange Light - 37.3 °C to 37.5 °C (99.1 °F to 99.5 °F)
- Red Light - 37.6 °C to 42.9 °C (99.6 °F to 109.3 °F)

8.4 MEMORY

The result of each measurement will be stored automatically into the memory. If the memory is full (30 records), the oldest measurement result will be removed.

To recall the memory:

- Press the **START** - button **1** to turn the power on.
- Press the **SET** - button **7** successively to scroll through the memory. Each result is displayed with date and sequence number.

8.5 DISPLAY ERROR MESSAGES

'HI'	The measured temperature is > 42,9 °C / 109,3 °F.
'LO'	The measured temperature is < 32,0 °C / 89,6 °F
'ErrE'	The ambient temperature is not within the operation specifications.
'ErrU'	Does not measure after pressing 'Measure'. Follow the steps mentioned in the user guide.
'ErrH'	An error is detected during self-check. Remove the batteries for 10 seconds and try again.
'ErrS'	Sensor error. Do not use the thermometer under extreme environment.
'ErrR'	Hardware problem. Return the unit to the service center.
'ErrL'	Battery low. Replace the battery.
'ErrP'	Hardware problem. Return the unit to the service center.

9 ADVANCED MODE

9.1 TIME SETTINGS

- Power on the unit by pressing the **START** - button **1** once. You will hear a short beep. Time mode is displayed.
- Press and hold the **SET** - button **7**. The 24hr/12hr system is blinking.
- Press the **SET** - button **7** to switch between 24hr and 12hr system.
- Press the **OK** - button **3** to confirm. The hour digits are blinking.
- Press the **SET** - button **7** repeatedly to increase the digit.
- Press the **OK** - button **3** to confirm. The minutes digits are blinking.
- Repeat previous steps for setting the minutes, year, month and day.
- After setting the day, the unit will enter sleep mode setting.

9.2 SLEEP MODE

When sleep mode is not activated, the display is continuously ON. The display will toggle between time and room temperature.

After setting the hour, sleep mode setting is displayed:

- Press the **SET** - button **7** to switch between **ON** or **OFF**.
- Press the **OK** - button **3** to confirm.

10 TEMPERATURE UNIT

- Power on the unit by pressing the **START** - button **1** once. You will hear a short beep. Time mode is displayed.
- Press simultaneous the **START** - button **1** and **OK** - button **8**.
- A single beep confirms that you have set the temperature to °C.
- A double beep confirms that you have set the temperature to °F.

11 DISPOSAL OF THE DEVICE (ENVIRONMENT)

At the end of the product lifecycle, you should not throw this product into the normal household garbage but bring the product to a collection point for the recycling of electrical and electronic equipments. The symbol on the product, user guide and/or box indicate this.

Some of the product materials can be re-used if you bring them to a recycling point. By re-using some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities in case you need more information on the collection points in your area.

12 FEATURES

Model	HV-T68
Temperature Sensor type	Infrared
Measuring temperature range	32 °C ~ 42,9 °C (89,6 °F ~ 109,3 °F)
Room temperature	5 °C ~ 59,9°C (41 °C ~ 139,8 °F)
Display accuracy	0,2°C or 0,4°F
Measurement accuracy	35,5°C ~ 42°C - (+/- 0.2°C) 32° C to 35.4 °C - (+/- 0.3 °C) 42,1C to 42.9 C° - (+/- 0.3 °C) 95,9°F ~ 107,6°F - (+/- 0,4°F) 89,6°F ~ 95,8°F - (+/- 0,5°F) 107,7°F ~ 109,3°F - (+/- 0,5°F) Room temperature +/- 1°C
Operating ambient temperature range	15°C ~ 40°C (59,0°F ~ 104,0°F)
Operating ambient humidity range	30 ~ 80% RH
Storage environment temperature range	-10°C ~ 55°C (-14,0°F ~ 131,0°F)
Storage environment humidity range	30 ~ 85% RH
Memory	Last 30 measurements
Battery Type	1 x CR2032 - DC 3V - Lithium
Automatic switch-off	30 seconds after last measurement
Weight	65g (with battery)
Classification 	<ul style="list-style-type: none"> • Type BF • Class IIa • Internally powered equipment • IPX0 • Not suitable for use in presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide • Continuous operation with short-time loading

13 CE MARK

The CE symbol on the unit, user guide and giftbox indicates that the unit complies with the essential requirements of the medical directive 93/42/EEC. The declaration of conformity can be found on:

<http://www.topcom.net/cedeclarations.asp>

14 TOPCOM WARRANTY

14.1 WARRANTY PERIOD

The Topcom units have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. There is no warranty on standard or rechargeable batteries (AA/AAA type).

Consumables or defects causing a negligible effect on operation or value of the equipment are not covered.

The warranty has to be proven by presentation of the original or copy of the purchase receipt, on which the date of purchase and the unit-model are indicated.

14.2 WARRANTY EXCLUSIONS

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories are not covered by the warranty.

The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation.

No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

Any warranty claims will be invalid if the unit has been repaired, altered or modified by the buyer.

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