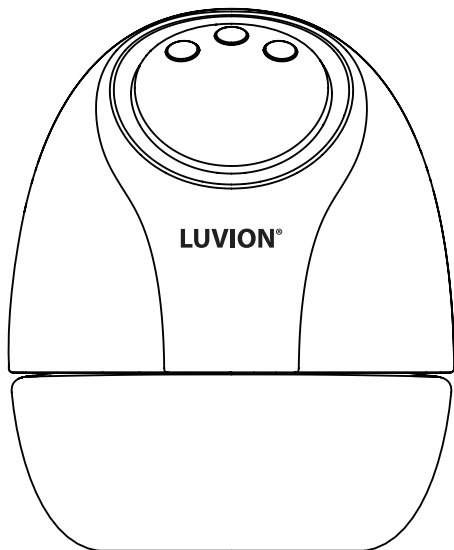
LUVION®

VIENNA

WEARABLE BREAST PUMP

Quick Start Guide

- NL** Snelstart Handleiding & Veiligheidsinstructies
- EN** Quick Start Guide & Safety Instructions
- FR** Guide de démarrage rapide & Consignes de sécurité
- DE** Schnellstartleitfaden & Sicherheitshinweise

WELKOM / WELCOME / BIENVENUE / WILLKOMMEN

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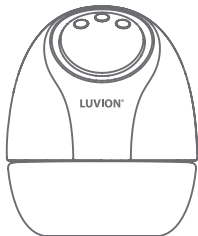
EN Thank you for purchasing the Luvion Vienna Wearable Breast Pump. Our Quick Start Manual will provide you with all the information to quickly start using your Wearable Breast Pump. Save the manual for future revision. For more information, download the full English manual as a PDF at: luvion.com/contact.

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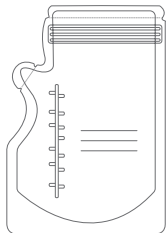
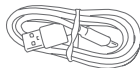
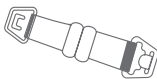
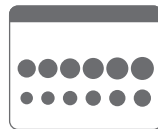
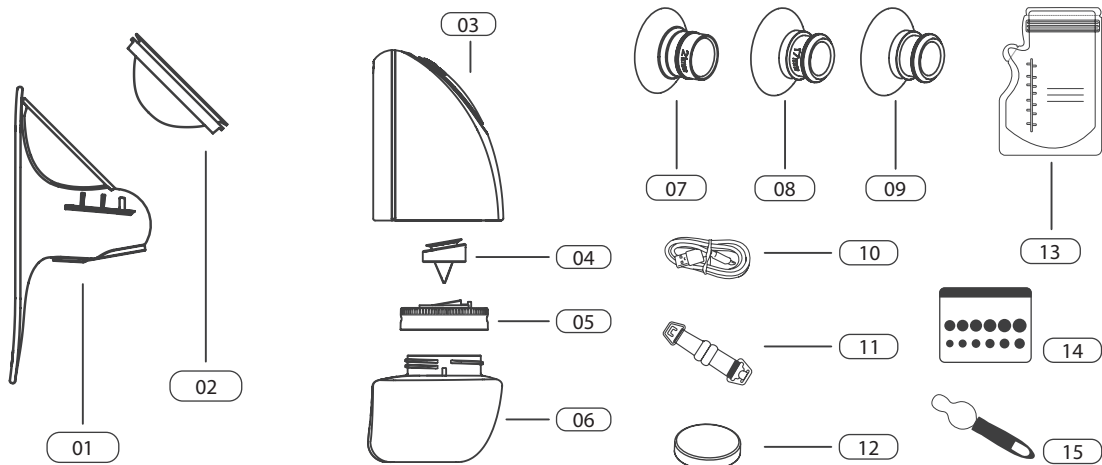
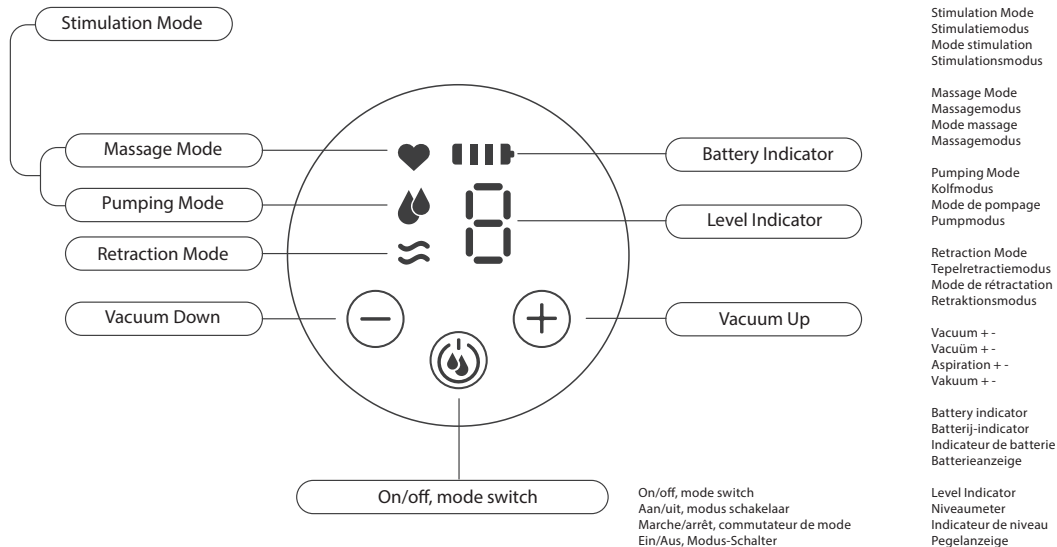
EN

The Wearable Breast Pump is an electric hands-free breast pump to comfortably pump your breastmilk. The device is intended for a single user.

The operating expectancy of the device is approximately 500 hours. The expectancy of the replaceable washable parts is estimated at six months, depending on the use and handling of the parts. We recommend replacing the washable parts after a long period of time when the product is not used. Or before you start pumping for your next child. This product has a built-in Li-ion battery. When the battery indicator light flashes during use, we recommend using a certified 5V 1A power adapter as soon as possible to charge the device.

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EN Cleaning and sterilizing

Clean and sterilize all parts that come into contact with the breast milk before your first use of the Wearable Breast Pump. Disassemble the parts that come into direct contact with the breast and milk immediately after use and clean them to remove any remaining milk. This prevents bacterial growth, which may be harmful to you and your baby. Be cautious when cleaning the breast shield, seal and spout. Your breast pump will not function properly when these parts are damaged.

The pump motor is not suitable for washing or sterilizing. This part does not come into contact with breast milk. You can wipe this part with a soft cloth to clean the housing. By using incorrect cleaning methods, your claim to the warranty lapses.

These parts are not suitable for the dishwasher.

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EN Breast shield measurements

The Wearable Breast Pump comes with a pre-assembled breast shield in size 25 mm and three additional silicone inserts in sizes 21 mm, 17 mm and 15 mm, so you can always find a comfortable fit.

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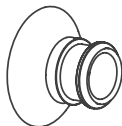
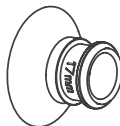
EN

- A. Make sure that all parts have dried before assembling them (after the parts have been cleaned and sterilized). Place the seal (02) in the upper part of the breast shield (01). The edge of the seal closes closely to the edge of the breast shield.
- B. Place the valve (04) on the spout (05). The edge of the cap closes closely to the opening of the spout.

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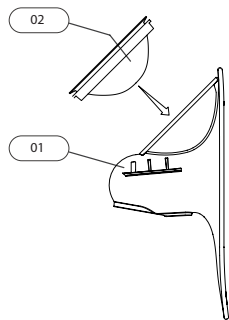
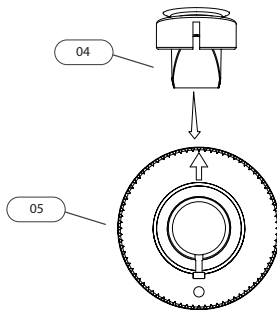
EN

- C. Place the spout on the 180 ml bottle and turn it clockwise until the arrow matches the stripe on the milk bottle.
- D. First place the breast shield (01) in the pump motor (03) and then place the milk bottle (06) in the bottom of the pump motor.

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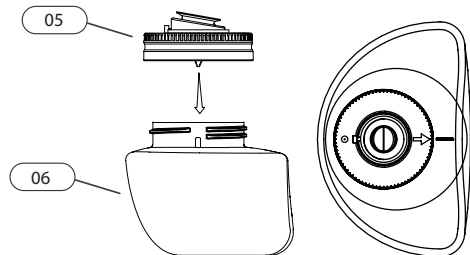
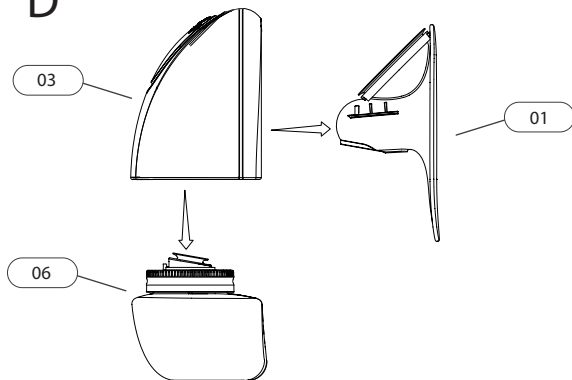
EN

- E. If the standard 25 mm (01) breast shield is too large for you, you can choose the 15 mm (09), 17 mm (08) or the 21 mm (07) silicone insert, which can be placed in the center of the breast shield (01).

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


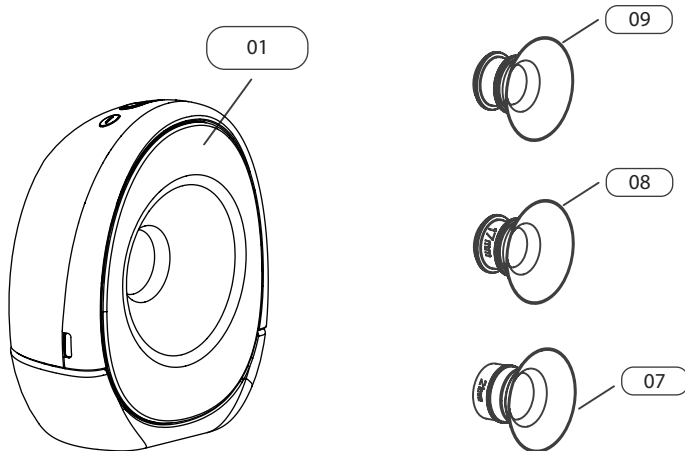
EN Correct alignment of the nipple in the breast shield is essential for comfort, effectiveness and performance. Your nipple must be correctly positioned in the breast shield tunnel before you start pumping to avoid discomfort.

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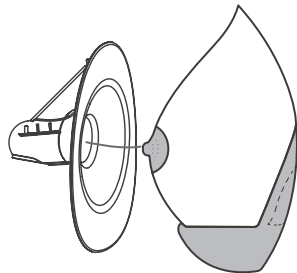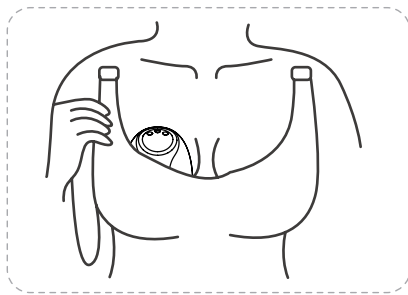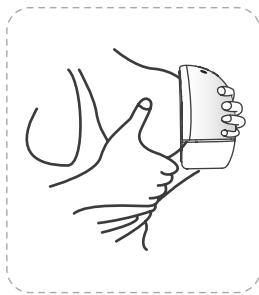EN Find the right compression

The Wearable Breast Pump is designed to be worn with your favorite nursing bra. You can use it in both sitting and standing position. Finding the right compression may affect the amount of milk produced. The breast pump should be pressed firmly against your chest, there should be no gaps between the device and your chest.

It is important to have good compression between the chest and the breast pump. Each bra offers a different level of support, so make sure to set the correct compression for each session. The included bra adjuster can be used to create more space in your bra. After the buckle is attached to the fastener from each end, adjust the length of the bra adjuster for more comfort.

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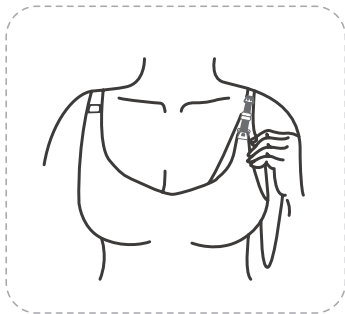
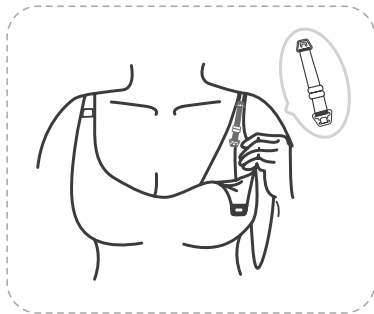
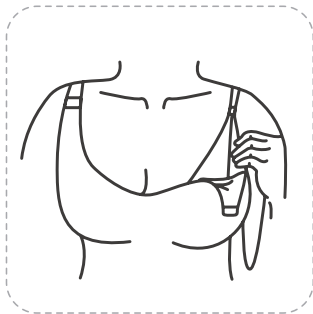
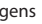
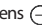
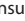
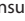
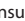
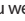
Button	Operation instruction	Display
	<p>Press  for 2 seconds to turn the device on / off. Press  briefly to select the working mode.</p> <p>The default function after startup is massage, the most recent intensity used is remembered for the next use.</p>	
	<p>Press  to increase or decrease the compression.</p> <p>The massage mode contains 5 operating levels. The pump mode contains 10 operating levels. The stimulation mode contains 10 operating levels.</p>	
	<p>Briefly press  to switch to Massage mode </p> <p>Then adjust  to increase or decrease the levels depending on your personal preference.</p> <p>After 2 minutes of massage, it automatically switches to pump mode.</p>	
	<p>Briefly press  to change to Pumping Mode  ;</p> <p>Then adjust  to increase or decrease the compression depending on your personal preference.</p>	
	<p>Briefly press  to change to Stimulation Mode  ;</p> <p>Then adjust  to increase or decrease the levels depending on your personal preference.</p>	
	<p>Briefly press  to change to Retraction Mode  ;</p> <p>Then adjust  to increase or decrease the vacuum strength depending on your personal preference.</p>	

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
EN When you have finished your session, disconnect the breast shield (01) and then unscrew the bottle (06) from the pump motor (03). You can store your breast milk in the bottle using the lid (12) that is included, or you can pour the breast milk into a container of your choice for storage.

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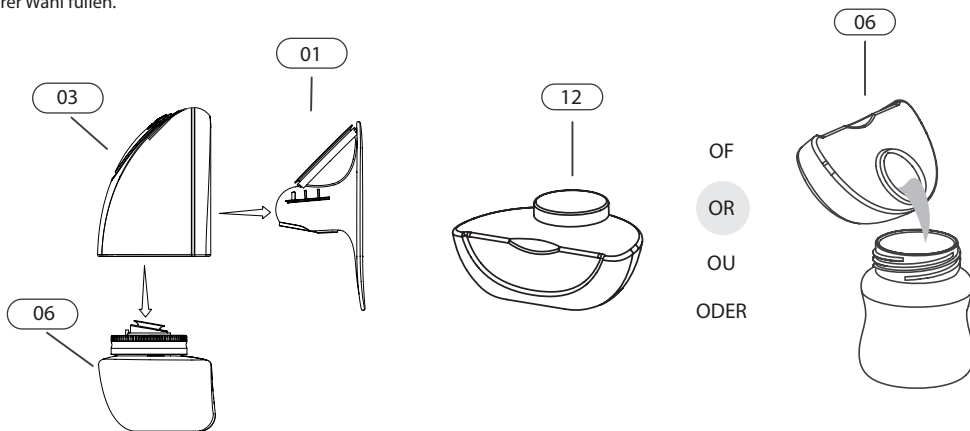SAFETY INSTRUCTIONS

Warning

- This product is for a single user. Usage by more than one person may cause health risks and will void the warranty.
- It is recommended to consult a licensed healthcare professional before using this portable breast pump if you have an active implanted device affected by a magnetic field (e.g. A pacemaker or an implantable cardioverter defibrillator).
- Inspect parts for damage, clean and sterilize the washable parts that come into contact with the breast and breast milk before using them for the first time.
- The device may heat up during use. If the device reaches the upper temperature limit (+30°C/+ 86°F) during the maximum session time (30 minutes), the temperature of the cushion can rise to + 45°C/+113°F.
- Dry the parts thoroughly. Mold may form due to residual moisture.
- Do not immerse the pump motor in water or other liquids. Clean the pump motor with a clean, damp cloth.
- This product contains small parts and a cable that may present a hazard for children. Keep out of reach of children.
- Only use the attached parts with this device. This also applies to the charging cable, do not substitute.
- It is recommended to use an adapter that is compatible with both IEC60601-1 and your regional power source.

Safety

- Always check all parts for damage before use and make sure they are clean.
- Always make sure that all parts are completely dry before assembling them.
- Do not store the pump motor in the refrigerator or freezer.
- This portable breast pump and its parts are not heat resistant. Keep away from heat sources.
- Do not use flammable materials while using this portable breast pump or while connecting it to a charger.
- Do not use this device while sleeping or if you are sleepy.
- Using this device should not cause pain. Do not try to force a too high or uncomfortable intensity. If excessive discomfort or pain occurs during pumping, end the session and remove the breast shield around the nipple.
- Contact your healthcare professional if you produce little or no milk, or if the expression is painful.
- For assistance with the correct size and comfort of the breast shield, see section 4.3 or consult a lactation or breastfeeding specialist.
- Extra caution is advised when using this portable breast pump near children.
- This portable breast pump should not be used while lying down, or when doing heavy exercises or activities at risk of injury.
- Always break the vacuum seal around your nipple before removing this portable breast pump from your breast.
- Do not expose this portable breast pump and parts to direct sunlight.

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EN

Power	5V /1A
Battery type and capacity	Li-ion Polymer Battery Specification: 1300mAh
Massage speed	80~130 CPM (Cycle Per Minute)
Suction speed	26~110 CPM (Cycle Per Minute)
Massage vacuum	0.007~0.016Mpa (+/-0.003Mpa), 5 levels adjustable
Pumping vacuum	0.011~0.033Mpa (+/-0.003Mpa), 10 levels adjustable
Stimulation vacuum	0.011~0.033Mpa (+/-0.003Mpa), 10 levels adjustable
Retraction vacuum	0.011~0.025Mpa (+/-0.003Mpa), 3 levels adjustable
Noise	< 55 dB(A)

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MORE INFO

www.Luvion.com

Luvion Premium Babyproducts
Delft, The Netherlands
Visit us on www.Luvion.com