TIKKENS ALCOHOLTESTER

Model: EK-923

1. Introductie

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België

1. Introduction

Thank you for your purchase! Read this manual attention before using the alcohol tester.

If you have any questions, you can always contact us by email: info@northwallproducts.com

2. Warning

Drinking and driving don't mix. The measured value is only an indication and does not confirm the driving skills. The digital alcohol tester can occasionally be a display incorrect and unreliable value.

Test results can never be used as legal evidence presented in the event of a conflict with the law. Law enforcement uses a different, professional one technology. This Tikkens alcohol tester is only designed for private use and NOT suitable for commercial use

3. Important!

A private alcohol test is never as accurate as a alcohol test with the high-quality equipment of the police. Use the results as a guideline only and never compromise alcohol behind the wheel. Take the manual carefully before use. Also read the tips below!

The blood alcohol level can be up to two hours after consumption of alcoholic beverages continue to rise. A permanently negative

4. Units of Measure

Select for the Netherlands & Belgium: EU (g/L):

EU - g/L: How many grams of alcohol are present in a liter of blood.

US - %BAC: How many grams of alcohol are present in 100 ml blood.

USA - ‰BAC: How many grams of alcohol are present in 1000 ml blood.

CN - mg/100mL: How many milligrams of alcohol are present in 100 ml of blood.

PL - BrAC: Breath alcohol content (the percentage of alcohol in the breath).

JP - mg/L: How many milligrams of alcohol are present in a liter blood.

5. Instructions

- Do not drink or smoke at least 20m before performing the test.
- Only perform the test at a temperature of 0°C to 4°C.
- Never blow smoke, liquids or food residues into the product.
- Conditions that contain an increased concentration of ketones **the blood such as diabetes or reduced calorie intake** ingestion through fasting can cause different results.
- Never clean the device with chemical cleaning agents.
- Keep the alcohol tester out of the reach of children.
- If the deviation between two successive positive measurements $> 0.05\text{mg/L}$, the test must be repeated.
- Attention! Wait at least 3 min. between two measurements

6. Package Contents

- 1 x Tikkens digital alcohol tester
- 10 x mouthpieces
- 1 x micro USB cable
- 1 x manual
- 1 x carrying case

7. Specifications

- Brand: Tikkens
- Material: ABS
- Colour: Grey
- Weight: 125g
- Dimensions: 138 x 58 x 21mm
- Certificates: CE, RoHS, FCC
- Battery: 1000mAh
- Input: 5V, 1A
- Output: 3.7V
- Note: This alcohol tester does not comply with the European NF X20-702 standard and is only suitable for personal use usage.

8. Test Range & Accuracy:

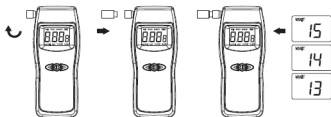
- 0~0.199%BAC(Deviation $\pm 0.01\%$ BAC)
- 0~1.99%BAC(Deviation $\pm 0.1\%$ BAC)
- 0~1.99g/L(Deviation $\pm 0.1\text{g/L}$)
- 0~0.995mg/L(Deviation $\pm 0.05\text{mg/L}$)
- 0~199mg/100ML(Deviation $\pm 10\text{mg}/100\text{mL}$)
- 0~1.99%Prom(Deviation $\pm 0.1\%$ Prom)

9. Operation - Unit setting

- Press the power button (1) for two seconds. In the upper left corner the word "WAIT" will appear followed by a count down timer from 15s. In these 15s. press the left button (2) and select the desired value (g/L for NL & BE).
- After selecting the correct value, press the power button to fix the value.

10. Operation - Alcohol test

- Press the power button (1) for two seconds. In the upper left corner the word "WAIT" will appear followed by a count down timer from 15s. This is the warm-up phase.

- After the warmup, the word "START" appears. Start blowing within 7 sec. Blow gently and continuously for 4-5 sec.
- Green screen: Value lower than the warning value. The device will beep once indicating a negative test.
- Yellow screen: The device turns yellow at a value between warning value and the danger value. The device beeps twice with a one s. interval.
- Red screen: The device turns red when the value is higher than the hazard value. The device beeps for 10 s.

Attention! Wait at least 3 min. between two measurements

11. Contact

You can always contact us for questions about Tikken's alcohol tester by email. We are happy to assist you and will do everything in our power to find a solution.

Email: info@northwallproducts.com
Wallcorp BV
Heikesveld 48 box 2
3910 Pelt
Belgium

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