



32dB
SNR



1. Zorg dat de oordopjes schoon zijn voordat u ze inbrengt. Houd het uiteinde van het oordopje tussen uw duim en wijsvinger.

2. Gebruik uw beschikbare hand om over uw hoofd te reiken en trek de bovenkant van het oor voorzichtig naar buiten en omhoog om het inbrengen te vergemakkelijken. Hierdoor wordt de gehoorgang rechtgetrokken zodat het oordopje er gemakkelijk en effectief in past.

3. Duw en draai het oordopje voorzichtig totdat het comfortabel in de gehoorgang past.

Steek het oordopje NIET te ver in en zorg ervoor dat het lipje van het oordopje voldoende uitsteekt zodat u het dopje na gebruik gemakkelijk kunt verwijderen.

Waarschuwing:

Uit de buurt houden van baby's en kinderen jonger dan 3 jaar. Onjuiste plaatsing of veronachtzaming van bovenstaande aanbeveling verminderde effectiviteit en kan leiden tot gehoorverlies of [etsel]. Laat het oordopje nooit boven de 55°C uitkomen. Voor het beste resultaat plaatst u de oordopjes een uur voor de landing. Oordopjes mogen op maximale hoogte worden verwijderd, maar altijd een uur voor de landing inbrengen.



1. Before inserting the earplugs, make sure they are clean. Hold the end of the earplug between your thumb and index finger.

2. To ease insertion, use your available hand to reach over your head and gently pull the top of the ear outward and upward. This will straighten the ear canal to easily and effectively fit in the earplug.

3. Gently push and twist the earplug until it seals comfortably in the ear canal. DO NOT over insert the earplug and ensure the tab of the earplug is protruding enough for you to easily remove the plug after use.

Warning:

Keep away from infants and kids below 3 years old. Improper fitting or disregard of the above recommendation will reduce effectiveness and could result in hearing loss or injury. Never let the earplug beyond 55°C. For best results insert one hour before landing. Earplugs may be removed at maximum altitude but always insert one hour before landing.

Minute wholesale
Raasdorperweg 191
1175 KV Lijnden
info@minutewholesale.nl

CE 2834

EN 352-2: 2020 (EU) 2016/425