EN This leaflet contains important information. It is worth taking the time to read it carefully.

If they are used properly, condoms help to reduce the risk of catching HIV (AIDS) and many sexually transmitted infections. No method of contraception can provide 100% protection against pregnancy or the transmission of HIV and STIs. If you are worried that you or your partner may have become pregnant or infected, contact your doctor immediately.

Using Durex condoms: Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Usage advice: Unless stated otherwise, these condoms are made from Natural Rubber Latex which might cause allergic reactions including anaphylactic shock if the user is allergic to latex. If you think you might be sensitive or allergic to latex, get advice from your doctor before using a condom.

Durex polyisoprene condoms are a synthetic alternative to natural rubber latex, and are considered suitable for people who are sensitive to natural rubber latex (however you may still experience an allergic reaction). Our synthetic condoms are Durex Real Feel or Durex Latex Free.

Use only lubricants recommended for use with condoms. Oil-based ones (petroleum jelly, baby oil and some pessaries) and some topical medicines that are applied to the penis or vagina can damage condoms. **STOP USE** if you feel discomfort or irritation while using condoms.

For anal sex use additional lubrication (do not use Durex Tingle or Embrace) on the outside of the condom. Please do NOT use Durex Climax Durex Intense or Durex Tingle Me condoms for anal sex. If you use a condom for anal sex first, you should use a new condom for any other type of sex that follows. Use a condom only once. Reuse may increase risk of failure or infection.

For Durex condoms containing benzocaine (Durex Extended Pleasure and Durex Mutual Climax): Take particular care when using condoms with benzocaine. Only use one benzocaine condom per sex act. Do not use these condoms if you or your partner has irritated or broken skin. If you or your partner experiences breathing difficulties, or get blue lips when using these condoms, stop using them immediately and call a doctor. Take care to not get the lubricant in your eyes, and wash it off after use.

Premature ejaculation is a common problem, which can have many causes and may require medical consultation/supervision. If these condoms don't help prolong your sexual experience, it's worth checking with your doctor whether any further action may be recommended.

Consult a doctor or pharmacist if you are using topical medicines applied to penis or vagina; irritation/discomfort persists after use; you are unsure if you are allergic to latex; a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

Store in a cool dry place away from direct sunlight.

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1

EN Handle the condom carefully, as it can be damaged by fingernails and sharp objects.

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3

EN Roll down to its base. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage. Soon after ejaculation the condom should be taken off. The condom should be held firmly in place at the base of the penis before withdrawal.

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