mr Safe

QUICK REFERENCE GUIDE

GB NL

FR D

DIGITAL ALCOHOL TESTER

QRG / MrSafe Alcohol Tester K5 / 11-16 / V01
Copyright©Mr Safe

1. IMPORTANT INSTRUCTIONS

! DO NOT DRINK AND DRIVE !
Also never let anyone else drink and drive or operate machinery.
Always use a designated driver to drive you safely!

! WAIT at least 20 minutes, without drinking or smoking before testing. Rinse your mouth with water before testing!

! The alcohol level in your blood can keep increasing up to 2 drinking your last alcoholic beverage. A continuous negative test-result cannot be guaranteed !

! ONLY perform tests in a temperature range between 10-40°C as this may affect the results !

! DO NOT blow smoke, liquid or food residue into the device !

! Conditions that increase the amount of ketones on your breath, such as diabetes or low calorie intake may cause false results !

! This test and your test-result cannot be held as juridical proof in case of law encounters !

The BAC results are for reference only and the test-results cannot be used as a tool to determinate whether you should operate a vehicle or not !
The manufacturer, distributor and importer disclaims any legal responsibility for the measured test-result.

2. PARTS & BUTTONS

A: Mouthpiece Slot
B: Digital LCD Display
C: LEFT button (Select / Begin Test)
D: POWER button (OK / Confirm)
E: RIGHT button (Recall last record / Access Menu)
F: Battery Compartment
G: Mouthpiece Holder (holds 2 pieces)

3. INSTALLING THE BATTERIES

Slightly press down the battery cover and slide it down the device.
Insert 2 X AA Alkaline batteries according to the scheme visible.
Slide the cover back in place until you hear it click.

4. ON / OFF

Press the POWER button (D) for 2 seconds to turn on the device.
You will hear a beep and see the current temperature displayed on the LCD Display (B).
Note: The device will automatically turn off after 30 seconds of inactivity

To turn off the device, press the POWER button (D) again for 2 seconds.

5. NAVIGATING THE MENU

When turning on the device, the LCD Display (B) will show the current temperature.
To enter the menu, press the RIGHT button (E) for 2 seconds.

There are 3 main structures:
USR: settings for user comfort (see chapter 7)
RCD: Display test result record (see chapter 8)
CAL: Calibration (not for end-user)
SCR: (not for end-user)

To navigate through the menu, press the RIGHT button (E).
To exit the menu, press the RIGHT button a few times until you see the current temperature again.

6. USER SETTINGS

To change the user settings, enter the menu as described in chapter 6. When "USR" is visible on the screen, press the POWER button (D) to enter the menu. You can now change the following settings:
Note: to go to the next setting without changing anything, simply press the POWER button (D) to confirm.

- Setting the Temperature format: Use the LEFT button (C) to change between °C or °F. Confirm with the POWER button (D)
- Set BAC display format: Use the LEFT button (C) to change between ‰BAC, ‰BAC or mg/l. Confirm with the POWER button (D)
- Set ALERT level: Use the LEFT button (C) to change between 0.30 / 0.50 / 0.80 ‰BAC. Confirm with the POWER button (D)
- Enable/Disable beep-sound when blowing: Use the LEFT button (C) to change between 1(=ON) and 0(=OFF). Confirm with the POWER button (D)
To exit the User Settings menu, press the RIGHT button (E).

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GB

7. RECALL YOUR PREVIOUS TEST RESULTS

The device stores the test results of your last 10 tests. To recall them on the display, press the RIGHT button (E) for 2 seconds to go into the menu structure. Press the RIGHT button again until "RCD" is visible on the display and press the POWER button (D) to confirm.
A corresponding number (1-10) will be visible on the screen and the test-result will follow. To navigate between the numbers press the POWER button (D). To exit the memory, press the RIGHT button (E).

To recall your last test-result, simply press the RIGHT button (E) when the current temperature is visible. When you press the RIGHT button (E) again, you will see the number of tests the device has made.

8. TESTING YOUR ALCOHOL LEVEL

STEP 1
Fold out the mouthpiece on the upper left side of the unit. Add an additional mouthpiece (optional) if necessary.

STEP 2
 Press the POWER button (D) for 2 seconds to turn on the device. You will hear a beep and see the current temperature displayed on the LCD Display (B).

STEP 3
 Initialize the sensor by pressing the LEFT button (C). The sensor will start its warm-up sequence and auto-cleansing process. On the LCD Display (B) you will see a timer countdown from 99-0

STEP 4
 Wait until "RDY" (Ready) appears on the LCD Display (B) and you will hear a beep to indicate that the device is ready to use

STEP 5
 Take a deep breath and blow steadily (softly, continuously and evenly) into the mouthpiece. "BL1-5" will appear on the display to indicate breath-analysis. Blow until you hear a beep sound to indicate the completion of the testing. The BAC reading will be displayed.

Note: If "FLO" appears on the screen it indicates that your blowing was less than 5 seconds. Restart the test again, blow softly and continuously again until you hear the beep-sound

STEP 6
Reading the result:
The reading will be in range from 0.00 to 5.00 ‰ (or 0.000 to 0.500 ‰BAC or 0.000 to 2.500 mg/l) depending on how the BAC display is set.
GREEN Background: Safe (negative) – Between 0.00 – 0.10‰BAC
YELLOW Background: Alert (positive) – Between 0.10‰BAC and the alert level set by user
RED Background: NOT Safe (positive) – Exceeds alert lever set by user

Note: If you want to take several tests, wait at least 1 minute with normal results, 3-5 minutes with high results.

9. ERROR NOTIFICATIONS

Sometimes the display can show an error notification:
REP: Insufficient or uneven blowing during test detected. Repeat the steps in chapter 9
SNR: Sensor not ready: this means that the device is not ready and that it needs another warm-up / auto-cleansing. Repeat the steps in chapter 9
Note: When the device has not been used for an extended period of time, the sensor might be polluted and the display will show the SNR notification. Let the device warm-up and auto-cleanse for a few times to make the pollution go away.

FLO: Not enough sample collected due to insufficient blowing. Repeat the steps in chapter 9
H: HIGH: Alcohol concentration is over 5.00‰BAC. Wait 15-30 minutes before trying again
Err: The sensor is severely damaged and cannot be used anymore. See "Support" for more details
LQ: Battery low, please replace the 2 AA Alkaline batteries in your device.

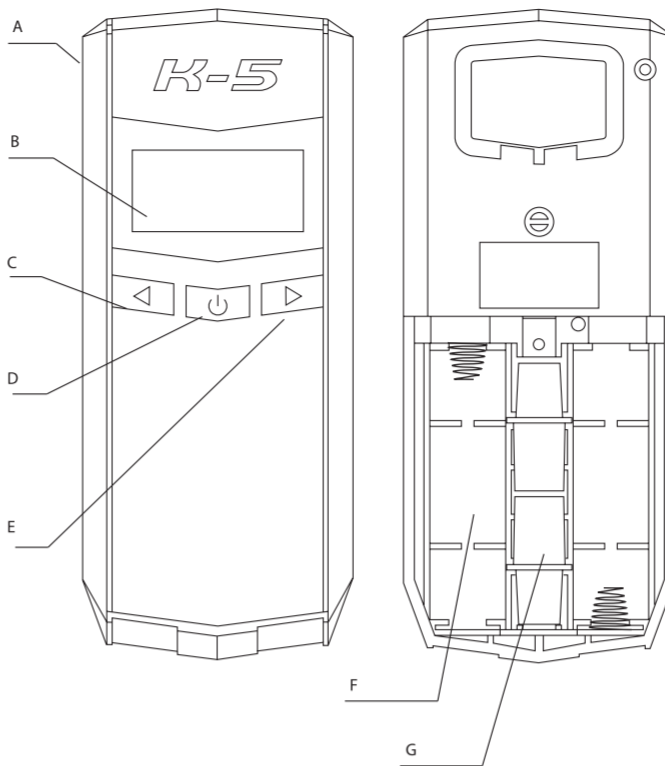
10. CUSTOMER SUPPORT

For all questions and additional information concerning this product, please contact support@mrsafe.eu.

11. GUARANTEE

Copyright © mr Safe®. mr Safe® is a registered trademark of TE-Group NV. The mr Safe® brand stands for superior quality products and excellent customer service. Therefore mr Safe® warrants this product to be free from defects in materials and workmanship for a period of two (2) years from the original purchase date of the product.

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