Digital Alcohol Tester InnovaGoods® - Gadget

who is responsible for their safety. This product is not household waste, so follow the environmental standards for recycling.

InnovaGoods®

User's Manual

ENGLISH

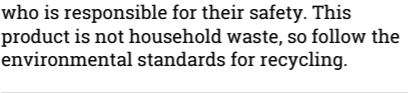
INSTRUCTIONS




- Put the 2 AAA batteries in the compartment and press the power button. Wait for 30-40 seconds.
- When Blow appears on the screen of the Digital Alcohol Tester InnovaGoods® is ready to use.
- A countdown will appear. Blow for 4-5 seconds continuously before the countdown ends.
- The Digital Alcohol Tester InnovaGoods® will emit a long whistle when the sample is sufficient, except when no alcohol has been consumed, when it will emit a short whistle when the countdown finishes.
- The result of the test will show on the screen for 15 seconds, accompanied by a colour-coded system which will change depending on the result of the test (green, amber, red).
- To change the unit of conversion (%BAC, %OBAC y mg/l BRAC) press the power button.
- For greater certainty, repeat the test 3 or 4 times.
- The Digital Alcohol Tester InnovaGoods® will turn itself off automatically after showing the result.
- When the word Lo appears on the screen, change the batteries.

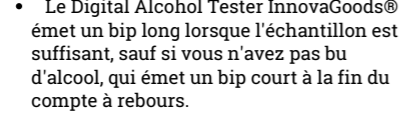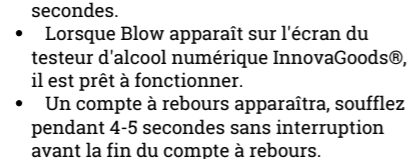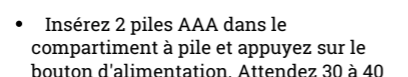Do not use the device immediately after smoking, consuming alcohol and/or food. Do not mix new and used batteries in the compartment. If you are not going to use the unit for a long period of time, remove the battery. Do not expose batteries to high temperature or direct sunlight. Do not attempt to open them. do not dispose of them in fire, there is risk of explosion. Clean the device with a slightly damp cloth. Do not use abrasive products or materials to clean it. Do not immerse in liquid. Do not handle it with wet hands. Do not use the device close to a source of water or in damp places. Suitable for domestic use only. Use the device for its intended purpose only. Store the product in a dry place with no dust. To avoid risks, do not use it if any parts are damaged and do not attempt to disassemble or repair it. This device is not designed for use by children or people with physical and mental disabilities, unless they are fully supervised by an adult

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ALL RIGHTS RESERVED. MADE IN P.R.C.