
## INSTRUCTIONS FOR USE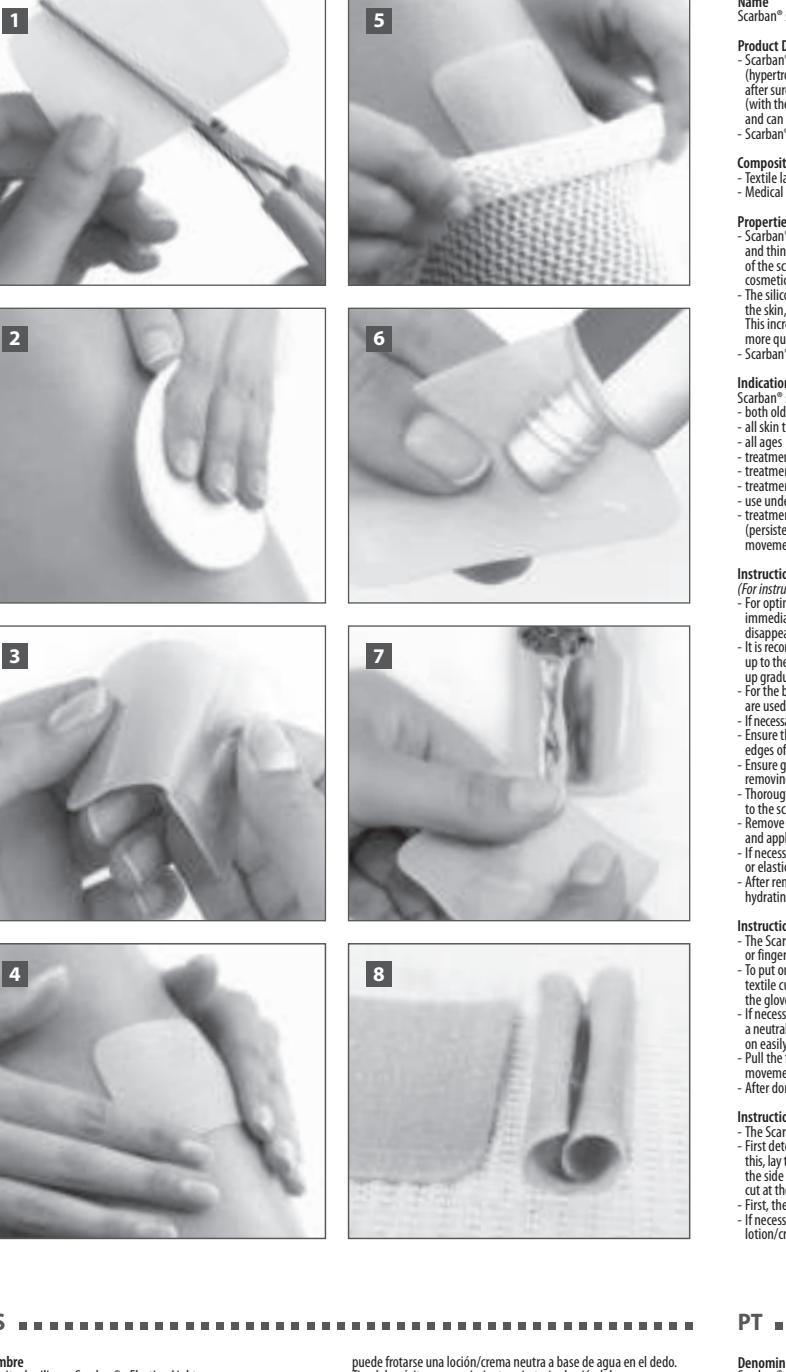

## EN

**Name:**  
Scarban® silicone sheets - Elastic and Light

### Product Description

Scarban® silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets glove and finger), washable and can be used for a long period of time.

### Care Instructions for Scarban® silicone sheets

(For instructions of glove and finger, see below)

- It is essential to clean the Scarban® silicone sheets properly, on a daily basis. This is necessary to ensure good adhesion.

- Rub the silicone sheet with a few drops of Scarban® Silicone Cleaning Soap or a neutral soap (fig. 6).

- Remove as much dirt as possible with soap and lukewarm water.

- Rub the silicone sheet thoroughly under lukewarm running water (fig. 7).

- After rinsing, dry the silicone sheet with a soft cloth or towel.

- If adhesive is still present, repeat the silicone sheet will no longer be adhesive and it will roll up (fig. 8).

- Leave the silicone sheet to dry on a clean, dry surface, with the sticky side facing upwards (fig. 8).

- While drying, the silicone sheet will unfold and its adhesive strength will gradually return.

- Use the silicone sheet only after it has completely dried.

- Trim edges of silicone sheets that are no longer adhesive, due to wear and tear.

**Care instructions for Scarban® silicone sheets: glove and finger**

(For instructions on how to use the glove and finger, see below)

- For optimal results preferably start using Scarban® silicone therapy immediately after wound closure (after removal of sutures and the dressing).

- It is recommended that Scarban® silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.

- For the best possible result, it is preferable that the silicone sheets are used for 12 hours and a maximum of 23 hours per day.

- If necessary, Scarban® silicone sheet can be cut to the correct size (fig. 1).

- Ensure that the silicone sheet extends two centimeters beyond the edges of the scar.

- Ensure good adhesion and durability of the silicone sheet by:

- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it (fig. 2).

- Apply Scarban® silicone sheet only on healthy skin (fig. 3).

- Use adhesive or elastic bandage (fig. 5).

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

**Instructions for use – Scarban® silicone sheets**

(For instructions on how to use the glove and finger, see below)

- To optimise results preferably start using Scarban® silicone therapy immediately after wound closure (after removal of sutures and the dressing).

- It is recommended that Scarban® silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.

- For the best possible result, it is preferable that the silicone sheets are used for 12 hours and a maximum of 23 hours per day.

- If necessary, Scarban® silicone sheet can be cut to the correct size (fig. 1).

- Ensure that the silicone sheet extends two centimeters beyond the edges of the scar.

- Ensure good adhesion and durability of the silicone sheet by:

- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it (fig. 2).

- Apply Scarban® silicone sheet only on healthy skin (fig. 3).

- Use adhesive or elastic bandage (fig. 5).

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

**Instructions for use – Scarban® silicone sheet: glove**

(For instructions on how to use the glove and finger, see below)

- To optimise results preferably start using Scarban® silicone therapy immediately after wound closure (after removal of sutures and the dressing).

- It is recommended that Scarban® silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.

- For the best possible result, it is preferable that the silicone sheets are used for 12 hours and a maximum of 23 hours per day.

- If necessary, Scarban® silicone sheet can be cut to the correct size (fig. 1).

- Ensure that the silicone sheet extends two centimeters beyond the edges of the scar.

- Ensure good adhesion and durability of the silicone sheet by:

- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it (fig. 2).

- Apply Scarban® silicone sheet only on healthy skin (fig. 3).

- Use adhesive or elastic bandage (fig. 5).

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

**Instructions for use – Scarban® silicone sheet: Finger**

(For instructions on how to use the glove and finger, see below)

- First determine the length of the silicone sheet that is needed. To do this, lay the finger on the sheet. There are several colored markings on the side of the sheet. If the sheet is too long, the silicone sheet can be cut to the correct size (fig. 1).

- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily on the finger.

- If necessary, the finger can be rubbed with a neutral, water-based lotion/cream to help it slide easily on the finger.

**Date of latest revision of instructions for use**

January 2021

**Date of the last revision of the instructions of use**

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### Composition

- Textile layer: polyamide, elastane  
- Medical grade silicones

### Properties

- Scarban® siliconen sheets zorgen ervoor dat scars worden flatter, soemer en thinner en reduceert compliant (like persistent redness, pulling of the scar, itching, reduction of points, pijn, comfort and cosmetic concerns).

- De siliconen sheet provides a semi-elastic (stretchy) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and if functional objections (complaints) are reduced.

- Scarban® silicone sheets provide UV protection (UPF50) as standard.

### Indications/applications

Scarban® siliconen sheets zijn geschikt voor:  
- both old and new scars  
- all skin types  
- treatment and prevention of abnormal scars (hypertrophic and keloid)  
- treatment of burn scars  
- treatment of surgical scars  
- use under clothing and in compression therapy  
- treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, reduction of joint movement, discomfort and cosmetic concerns)

### Instructions for use – Scarban® silicone sheets

(For instructions on how to use the glove and finger, see below)

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