

# Smart Eye Massager

## User Manual



The product is for the left eye only. However, it is designed with the right eye in mind. Please use the right eye as a reference when using the device.

### 1. Operation Guide


#### Bluetooth Version



- 1. Power: Long Press ON/OFF  
Short Press Switches Modes
- 2. Battery: LED Shining with no power  
LED Shining when Charging  
LED Lighting when full charging
- 3. Heat
- 4. Air Compressor
- 5. Heating
- 6. Vibration

Mode	ON	Heating	Vibrating	Bluetooth	Timer
Energy Mode	✓	✓	✓	✓	10
Relax Mode	✓	✓	✓	✓	15
Relieve Mode	✓	✓	✓	✓	15
Sleep Mode	✓	✓	✓	✓	20

#### Music Version

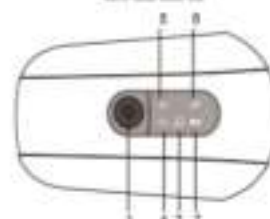

- 1. Power: Long Press ON/OFF  
Short Press Switches Modes
- 2. Battery: Shining with no power  
Shining when Charging  
Lighting when full charging
- 3. Heat
- 4. Air Compressor
- 5. Heating
- 6. Vibration

Mode	ON	Heating	Vibrating	Music	Timer
Energy Mode	✓	✓	✓	✓	10
Relax Mode	✓	✓	✓	✓	15
Relieve Mode	✓	✓	✓	✓	15
Sleep Mode	✓	✓	✓	✓	20

### 2. Product Function

- Buzzing Alarm
- Multiple Vibration Function
- 42°C Heating
- Single button operation
- Type-C port
- Buzzing sounds every 2 minutes and LED Indicator shining blue if low power. It will be turned off meanwhile sounds like Di Di Di when out of power.
- Four Modes: Energy Mode, Relax Mode, Relieve Mode, Sleep Mode.

### 3. If you

- Are pregnant
- Are strabismic over 200 degrees
- Are seriously glaucoma patient
- Are under 7 year-old
- Are allergic to electricity
- Are eye injury in the period of rehabilitation of strabismic surgery (strabismic eye)
- Have Glaucoma, cataract, retinal detachment and other serious ocular health surgery
- Are suffering from serious eye inflammation

We advise you to consult with your doctor before using this device.

### 4. Safety Note

Attention: This device is designed for private use, not intended for medical or commercial purposes. Children should be supervised to ensure that they do not use this device as a toy.

- Suggested to Adjust elastic suitable and fit bearing against the soft or skin when using. That will be more comfortable to use.
- Suggested to use this device one or two times per day. Each time should be 10-20 minutes. And do not open your eyes when using this device.
- Please using this device after charging.
- Please cleaning surface of this device with wet towel or cotton ball with 75% alcohol alcohol.
- Do not put this device close fire, cigarette, light, and other adverse environments.

### 5. Product Specifications

- Name: Smart Eye Massager
- Input: DC5V/1A
- Rated Power: 5W
- Battery: Built-in Li-ion battery 1.7x1200mAh