 SILVERGEAR

BLOOD PRESSURE METER

Thank you for purchasing the Silvergear blood pressure meter. This manual will provide directions for effective use of the meter. Please read the manual carefully before using this product. Store the manual in a safe place for future reference.

This device uses the oscillometric method to measure the arterial blood pressure and pulse rate. The arm band is wrapped around the arm and is automatically inflated by an air pump. The sensor of the device catches weak fluctuation of the pressure in the arm band produced by extension and contraction of the artery of the arm in response to each heartbeat. The amplitude of the pressure waves is measured, and is displayed by digital value. This device is easy-to-use and accurate, will facilitate your daily blood pressure regimen.

Table of contents

1. Package contents
2. Technical data
3. Safety instructions
4. Description of parts
5. How to use
6. Maintenance and storage
7. Troubleshooting
8. Warranty
9. Disposal
10. Appendix A. EMC Information

1. Package contents

- Monitor unit
- Arm band
- Pouch
- Manual

2. Technical data

Product name	Silvergear blood pressure meter
Work method	Auto-inflation; Auto-measurement
Measurement method	Oscillo metric method
Display	LCD Digital display
Cuff pressure range	0~ 280mmHg (0.0 ~37.3kPa)
Pulse range	40~ 199 beats/ min
Measurement range	DIA:40-260mmHg (5.3~34.7kPa)
SYS:	40-260mmHg (5.3~34.7kPa)
Over-pressure protection	280mmHg (37.3kPa)
Pressure indication error	Pressure: $\pm 3\text{mmHg}(\pm 0.4\text{kPa})$
Pressure measurement	Pressure transducer
Power source	4 "AA" alkaline batteries (DC1.5V*4)
Content	Monitor unit, arm cuff, instruction manual
Monitor Unit Dimension	140mm×110mm×81 mm
Weight	280g (excluding battery
Cuff circumference	23~ 33cm
Memory	2X192
Operating temperature/ humidity/air pressure	+5~+ 40°C / 15%~ 85% RH/ 86~ 106kPa
Storage temperature/ humidity/air pressure	-20~55°C / 15%~ 93%RH/ 86 ~ 106kPa
Applied part	 = Type BF
Protection against electric shock	Internally powered ME equipment (Only using the batteries)  = Class II ME equipment (Adapter*) *Not included
Expected service life	5 years
Software version	001

3. Safety instructions

- Patients with severe arrhythmia are prohibited from using this product.
- Children are prohibited from using this product.
- Warning! Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. (General usage)
- DO NOT adjust medication based on measurement results from this blood pressure monitor.
- Take medication as prescribed by your physician. Only a physician is qualified to diagnose and treat high blood pressure.
- This device is not intended to be a diagnostic device. Always consult your physician.
- Self-diagnosis of measurement results and self-treatment are dangerous.
- Do not use the device on an injured arm or an arm under medical treatment.
- Do not use the device in the case of flammable anesthetic gases mixed with air or flammable anesthetic gases mixed with oxygen or nitrous oxide.
- Consult your physician before using the device for any of the following conditions: If you have had a mastectomy, common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, age, pregnancy, pre-eclampsia or renal diseases.
- People with severe blood flow problems or blood disorders as cuff inflation can cause bruising. Acute internal bleeding may be caused by compression of the arm during measurement.
- Do not take more measurements than necessary. It may cause bruising due to blood flow interference.
- Do not apply the arm cuff on the arm while being on an intravenous drip or blood transfusion.
- Consult your physician before using the device on the arm with an arteriovenous (A-V) shunt.
- Do not use the device in the area of HF surgical equipment, a MRI or CT scanner, or in an oxygen rich environment.
- Do not use the device with other medical electrical (ME) equipment simultaneously.
- The air tube or the adapter cable may cause accidental strangulation for children.
- Contains small parts that may cause a choking hazard if swallowed by children.

Caution! Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

(General usage)

- NOTE: motion, trembling or shivering may affect the measurement reading.
- Remove the arm cuff if it does not start deflating during the measurement.
- Each use time during the measurement process is about 2-6 minutes. Remove the arm cuff immediately after the measurement is complete (the measurement results appear on the screen and the cuff is deflated).
- Do not use this device on infants, children or persons who cannot express their intentions.
- Do not use the device for any purpose other than measuring blood pressure.
- Use only the special arm cuff provided by the manufacture for this device. Use of other arm cuffs may result in incorrect measurement results.
- Do not use in a location with moisture, or a location where water may splash on the device. This may damage the device.
- Do not use the device in a moving vehicle (car, airplane).
- Precision components were used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Avoid folding the cuff tightly or storing the tube in a tightly twisted position for long periods, as such treatment may shorten the life of the components.
- Measurements may be impaired if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields. The device and cuff are not water resistant. Protect the device and cuff from rain, sweat and water.
- Please keep the pressure of the arm cuff under 280mmHg (37.3kPa). Inflating to a higher pressure than necessary may result in bruising and numbness where the arm cuff is applied.
- Changes or modifications not approved by Silvergear will void the user warranty. Do not disassemble or attempt to repair the unit or components by yourself.
- Do not forcibly crease the arm cuff or the air tube excessively.
- Do not press the air tube while taking a measurement.
- Do not drop the device to strong shocks or vibrations.
- Do not inflate the arm cuff when it is not wrapped around your arm.
- Do not kink the connection air tube during measurement, otherwise the continuous cuff pressure may occur, and causing blood flow interference and harmful injury to the patient.
- Pressurization of the cuff can temporarily cause loss of function of simultaneously used monitoring equipment on the same limb.
- Make sure that the operation of the device does not result in prolonged impairment of the patient's blood circulation.

- Do not use the device outside the specified environment. It may cause an inaccurate reading.
- Dispose of the device, components, batteries and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- In case of pressure discomfort or pressure that doesn't stop, immediately turn off the unit or pull out the tube plug and quickly deflate.
- Avoid eating, drinking tea/caffeine/alcohol, exercising, bathing and smoking for 1 hour before taking a measurement. Rest for at least 15 minutes before taking the measurement.
- Please measure it in the correct position in a quiet, stable mood.
- Please do not speak or move your body during measurement.
- This product is only applicable to upper arm measurement and is not suitable for measurement of the wrist or other parts.

Battery usage

- Do not insert the batteries with their polarities incorrectly aligned.
- Silvergear does not provide the batteries, the users are required to buy batteries which meet the requirement of IEC 60601-1 medical device electrical safety standards. Use only 4 "AA" alkaline batteries with this device. Do not use other types of batteries. Do not use new and used batteries together.
- Remove the batteries if the device will not be used for three months or more.


Adapter (not provided) usage

- Do not use the adapter if the device or the power cord is damaged. Turn off the power and unplug the power plug immediately.
- Plug the adapter into the appropriate voltage outlet. Do not use in a multi-outlet plug.
- Never plug in or unplug the device from the electric outlet with wet hands.
- Fully insert the power plug into the outlet.
- When disconnecting the power plug from the outlet, do not pull the power cord. Be sure to pull from the power plug safely.
- Unplug monitor when not in use.
- Disconnect the power plug before cleaning.
- Silvergear does not provide the adapter, the users are required to buy adapter (DC 6V/500mA) which meet safety standards. Use of an adapter which does not meet the requirements may damage and/or may be hazardous to the device and users.

4. Description of parts

Contents: Monitor unit, arm cuff, instruction manual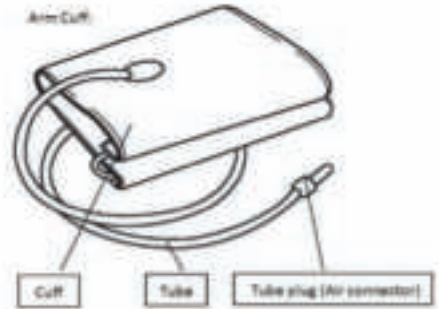


5. How to use

Before taking a measurement

1. Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for 1 hour before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. Measurements should be taken in a quiet place.
4. Remove tight-fitting clothing from your arm.
5. Sit on a chair with your feet flat on the floor. Rest your arm on a table so that the arm cuff is at the same level as your heart.
6. Remain still and do not talk during the measurement.
7. Keep a record of your blood pressure and pulse readings for your physician.
8. A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.

Preparation

Know the display information of your device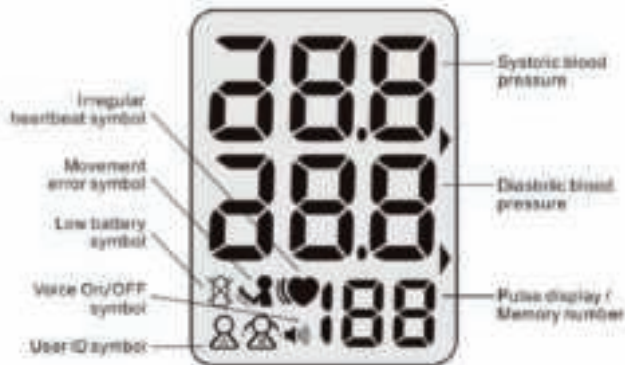



Display symbols

Irregular heartbeat symbol:

When the unit detects an irregular rhythm during the measurement, the Irregular heartbeat symbol will appear on the display with the measurement values. An irregular heartbeat rhythm is defined as a rhythm that is 25% less or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure. If the irregular heartbeat symbol shows up in the display, we recommend you consult your physician. Follow the directions of your physician.

Movement error symbol: The movement error symbol is displayed if you move your arm/body during the measurement. Please remove the arm cuff, wait a few minutes, and then take another measurement, remain still during measurement.

Blood pressure level indicator: The World Health Organization (WHO), the International Society for High Pressure (ISH) developed a blood pressure classification as shown below:

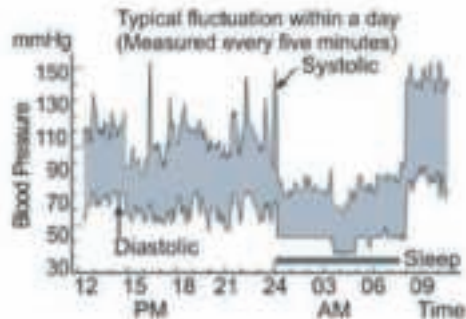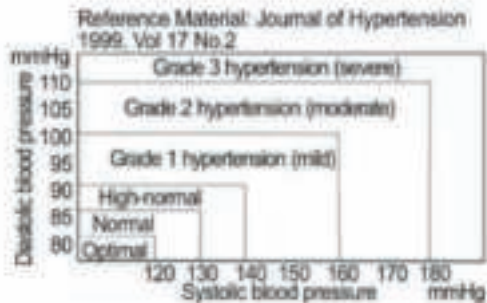Please note that other risk factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration and may affect these figures. Consult your physician for advice.

Note:

1. The graph is not exact, but may be used as a guide in understanding non-invasive blood pressure measurements. The device is only intended for use for adults.
2. Blood pressure measurements determined by this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, Manual, electronic, or automated sphygmomanometers.

Installing the battery

Note:

- Insert the batteries as shown in the battery compartment. If not inserted correctly, the device will not work.
- When the LOW BATTERY mark  appears on the display, turn the unit off, then replace all batteries with new ones at the same time. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- The measurement values continue to be stored in memory even after the batteries are replaced.


- The LOW BATTERY mark  does not appear after the batteries are drained.

Remove the battery cover and insert new batteries into the battery compartment as shown, make sure that the polarities (+) and (-) are observed. Silvergear does not provide the batteries, the users are required to buy batteries which meet safety standards. Use only 4 "AA" alkaline batteries with this device. Do not use other types of batteries. Do not use new and used batteries together.

- Remove the batteries if the device is not to be used for a long period of time (more than 3 months).

Otherwise, the batteries may leak and cause a malfunction.

Display Unit, Date, Time, Memory Clear and User Switch Setting

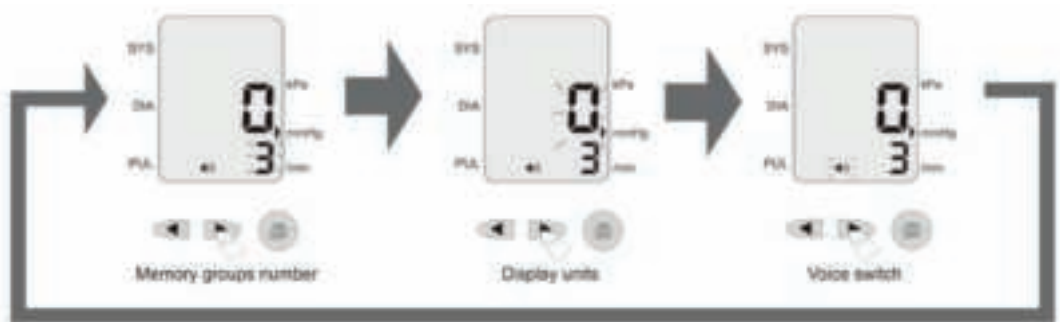This product provides two display units, mmHg and kPa (Pascal). This manual is illustrated in the unit of mmHg. The factory default setting is mmHg.

1. Shutdown Press and hold **[▶]SET** button (about 3 seconds) to turn on.

2. Press the **[▶]SET** button to cycle through the following settings.
* indicates the flashing state.

3. Memory Clear:

When the Memory Group Number symbol is flashing (as described in Step 2), press the **[▶]MEM** button to change the value to "0".

*Partial saved data cannot be selected for deletion. If you need to clear the memory data, all the measurement results of this blood pressure monitor will be deleted.

4. Display Unit Setting:

When the Display Unit symbol is flashing (as described in Step 2), press the **[▶]MEM** button to modify the value.

5. Voice On/Off Setting:


When the Voice On/Off Symbol is flashing (as described in Step 2), press the **[▶]MEM** button to choose the On/Off of the voice.

6. User Switch Setting:

In the shutdown state press the **[▶]SET** button, you can complete the A or B user switch.

7. Save the setup result:

Press the (On/Off) button to turn off the unit and save the settings successfully.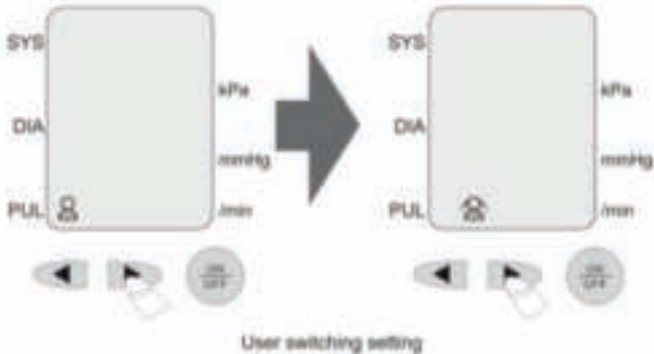

Measurement

1. Cuff tube connection:

Remove tight-fitting clothing or tight rolled up sleeve from your left upper arm. Do not place the arm cuff over thick clothes. Insert the air connector plug into the tube plug (air socket) firmly.

2. Attaching the arm cuff:

Take off the coat, sweater and other thick clothing, place the cuff directly against the skin, as clothing may cause a faint pulse, and result in a measurement error.

- Wrap the arm cuff around your left upper arm, about 2~3cm above the elbow, as shown.

- Palms up, the tube straightened and the middle finger in the same extension line, pressed nylon clasp, so that the cuff tied firmly.


Note: The patient is an intended operator. Constriction of the upper arm, caused by rolling up a shirt sleeve, may prevent accurate readings. Do not roll up the sleeves to measure, and make sure the cuffs are in good contact with the arms, similar to those wearing tights. Be careful not to rest your arm on the air tube. Left and right arm can be measured, left and right arm blood pressure may be different, so the recommended blood pressure values may also be different, it is recommended that you always use the same arm to get a correct reading.

If the measured values of the two arms vary widely, use it under the guidance of a doctor. Do not attempt to repair the device or components by yourself. Please consult customer service. Do not service or maintain the equipment while it is in use.

For the maintenance please refer to the "12. Maintenance and Storage"

How to sit correctly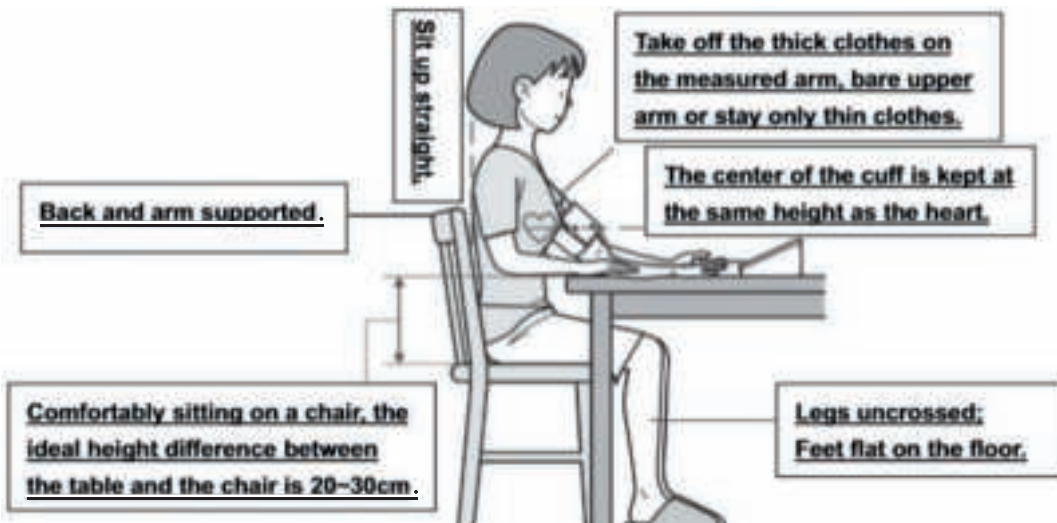

Taking a measurement

The unit can store 2 sets of measurement results (A and B), each group can store up to 192 groups. This manual explains the measurements of group A as an example and group B as a reference to the measurement of group A.

Note:

The monitor will not inflate above 280mmHg (37.3kPa).

Inflating to a higher pressure than necessary may result in bruising where the arm cuff is applied.


If the high pressure value is expected to exceed 200mmHg, press the **[▶MEMORY]** button, the inflation pressure value will be specified to pressurize to 260mmHg. Each time you press the **[▶MEMORY]** button once, the inflation pressure value increase to 20mmHg, please put the inflation pressure value.

Press the ON/OFF button

Start measuring

Press the ON/OFF button, the device starts and the cuff starts inflate. Pressurize to predetermined value, automatic uniform deflate, detect the blood pressure and pulse rate.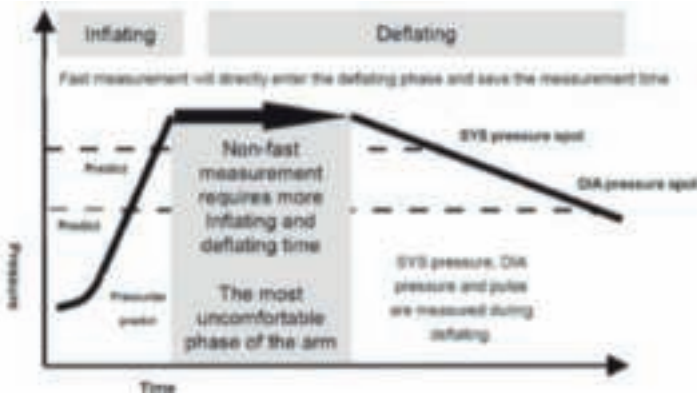

Fast measurement

This pressurize process of the blood pressure monitor could predict high and low pressure, shorten the measurement time, each user can quickly and easily measure.

Note: Before performing another blood pressure measurement, please rest for 3 minutes to allow the artery to return to the state between blood pressure measurements.

Remove the cuff

Press the ON/OFF button to turn off the unit

_ The unit automatically saves the measurement results.

_ The unit has an auto power off function; after about 3 minutes without any operation the device will automatically shut down.

Note:

* Press the ON/OFF button to turn off the main unit or pull out the cuff tube immediately when the pressure does not stop.

Measurement Results, memory and query

Measurement results:

1 The device could voice the measurement results.

As shown in Picture 1: High pressure: 128mmHg

Low pressure: 78mmHg

Pulse: 72PUL/min.

Measurement results are automatically saved. The serial number is 0, the previous result storage number is 1, followed by column push.

2 When a measurement error occurs (the blood pressure monitor shows the error E as shown in Picture 2), please wait for 3 minutes after resting. Follow the measurement procedure to re-measure.

Memory:

- The unit can store 2 sets of measurement results, each set can store up to 192 group results. When saving the 193 result, the device will delete the oldest measurement result.
- Before displaying the measurement results, the serial number will appear for about 1 second. The latest group of measurement result is "0".

Inquire:

- After measurement or in the state of shutdown, press the **[▶]**

MEM button (as shown on the right) to view the saved data.

- Press the **[▶]**MEM button or the **[▶]** SET button to cycle through the read memory data.

Showing an error message
LCD error prompt and error cause
correspondence table

Symbol	Cause	Solutions
	Low battery Cuff inflating is insufficient	Replace the new battery Please read and repeat the steps listed in the Measurement Section
	The tube plug (air connector) is disconnected	Plug the tube plug fitting
	During the measurement, arm/body movement and other interference	Keep the body quiet and re-measured
	Inflation pressure is not higher than the expected high pressure 30mmHg	Manual pressure to meet the needs of inflatable pressure
	Attaching the cuff incorrectly	Clothes oppression the arm
	Correctly attach the cuff	Take off clothes and then measure
	Clothes oppression the arm	Take off clothes and then measure
	Cuff leaks	Replace a new cuff Please call customerservice

Symbol	Phenomenon	Possible Causes
	Not pressurized	Attaching the cuff incorrectly The cuff is not connected with the unit Cuff leaks
	Measurement failed Pressure greater than 280mmHg	

Electromagnetic compatibility instructions

- The Silvergear blood pressure meter meets the requirement of electromagnetic compatibility in IEC60601-1-2.
- The Silvergear blood pressure meter is suitable for home healthcare environments.
- The user needs to install and use according to electromagnetism compatibility information which is attached with it. Guidance and manufacture's declaration stated in the appendix A.

Warning: Do not place near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this Silvergear blood pressure meter adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment may influence performance, it should be used no closer than 30 cm (12 inches) to any part of the Silvergear blood pressure meter. Otherwise, degradation of the performance of this equipment could result.

The explanation of graphic symbols

	Manufacturer		CE mark
	Authorized representative in the European Community		Date of manufacture

	This way up		Keep away from sunlight
	Type BF applied part		Keep dry
	Direct current		"WEEE (Waste Electrical and Electronic Equipment)". The waste products should be handled legally.
	Class II equipment		Refer to instruction manual
	Heartbeat symbol		Movement symbol
	Irregular heartbeat symbol		Handle with care
	User A		Low power symbol
	User B		

6. Maintenance and storage

To protect your device from any damage, please observe the following:

- 1 Do not place the blood pressure monitor and cuff in high temperature, moisture, water vapor, or direct sunlight. Do not tighten the cuff or air tube too much.
- 2 Do not disassemble or attempt to repair the device or components. Changes or modification not approved by the manufacturer will void the user warranty. Consult Customer Service.
- 3 Do not let the blood pressure monitor be strongly impacted or shaken (e.g. the blood pressure monitor falls on the ground).
- 4 Do not use a corrosive liquid to clean the blood pressure monitor.
- 5 Please use a soft dry cloth to clean the blood pressure monitor. If the unit is particularly dirty, you could use a soft cloth with water/neutral detergent, wring the cloth and wipe the unit. If necessary, gently wipe the surface of the unit with an absorbent cotton and disinfect with alcohol.
- 6 Do not use gasoline, thinner or similar solvents to clean the cuff.
- 7 Please use a soft damp cloth and soap to clean the cuff.
- 8 When cleaning the equipment, please be careful not to pour the liquid on the device to make sure that no liquid can enter the inside of the device.

Environment protection

After the product reaches the end of its life, please send it to the right place to protect the environment. (Note: on the waste, residue should be consistent with the corresponding national laws and regulations.)


Calibration and repair

- 1 The accuracy of this device has been rigorously tested and it is generally recommended that the blood pressure monitor is inspected and calibrated at least every two years to ensure that the function is normal and accurate. The inspection and calibration should be implemented by a laboratory, manufacturer or the maintenance center authorized by manufacturer.
- 2 Do not attempt to repair the device or components by yourself. Please consult Customer Service.

Storage

When not in use, please put it in the box

- 1 Unplug the tubing connector from the tracheal socket
- 2 Keep the tube gently in the cuff and be careful not to fold the tube
- 3 Place the unit and cuff in the box, and store them in a clean and safe location
- 4 Do not place the unit where it is:
 - Easy to splash water
 - High temperature, humidity, direct sunlight, dust or near corrosive gas
 - Easy to tilt, produce vibration and impact.

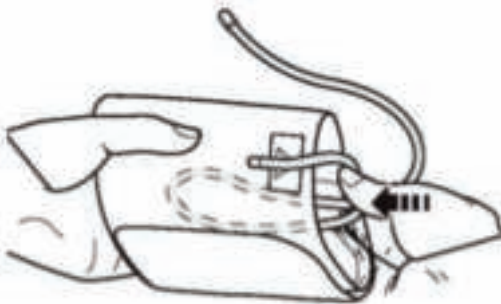The components are not provided: batteries & adapter

Using the adapter

- 1 Insert the adapter cord into the power interface of the monitor.
- 2 Plug the adapter into an electrical outlet.
- 3 To disconnect the adapter, unplug the adapter from the electrical outlet first, and then remove the adapter plug from the monitor unit.
- 4 Our company does not provide the adapter; the users are required to buy adapter (DC 6V/500mA) which meet safety requirements by themselves. Use of an adapter which does not meet the requirements may damage and/or may be hazardous to the device and users.

7. Troubleshooting

Anomalies	Cause	Solutions
The reading is too low (or too high)	The cuff is incorrectly attached	Correctly attach the cuff
	Moves or speaks during the measurement process	Keep quiet during the measurement The clothes oppress the arm
	The clothes oppress the arm	Take off the clothes that oppress the arm
Cuff pressure does not rise	The tube plug is not securely connected	Make sure the tube plug (air connector) is securely connected
	Cuff leaks	Replace the new cuff. Please call customer service
Cuffs deflating air too fast	Cuffs are attached too tightly	Please attach the cuff correctly
Cannot be measured, or the reading is too high or too low	Cuff inflating is insufficient	Please read and repeat the steps listed in the Measurement Section
Cannot boot	The battery has run out	Replace the new battery Install the battery according to the manual
	The battery is not installed correctly	
Other faults	Press the On/Off key to re-measure If the problem persists, try replacing the new battery. In addition to this, you are unable to measure, please call the customer service telephone for consultation.	

8. Warranty

1. The Silvergear blood pressure meter is warranted from manufacturing defects for 2 years from date of retail purchase. (Except for consumables, such as cuff, display screen, etc.)
2. The free repair service does not cover the damages resulting from personal reasons listed as below:

- Damage resulting from unauthorized adjustment and repair.
- Damage resulting from improper repair of unauthorized maintenance/repair shops.
- Damage resulting from inappropriate maintenance.
- Damage resulting from misuse by not following guidelines of this manual.

3. Maintenance services outside the warranty scope will be reasonably charged.
4. When apply for free repair service, please submit purchase proof of purchase date, and contact the distributor.

9. Disposal

The crossed out wheellie bin symbol on this product is to remind you that electrical products should not be disposed of with household waste. Please dispose of responsibly at an approved waste or recycling facility. Check with your local authority for recycling advice.

10. Appendix A. EMC Information

Guidance and manufacturer's declaration - electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not application

ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380 –390	TETRA 400	Pulse modulation 18 Hz	1,8	0.3	27
	450	430 –470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
	710	704 –	LTE Band 13, 17	Pulse modulation 217 Hz	0,2	0.3	9
	745	787					
	780						
	810	800 –	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
	870	960					
	930						
	1720	1 700	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
	1845	–					
	1970	1 990					
	2450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240	5 100	WLAN 802.11 a/n	Pulse modulation 217 Hz	0,2	0.3	9
	5500	–					
5785	5 800						

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