
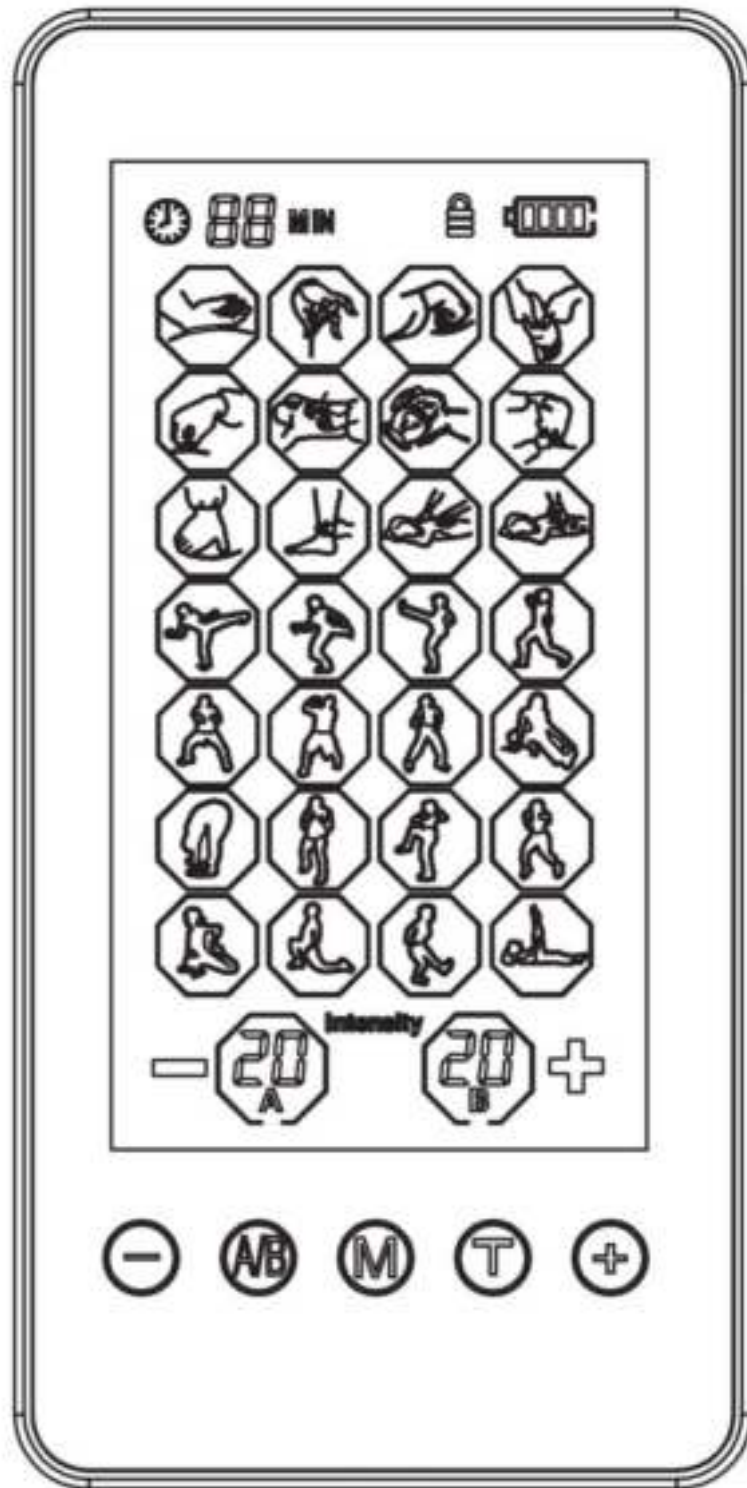
Kluvaro TENS Pain Relief Machine KRES200C

KRES200C

Nederlands

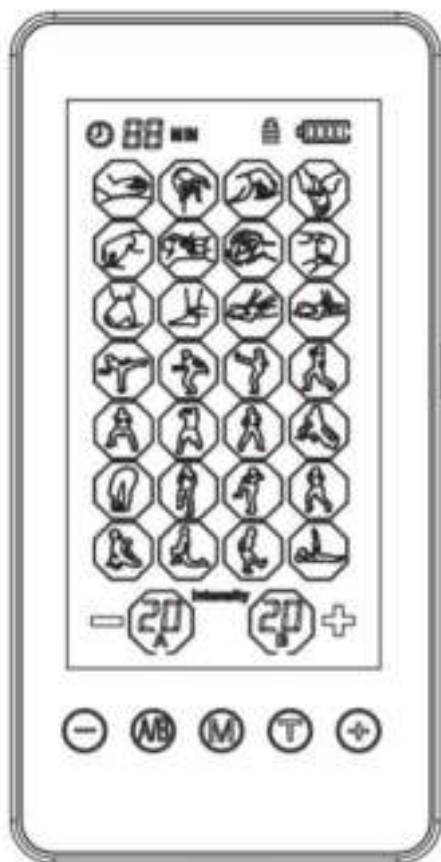KLUVARO[®]
CARE

TENS Pain Relief Machine

Introductie:

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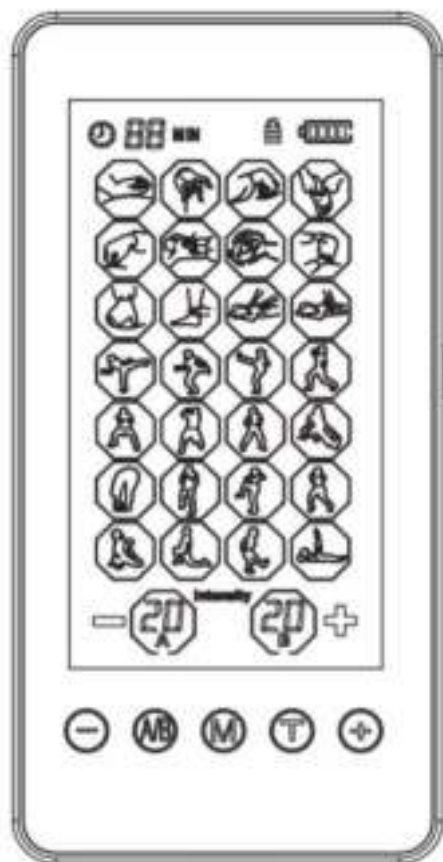
TENS Pain Relief Machine

Introduction:

This user manual will help you understand how to use the Kluvaro KRES200C machine to relieve your pain or relax your muscles. In this manual you will find all the necessary information and technical data to make responsible use of the stimulator and its accessories.

Please read this manual carefully before using the Kluvaro KRES200C and retain the product manual after reading. If you are receiving treatment from a doctor or therapist, always follow their instructions first! They are the right people to give you further explanations.

This portable TENS/EMS machine from Kluvaro is compact, stylish and equipped with safe and soft Kluvaro electrodes. It is an advanced device with many technical stimulation possibilities. Many people use the Kluvaro KRES200C for pain relief or muscle relaxation. We hope that you too will benefit from this device. We wish you good luck with your therapy.

What's pain?

Pain is a warning signal that indicates that our body is damaged or in danger of being damaged (for example, if you put your hand too close to a flame).

Pain can also have a repair function, preventing damaged parts, such as broken limbs, from being overburdened and making it harder to heal.

These types of pains are usually referred to as ACUTE pains.

Acute pains generally last a short time and disappear once the damage or infection has passed. Acute pains can be intense and unpleasant, but have a clear protective function.

Another type of pain, which can also be very unpleasant, is the long-lasting or CHRONIC pain. This pain no longer has a warning or recovery function and is caused by a disturbed functioning of the nerves. Often, there is no obvious cause.

Pain relief through nerve stimulation (TENS)

The Kluvaro KRES200C machine transmits electrical pulses through the skin to the nerves by means of electrodes placed on the body. This stimulation of the skin has a dual purpose. On the one hand, the nerves are 'distracted' from the pain stimuli present, so that fewer pain signals are transmitted to the brain. On the other hand, the body is encouraged to produce endorphins, which have an analgesic effect.

The degree of pain relief during stimulation, as well as the number of hours of pain relief after you stop stimulation, differs for everyone.

The application of TENS has no healing effect. It is a symptomatic treatment that suppresses the sensation of pain. Prolonged stimulation is not dangerous or harmful. However, long-term stimulation in the same place can cause skin irritation in the long run.

Kluvaro KRES200C stimulator

The KRES200C is a digital two-channel stimulator. At the bottom of the device there are two outputs, indicated with the letters 'A' and 'B'. The two channels work independently and do not influence each other. You can also use the device as a single channel, with the other channel switched off. The device has a clear LCD display with touch screen. With the supplied pen, the device is easy to operate.

The stimulator is powered by a rechargeable lithium battery and delivers current pulses via cables, electrodes and the skin to underlying nerves. Charging is done via the USB cable and adapter. To do this, use the outlet at the bottom of the device.

Furthermore the device is delivered with 16 electrode pads. A maximum of 8 electrodes can be connected at the same time via the 2 electrode wires. Each wire has 4 outputs.

The device requires little or no maintenance. It is best to clean the device regularly with a damp cloth. Never use aggressive cleaning agents, such as ether, alcohol or abrasives. Never immerse the device in a cleaning liquid or the like!

Intended use

TENS: The device is used for the symptomatic relief of chronic pain syndromes and the temporary relief of pain associated with tender and painful muscles in the arm, shoulder, neck, back, waist, abdomen and leg as a result of exertion or through normal household and work activities. It is also used for post-operative and post-traumatic acute pain problems.

EMS: The device is used for relaxation of muscle spasms, increasing blood flow in the circulatory system, prevention or slowing down of atrophy, muscle re-education, maintaining or increasing range of motion, stimulation of calf muscles immediately after surgery to prevent venous thrombosis.

General comments:

1. Read and follow the instructions carefully before use.
2. If you feel uncomfortable using this product, stop using it immediately.
3. Turn off the device when removing or moving the electrode plate of the portable stimulator.
4. Before use, check that there is no contact between metal and the electrode pad of the portable TENS & EMS machine.
5. Do not use this device for the same body part more than twice a day.
6. Do not repair or disassemble the product yourself.
7. Recycle the unit in accordance with the marking on the packaging. Do not simply dispose of the device.

Prohibitions:

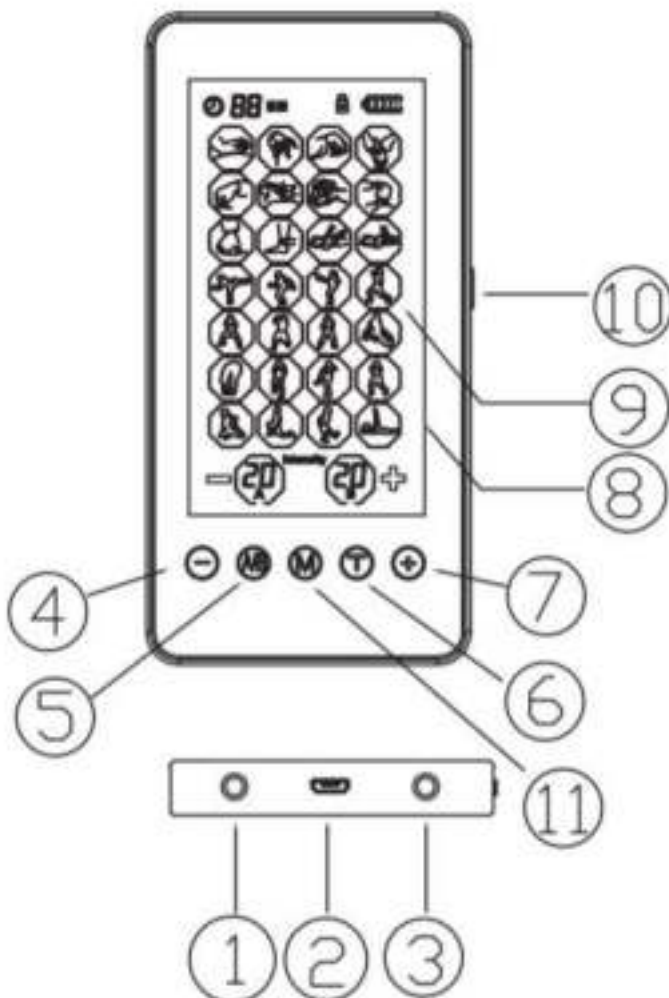
1. If any of the following conditions occur: DO NOT use the TENS Pain Relief Machine:
2. Please do not apply to the areas near the heart or above the neck and to wounds from acute or chronic illnesses.
3. Please do not use in case of cardiac arrhythmia or other heart problems, skin diseases, fever, epilepsy or other diseases. In case of doubt, always contact a doctor first!
4. This product should not be used by people who use a heart rate controller, artificial heart, pacemaker or other electronic medical devices.
5. Please do not use during pregnancy, less than 37 weeks.
6. Do not stick this product in the same place for a long time without removing it.
7. Please do not use while sleeping, driving or taking a bath.
8. Do not place the product in an environment that is too dry or humid.
9. Please do not use in inflammable, explosive and unstable areas.
10. The device should only be used with kluvaro accessories.

General user instructions:

1. The TENS machine should be charged before use. For a longer service life, we recommend charging the TENS machine to 100% before use.
2. Connect the electrode wires to the TENS machine. Use the outputs 'A' and 'B' at the bottom of the TENS machine.
3. Connect the desired number of electrode pads to the cables by snapping them into place. Use a minimum of 2 and a maximum of 8 electrode pads at a time. It is important to do this correctly to prevent the impulses from not being transmitted correctly. Also read the instructions under 'Pairing the electrode pads'.
4. Remove the protective film from the electrode pads and stick them where you want them. Make sure the electrodes do not overlap.
5. Turn the device on by pressing and holding the 'power button' on the side of the device for 4 seconds. Set the desired mode, intensity and time. The intensity should be set at a level that causes the muscles to contract and move slightly. The intensity of the pulses should remain at a comfortable level and should not be painful. The setting of the TENS machine is explained with the help of the schematic picture.
6. It is recommended to use the TENS machine 1/3 times a day, for a minimum of 10-20 and a maximum of 40-60 minutes per session.

Composition and operation of the KRES200C

- 1. Output channel A**
- 2. USB output**
- 3. Output channel B**
- 4. Intensity decrease button**
Press this button successively 1-20 times to decrease the desired intensity per channel.
- 5. Uitput switch**
Press this button to adjust the settings per channel.
- 6. Time setting button**
Press this button successively to set the desired duration for each channel.
(minimum 10 minutes, maximum 60 minutes).
- 7. Intensity increase button**
Press this button successively 1-20 times to increase the desired intensity per channel.
- 8. LCD- display with touchscreen**
- 9. Mode indication lights**
Switch easily between the different programmes.
- 10. ON/OFF button**
Press and hold the power button for 4 seconds to turn the device on or off.
- 11. Mode switch**
Press this button to switch between different programmes.


NB: Use the buttons on the device or the touch screen pen to set the desired programmes.

Mode:	Name:	Description:
1	Tuina	Traditional Chinese Tuina massage is a method using pushing, tapping and pinching with fingers, palms, fists and elbows. It's effective against various muscle and joint pains around the neck and shoulders and/or stress-related complaints.
2	Acupuncture	The acupuncture massage simulates with electric pulses a real acupuncture treatment with needles. By finding the right meridians, the energy flow in certain body parts will be restored.
3	Crosscut fist kneading massage	This mode simulates a hard knocking massage, especially for kneading muscles. This gives a revitalised feeling.
4	Footplate massage	By massaging certain foot points, blood circulation in the entire body is stimulated from the feet.
5	Vertical fist kneading massage	This mode simulates a firm massage with the fists. Suitable to relieve pain and stiffness of muscles.
6	Acupuncture kneading massage	A combination of acupuncture and kneading. A scratching and kneading simulation relieves various pains in muscles and joints.
7	Cupping	A constant stream of impulses simulates 'cupping'. Suitable for back and neck pain, stiff muscles, tension, anxiety, fatigue, migraines and rheumatism. This mode is also used to combat cellulite.
8	Ear press kneading	By stimulating earpoints, nerves relax and the entire blood circulation is stimulated and improved from there.
9	Lateral palm kneading massage	This mode simulates a massage with lateral palms. It is suitable for soothing pain in small muscles.
10	Reflexology	This mode is suitable for neurological disorders such as neuropathy and numbness in arms, legs and feet. The massage causes a physical reflex in legs and arms.
11	Lifting pinching manipulation	This mode simulates the stretching and pinching of the skin. This helps to improve skin and muscle elasticity and promotes blood circulation.
12	Shiatsu	Shiatsu is an alternative massage technique in which firm pressure is applied with the fingers and palms of the hands. It is a method that can be used very widely, but is effective for migraines, back pain and digestive problems, among others.
13	Foot reflexology	This mode is suitable for neurological disorders, specifically aimed at the feet and lower legs. The massage causes a physical reflex and can be used to reduce stiffness and neuropathy in the legs.
14	Knocking	This mode generates beating impulses (Electrical Muscle Stimulation). It helps to relax muscles and stimulates fat burning. Similar to aerobic exercise.
15	Trigger point	This mode simulates a soft massage with fingertips and small pressure techniques. Suitable as a local acupuncture massage. It helps to relieve pain through acupuncture points.
16	Strike	The continuous pulse streams in this mode help muscles relax after exertion. This massage is used after an intensive workout or intensive training.
17	Stepping	This mode simulates a gentle massage after exertion. Suitable to soothe and relax muscles and joints.
18	Stretchings	This mode simulates stretching movements in the muscles. Prevent injuries and muscle complaints by using this mode before and after intensive training.
19	Swedish thai	Swedish thai is suitable for loosening chronically stuck muscles, but is also used after a workout to help muscles recover more quickly.
20	Rubbing	This mode exerts additional pressure on the veins. It helps open veins and improve blood circulation.
21	Kneading	This mode simulates a standard kneading massage. Suitable for helping muscles relax and recover.
22	Nudge	This mode simulates a firm massage with the elbows. It helps to relax and opens veins.
23	Weight loss	The pulses in this mode cause muscles to vibrate regularly. This simulates aerobic exercise and helps lose weight. Place the electrodes on the abdomen and put the body into fat-burning mode.
24	Tai Chi	Tai Chi provides stretching impulses and relaxes the body.
25	Shiatsu-deep	This massage is similar to the Swedish massage, but penetrates deeper and therefore helps better with chronic complaints. It is effective for sports injuries, whiplash, arthritis, muscle spasms, fibromyalgia and joint complaints.
26	Acupuncture	The acupuncture massage simulates with electric pulses a real acupuncture treatment with needles. By finding the right meridians, the energy flow in certain body parts will be restored.
27	Pressure Body	This mode simulates a firm massage with the fists. It is ideal for relieving pain and stiffness in deeper muscle layers.
28	Sports	Helps to strengthen muscles and prevent muscle pain. This mode can be used in many different situations.

Connecting the Electrode Pads

It is important to connect the electrode wire correctly to the electrode pads to prevent the impulse from not being delivered or not being delivered completely. A plus pole (+) and a minus pole (-) should always be used at the same time. These can be recognised by a blue and a white end. With the KRES200C you can connect 2, 4, 6 or 8 electrodes at the same time and divide these over two electrode wires.

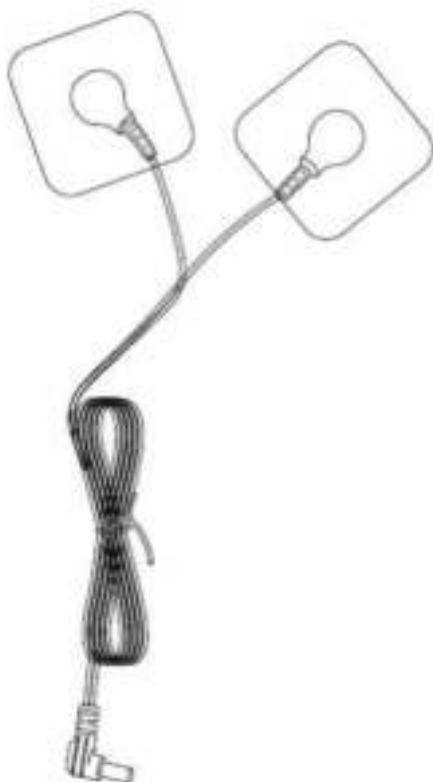The KRES200C is supplied with two electrode wires. Each wire is divided into two equal strands. Each strand contains two loose ends to attach electrodes (a plus and a minus pole). Per strand, plus and minus must always be connected at the same time. If this is not the case, the pulses will not be delivered.

Functioning of the electrodes

1. Attach the electrode cable to the electrode pad (as shown in the illustration on the left).
2. Before use, clean the skin and gently tear off the protective film before use (re-tape the film after use to protect the life of the electrodes).
3. Stick on the correct body parts according to the instructions.
4. Be sure to re-tape the protective film to the original electrode after use to prevent dirt buildup.

Note: Because the conductive adhesive has a strong ability to stick to material, there will be a strong adhesion force.

Do not pull forcefully when tearing the storage foil, to avoid the conductive adhesive becoming detached from the electrode.

Placing the electrodes

The electrodes can be placed in two different ways:

Twin mode: the electrodes are placed close together on the same side of the body and in the same direction.

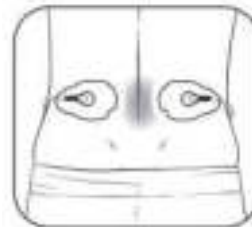
Opposed mode: the electrodes are placed on both sides of the body, but each on a different side. For example, one at the top and one at the bottom of the elbow.

On the next page some possible placements are shown schematically.





Twin mode

Opposed mode

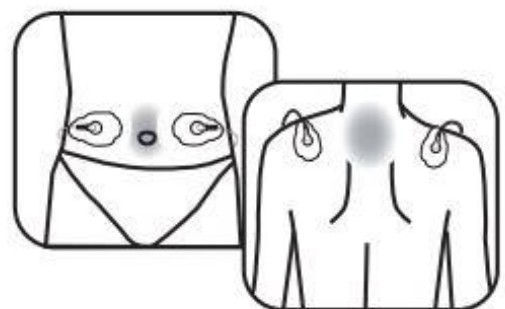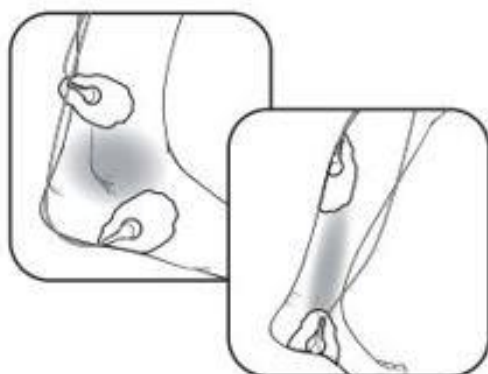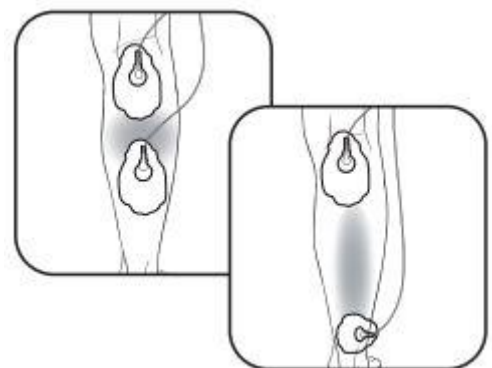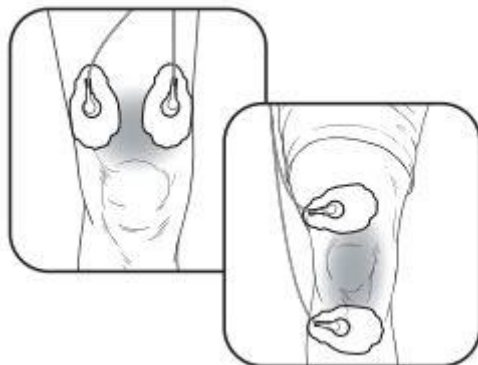Some application areas represented schematically

Kluvaro electrode pads

Kluvaro electrodes are of high quality and suitable for various TENS/EMS devices, muscle stimulators and other electrode therapy devices. The special gel on the pads provides extra adhesive strength and also improves the conductivity of the current impulses. These electrodes contain no latex, which reduces the risk of allergies.

For best results and longer life of the electrodes, it is important to clean the skin well with soap and water or alcohol before use. Places with a lot of hair should be shaved first. Carefully remove the electrodes from the foil and place them in the right place on the body. After use, slightly moisten the pads with clean water before sticking them back on the foil. The pads have a longer life span if they are stored in a cool place (refrigerator).

The electrodes wear out through use. This is visible, as the electrodes become dull, and noticeable, as you may experience a burning sensation. Check the pads regularly and replace them in time to avoid unnecessary skin irritation.

Make sure that the electrodes do not touch each other during use, otherwise a leakage current will occur between the electrodes, reducing the effect of the stimulation.

The TENS gel ensures an even distribution of the current under the electrode. The use of the gel prevents the current pulses from being concentrated in too small an area, thus reducing the risk of skin irritation.

The plasters ensure that contact between the skin and the gel electrodes remains optimal. In some cases, skin irritation can occur. To prevent this, press the plaster from the centre to the sides without stretching the skin.

Warnings when using Kluvaro electrodes

- 1.** Only use Kluvaro electrodes with this device. When using other brands of electrodes, we cannot guarantee proper functioning.
- 2.** Do not use the same electrodes for different persons. Each user should use his own set of electrodes.
- 3.** Always turn off the device before removing or moving the electrodes.
- 4.** Clean the skin thoroughly and remove excess hair before use. The skin must be dry and free of lotions or creams.
- 5.** Electrodes that no longer adhere properly should be replaced.
- 6.** Stop stimulation if skin irritation persists due to electrode use.
- 7.** After use, stick the electrodes back on the foil or adhesive plate and store in a cool and dry place.

Product specifications:

Productmodel: KRES200C
Material: ABS
Channels: Two channels
Battery: Rechargeable lythium battery
Wave form: bi-directional wave
Sound button: No
Display: LCD with touchscreen
Modes: 28 modes

Output frequency: 1~120HZ
Output pulse width : 100~250uS
Output voltage: 40V ±20%
Intensity: 0 ~ 20 niveaus
Weight: 360 (g)

Working environment:

Temperature: 5°C to 40°C (41°F to 104°F)
Humidity: 15%RH to 93%RH

Storage Environment Temperature:

-25°C to 70°C (-13°F tot 158°F)
Humidity: 15%RH tot 93%RH

Service and warranty

If you need service or information about this product, please do not hesitate to send an e-mail to info@kluvaro.nl.

We offer a two-year warranty on our products. The customer must be able to show proof of purchase and the warranty must be claimed within two years of the purchase date.

Please note! The warranty does not apply:

- For damage caused by improper use or repair;
- For parts subject to wear and tear;
- For defects of which the customer was aware at the time of purchase;
- For defects caused by the customer's own fault;
- For damage caused by third parties.

Disclaimer

Changes subject to change; specifications subject to change without notice.

Flux Forward
Van Neslaan 67
3742MP Baarn
Netherlands
03/2021 – V1.

©Kluvaro
www.kluvaro.nl