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Thanks for choosing the **OEMMIE** electric breast pump.

IMPORTANT:

- Read the instructions in this manual carefully before using the device. Failure to comply with the instructions given in the manual or using this device in ways other than the ones mentioned in this manual may result in injury to yourself or damage to the device.
- Make sure to include this manual when passing this device on to third parties.
- The images in the user manual are for illustrative purposes only and may differ slightly from the device.

SAFETY INSTRUCTIONS

Medical Considerations:

- It is recommended that you consult a medical professional before using the breast pump if you have an active implanted device that could potentially be affected by a magnetic field, such as a pacemaker or an implantable cardioverter defibrillator.
- Discontinue use if skin irritation occurs due to direct contact with the product.
- This product is intended for individual use. Use by multiple people may pose a health risk.
- Pumping can induce labor. Wait until after delivery before pumping. If you become pregnant while breastfeeding or pumping, consult a qualified physician before continuing to use the breast pump.

General safety instructions:

- Do not use the breast pump for any other purpose other than its intended use.
- Handle the device carefully. Jolts, impacts, or a fall even from a low height can damage the device.
- All parts must be correctly assembled to ensure the safety of the device.
- The breast pump is not a toy. Keep and store it out of the reach of small children.
- Keep the device away from heat sources and away from areas where the ambient temperature is relatively high (e.g. stoves or other heat sources). Never use the device near naked flames (such as burning candles) or on hot surfaces.

Electronics:

- You may not reconstruct, self-repair, or disassemble the breast pump. This can damage the device.
- Only use a power adapter with 5V / 1A output (not included in the package) to power or charge the breast pump. Another adapter with the wrong specifications may cause malfunction and/or damage.
- Check the charging cable periodically for damage. Never use the cable if there are any signs of damage.
- Never unplug the charging cable from the power adapter or the breast pump by pulling the cable. Always pull on the connector.
- The breast pump may become warm during operation or while charging. This is normal.
- Do not cover the device while it is being charged as this may cause the breast pump to overheat. This poses a risk of fire, electric shock, or injury.


Water:

- This device is not waterproof.
- Do not use the breast pump in or near a bath, shower, or swimming pool.
- Do not use in wet environments and do not submerge in liquid. Never pour liquids over or near the breast pump. There is a risk of electric shock/short circuit. If liquid does get into the housing unplug immediately and then turn off the breast pump.
- Do not reach for the device if it has fallen into the water while charging the battery. Immediately unplug the power adapter from the power socket and only then remove the device from the water.

Battery:

- Do not charge the battery at temperatures below 5°C or above 40°C.
- Do not continue charging the battery when the battery is fully charged.
- Do not charge the battery near flammable materials (fire hazard).
- Do not attempt to charge the battery if the battery emits any substances.
- Do not attempt to pierce, modify, disassemble, or change the battery.
- Never throw the battery into a fire.

PRODUCT PARTS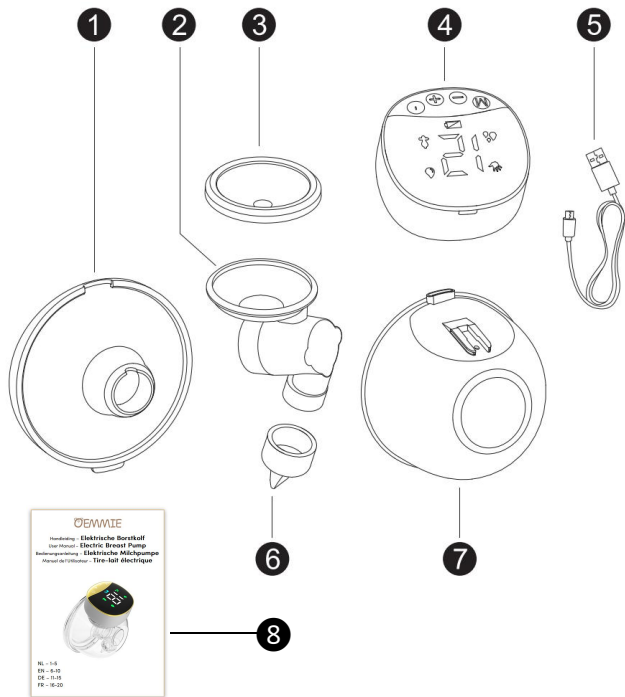

1 Silicone shield x1

3 Silicone diaphragm x1

5 USB charging cable x1

7 Milk collector x1


2 Tee x1

4 Breast pump x1

6 Duckbill valve x1

8 User Manual x1

CONTROL PANEL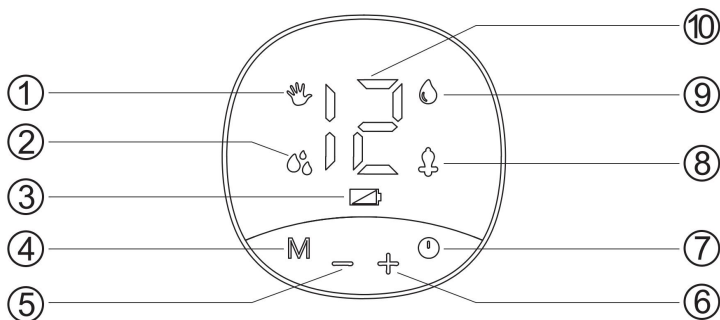

- | | |
|-----------------------|-------------------------------------|
| 1 Lactation mode | 2 Pumping mode with different speed |
| 3 Battery capacity | 4 Mode button |
| 5 Level down button | 6 Level up button |
| 7 On/Off/Pause button | 8 Pumping mode |
| 9 Sucking mode | 10 Suction level |

Note:

This product has a built-in lithium battery. When the battery indicator is flashing, we recommend that you charge the breast pump ASAP with a 5V DC 1A power adapter.

CLEANING AND DISINFECTION

1. Before using the breast pump for the first time and before each use, you must remove and clean all parts that make contact with the breast milk.
2. Drinking water is recommended for cleaning.
3. To avoid bacteria, remove and clean all the parts every time after use.

CLEANING WARNINGS:

1. Do not let the breast pump and charging cable make contact with water. You can clean them with a soft dry tissue or cloth.
2. Be careful when cleaning the duckbill valve and the silicone diaphragm. The breast pump will not work properly if one of these parts is broken.
3. Clean the duckbill valve carefully. Do not rub it and do not insert anything, otherwise, it may cause damage.

Using a power adapter for the power supply:

Insert the power adapter into the power socket and make sure that it is in place. Hold the On/Off/Pause button for 3 seconds to turn on the power, and select the mode and suction according to your comfort level.

Power supply from the Lithium battery:

This product has a built-in lithium battery. When in use, directly press and hold the On/Off/Pause button for 3 seconds to turn on the power. When the battery is charged, choose the mode and suction power according to your comfort level. The battery symbol flashes when the battery is low. Charge the battery in time.


Charging the battery:

Insert the power adapter into the power socket and make sure that it is in place. Charging is not possible when the breast pump is on. During the charging process, the red light of the power symbol is on and the battery symbol flashes blue. If the blue light remains solid, it means the battery is fully charged. In this case, unplug the breast pump from the power adapter.


ASSEMBLY

To assemble the electric breast pump, follow the steps below:


- ① Put the silicone diaphragm into the tee. Use your hand to press so that the edges are firmly fastened and that it keeps it completely sealed.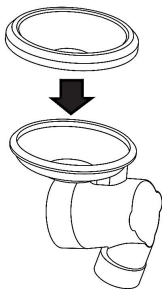


- ② Install the duckbill valve into the bottom of the tee. Make sure that it is mounted in the correct direction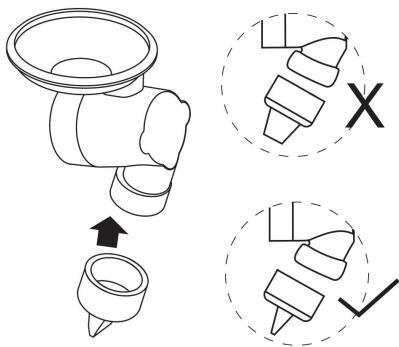


- ③ Put the assembled tee part into the milk collector, and align the "ear" part of the tee with the milk collector. Press the disc part of the tee with both hands to fasten the edge firmly to ensure safety and sealing.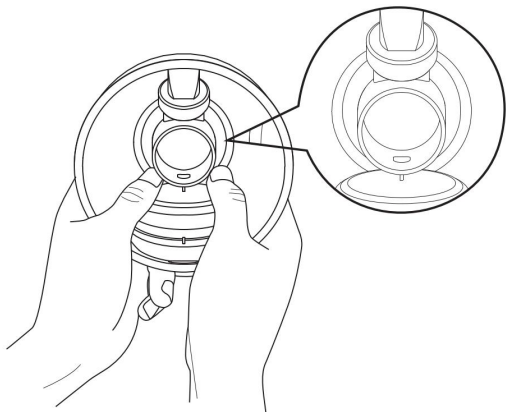


- ④ Install the silicone shield on the milk collector. Align the guide groove on the silicone shield with the guide groove on the milk collector, and press the edge with both hands to ensure that it fits in place.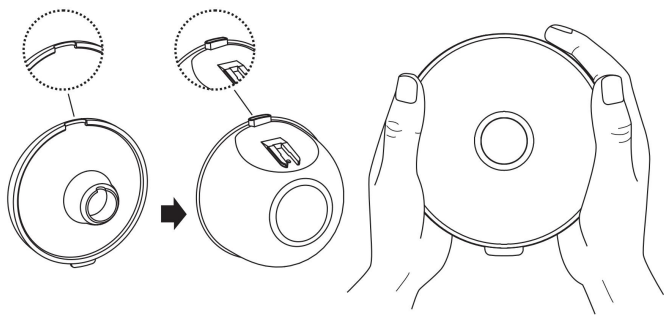

- ⑤ Push the breast pump into the slot of the milk collector, making sure that it is fully seated on the milk collector.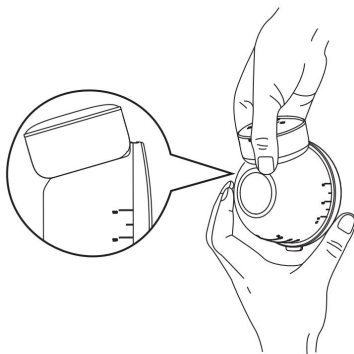

- ⑥ Connect the charging cable to charge the battery.

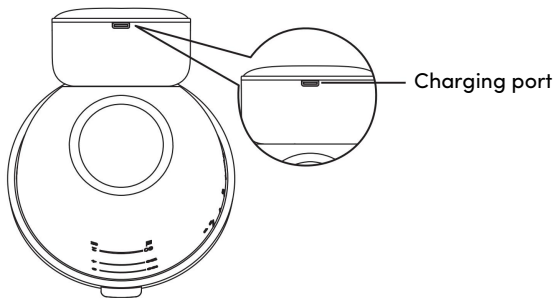
REMINDER:


The power adapter is a common standard accessory available on the market. The adapter is not in the scope of delivery but you can buy it separately if you need it. Power adapter specification: 5V DC 1A

USING THE ELECTRIC BREAST PUMP


- ① Fit the silicone shield of the breast pump tightly to your breast and leave no gap. Secure your nipple center and keep it upright. Be careful not to distort the position of the product and the breast, otherwise there will be no suction.

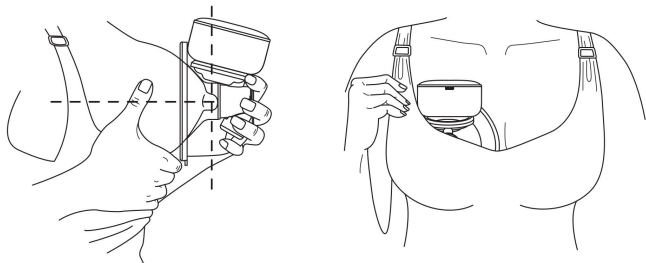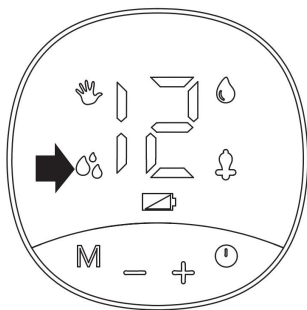
- ② Press and hold the power button ① for 3 seconds to start the breast pump. In the power-on state, long press to turn off, a short press to pause, and then a short press to continue working. During the pause, the indicator light flashes on the display. If there is a pause of 10 minutes and without operation, an automatic shutdown takes place.

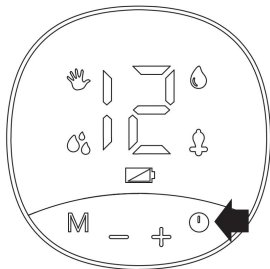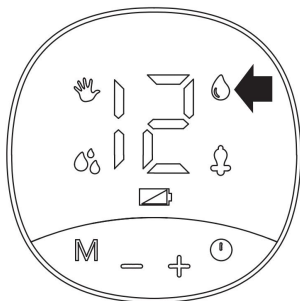
- ③ You can short press the **M** button to select the lactation mode . There are 12 levels in this mode. Levels 1-6 are automatic levels. If there is no other operation, the level will be automatically upshifted every 45 seconds. Except for pause, if there are other operations, the automatic upshifting will fail. Level 7~12 needs to be manually adjusted.

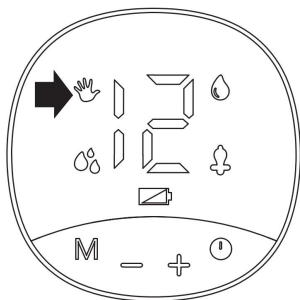
- ④ Short press the **M** button to select the pumping mode with different speed . In this mode, there are five light pumping rhythm.

- ⑤ Short press the M button to select the suction mode . In this mode, there are two short and one long, three short and two long pumping rhythms.


- ⑥ You can short press the M button to select the breast pumping mode . This mode is the normal pumping rhythm.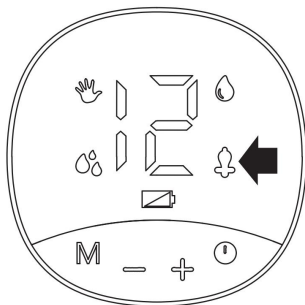


REMARK:

The "+" and "-" buttons are used to adjust the lactation, the pumping speed, the sucking and pump mode levels (1-12).

- ⑦ After pumping, press the On/Off/Pause button ① to turn off the breast pump and then carefully remove the breast pump from the breast.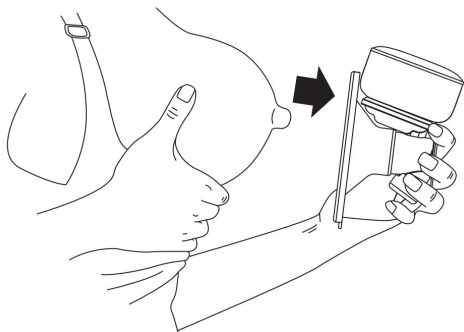

- ⑧ Remove the breast pump from the milk collector by sliding it out of the slot. Pour the milk into a storage bottle, making sure the spout is aligned with the bottle.

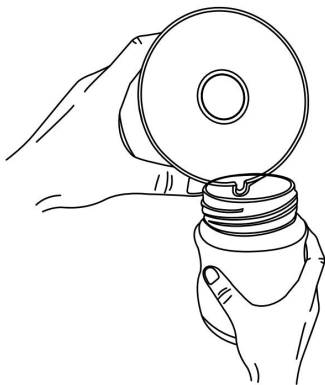TECHNICAL DATA

Charging voltage	5V DC 1A
Recharging current	650mA
Battery Type	Polymer battery
Battery capacity	1100mAh
Rated voltage	5V
Rated current	0.6A max
Usage time	>1.5h
Pumping Strength	40mmHg ~ 300mmHg
Noise	≤ 60dB
Working environment	+10 ~ +30°C RH 15% ~ 93% 700 ~ 1060hPa
Storage environment	-25 ~ +50°C RH 15% ~ 93% 700 ~ 1061hPa

STORING BREAST MILK

Storing your breast milk:

Expressed milk can be placed in a clean and sterilized storage bag or a bottle with a seal and immediately refrigerated or frozen.

NOTE:

Only store breast milk collected from a sterilized breast pump.

CAUTION:

Expressed milk can be refrigerated for up to 48 hours (inside the refrigerator) or in the freezer for up to three months, with the bottle or storage bag clearly labeled with the date and time of expression. The first expressed milk should be consumed first.

Thawing of breast milk:

Thaw frozen breast milk overnight in the refrigerator. In an emergency, you can freeze breast milk with a bowl of hot water. After thawing breast milk, it should be consumed within 24 hours.

Heating of breast milk:

For breast milk heating, heat the bottle upright in a bowl of hot water, or add a bottle with milk in a bottle warmer to heat the breast milk safely and quickly.

WARNINGS:

1. Always check the temperature of your breast milk before feeding your baby.
2. Do not microwave breast milk as this may destroy important nutrients and antibodies. In addition, uneven heating may cause the liquid to overheat locally, which may cause burns.
3. Do not heat the milk storage bottle by immersing it in boiling water to prevent it from breaking or cracking.

General cautions:

1. Do not freeze breast milk repeatedly.
2. Do not mix fresh breast milk with frozen breast milk.
3. Be sure to discard the remaining breast milk after feeding.

DISPOSAL

- The packaging material is recyclable. Dispose of it in an environmentally friendly manner and take it to a recycling center.

- Dispose of the device in an environmentally friendly manner. Dispose of it at a recycling center for used electrical and electronic devices. You can obtain more information from your local authorities.

WARRANTY

We grant a 1-year warranty on our products. The customer must be able to show proof of purchase and the warranty must be made within 1 year from the date of purchase. Please note that the warranty does not apply:

1. Damage caused by improper use or repair.
2. Parts subject to wear.
3. For defects of which the customer was aware at the time of purchase.
4. In the event of defects due to the customer's fault.
5. In case of damage by third parties.

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© 2024 OEMMIE
Dennepark B.V.