my  
nails

# Miconail® Patch *forte*

## IT **T** Trattamento Anti-Micosi in cerotti

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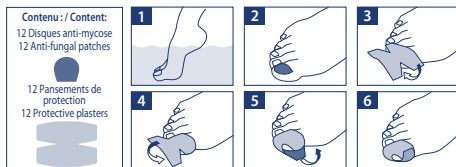
## EN Patch Treatment for Fungal Nails

**USE:** 1. Soak the fungal nail in warm water for about 10 minutes. Dry thoroughly. 2. Peel off a patch from the backing sheet and apply to the nail. If the patch is larger than the fungal nail, reduce it to the right size to cover only the affected surface. The patch can be cut easily. 3. If necessary, use the protective plaster to fix and protect the patch. For small nails, cut the patch in half. 4. After fixing the first two wings, fold the other two wings under the big toe. 5. Wrap the patch around the nail by folding its wings above the big toe. 6. Replace the patch and plaster every day for 1 to 3 weeks of treatment and in any case until the fungal infection has completely healed.

**WARNINGS:** Do not use for longer than recommended. Long treatments may sensitise the skin; in this case, treatment should be discontinued. Do not apply to mucous membranes or to the skin. Do not apply to lesions. Do not use if you are allergic to any of the components. Keep out of reach of children. For external use. Do not reuse. Store in a cool, dry place.

**REF**  
208V24DV

Dispositivo Medico  
Producto Sanitario  
Dispositif Médical  
Medical Device