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**(UK)** Sea-Band's Mama! Sea-Band operates by exerting pressure on the acupressure point (called the Nei-Kuan Point) on each wrist. It takes only a few minutes to become effective. The Sea-Band provides drug free and effective relief from the symptoms of morning sickness without causing drowsiness or other side effects.

It has also been found to be effective in other conditions that produce nausea, such as travel sickness, anaesthesia and chemotherapy. The Sea-Band may be used repeatedly or worn continuously and can be hand washed using soap or mild detergent. Remove the bands if any swelling occurs.

Sea-Bands for travel sickness are also available in a range of colours for children and can be found in all good pharmacies.

#### HOW TO POSITION THE SEA-BANDS CORRECTLY

- Place your middle three fingers on the inside of your wrist with the edge of third finger just below the first wrist crease. The Nei-Kuan point is located underneath your first (index) finger in between the two wrist tendons.
  - Position the button facing downward over the Nei-Kuan point. Repeat the process for the other wrist.
- One band must be worn on each wrist to be effective. Sea-Bands can be put onto your wrists before morning sickness sets in, or once the symptoms have started.

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SEA-BAND.  
MAMA!

Sea-Band Limited  
8 Hawley Road, Hinckley  
Leicestershire, LE10 0PR England  
Tel: +44 (0) 1455 639750.  
Email [hq@sea-band.com](mailto:hq@sea-band.com)  
[www.sea-bandmama.com](http://www.sea-bandmama.com)  
[www.facebook.com/seabandnausea](http://www.facebook.com/seabandnausea)  
© Sea-Band is a Registered Trademark.  
Manufactured in England. 6720 Rev 2

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