

**GB** Important information for PASANTE Natural Rubber Latex Condoms, please read carefully. Always handle the condom carefully including removal from the package so as to avoid damage to the condom by finger nails, jewellery, etc. Condoms made from natural rubber latex may cause allergic reactions including anaphylactic shock. A condom should never be used for a second time, condoms are for single use and if reused there might be an increased risk of failure or infection, the risk of breakage is also higher. The use of a condom cannot guarantee 100% protection against pregnancy, HIV, or Sexually Transmitted Infections (STIs). If in doubt always consult a doctor. If additional lubricant is desired, use the type of lubricant recommended for use with condoms e.g. for Natural Rubber Latex Condoms a water-based lubricant such as Pasante TLC or Light Lube and the need to avoid the use of oil based lubricants, such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as these are deleterious to the integrity of the latex condom. Condoms used for anal sex may have an increased risk of slipping off or being damaged. It is recommended to use additional lubrication if vaginal lubrication is insufficient or for anal sex to minimize the risk of condom breaking or coming off. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom. Carrying condoms in your wallet, back pocket, handbag, or similar is not advisable. If you have unprotected sex or your method of contraception has failed, contact your Sexual Health Clinic or GP for emergency contraception advice preferably no later than 72 hours.