

INGRDIENTS: Silicone gel, polydimethylsiloxane

HOW TO USE:

- 1- Clean and dry scar area.
- 2-Cut scar sheet based on scar size and shape.
- 3- Remove liner from the adhesive side of the scar sheet.
- 4-Press scar sheet gently onto the scar area. Apply scar sheets on post-surgical scar 1-2 weeks after stitches are removed and when there is no oozing or crusting. you can apply immediatly on old scar. Cozy World scar sheets can be used for up to 2 weeks.

TIPS:

Use this product once scar or wound has been healing for the least days. You may start using immidiately on older scars. Silicone sheets should be worn for 4 -8 hours per day for the first few days, and gradually increased upto 24 hours a day. Use a minimum of 12 hours a day for maximum result. if the sheets get stained. gently wash them in warm water and let dry or use a hair dryer to dry it.

Start using the silicone working your way up according to table below

1day	3days	5days	7days	9days
4 hours	8 hours	12 hours	14 hours	16 hours
9days	10days	11days	12days	12days and later
18 hours	20 hours	22 hours	24 hours	24 hours

STORAGE: Store in cool dry place away from sun light.

Cozy World

Company Empiry

The Hague

The Netherlands

