

1. Principle of treatment:

1. Laser Rhinitis therapeutic apparatus is based on modern medicine and clinical practice research and development of semiconductor laser therapeutic instrument, the instrument adopts the semiconductor Light produce wavelength of 660 nm, 8-12 mw low visible laser intensity, irradiation treatment to the nasal cavity, activate blood physiology

2. Applicable to the crowd:

1. Medicaments Pharmacological rhinitis, allergic rhinitis, acute or chronic
2. Drying rhinitis have rhinitis, sinusitis, etc all kinds of rhinitis patients caused by nasal congestion, sneezing, dizziness, headache, blurred vision, listening comprehension, memory decline, depression symptoms
3. Long-term medication, still cured, and drug tolerance of patients
4. Have used a variety of treatments, still break out repeatedly, has lost confidence in treatment of patients



3. Instructions for use:

1. Remove the battery cover;
2. Put 1* 9V Alkaline battery into the battery box;
3. Connect the wire and the device.
4. Turn on the power button, (Note:Don't take off the small pin cover). Put the nose clip into nose, make sure 2 light point into the nose about 1 cm to 1.5 cm depth;
5. The Green light keep lighting shows working normal, if it is flashing, shows operation failure(Low battery, or the wire didn't put in a correct way)
6. The Green light will auto-turn off after one treatment session is over (15 minutes).
7. Use it 1 session a day.

4. How to clean:

1. Use a soft cloth to clean the light point, please don't immersed in water.
2. Disinfection: The light point can be disinfected by 70 percentage of alcohol.
3. Protect: Please don't use the same device for different person.

5. Technical category:

- * 660nm Narrowband light waves
- * Treatment session timer: 15 minute ($\pm 10\%$)
- * Electricity: 4-8mW for each pin.
- * Energy for each time: 1-2.5J



Note:Please do not take off the protective cover when using it.