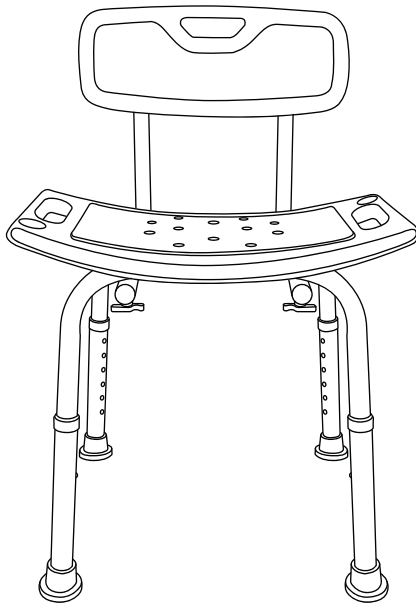
Bellottini

DOUCHESTOEL | SHOWER CHAIR | DUSCHSTUHL DOCCIA SEDIA

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Productnaam: Bellettini – Doccia Sedia - Douchestoel

Item no: B30520

PLEASE READ THE USER MANUAL CAREFULLY AND KEEP FOR REFERENCE. PREVENT ACCIDENTS AND FOLLOW THE USER MANUAL CAREFULLY.

WARNING!

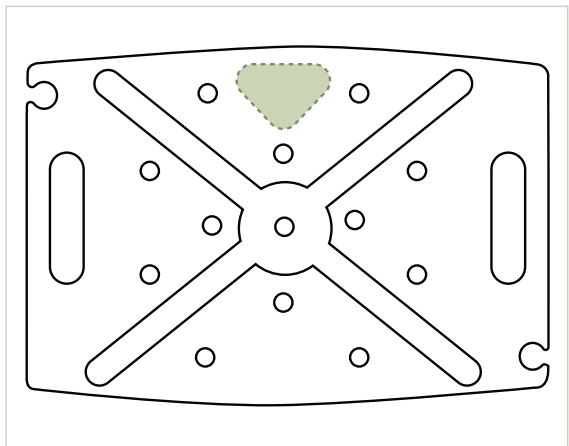
- » To be assembled by an adult! Keep this information for future reference.
- » Remove all packaging materials before giving the toy to your child.
- » This product contains small parts which are necessary for assembly. Prior to and during assembly of this product, keep small parts out of children's reach. Small parts pose a choking hazard to children under 3 years.
- » We recommend regularly checking that the mounted shower chair is still securely fastened. Should this not be the case, do not use the shower chair again. This, to avoid personal injury.

ASSEMBLY INSTRUCTIONS

STEP 1

Flip the seat over. On the back of the seat is a triangle.


Lay the seat down in front of you, as shown to the right.

STEP 2

Look carefully at the image to the right. Lay the curved tubes crosswise on top of each other, as shown to the right.

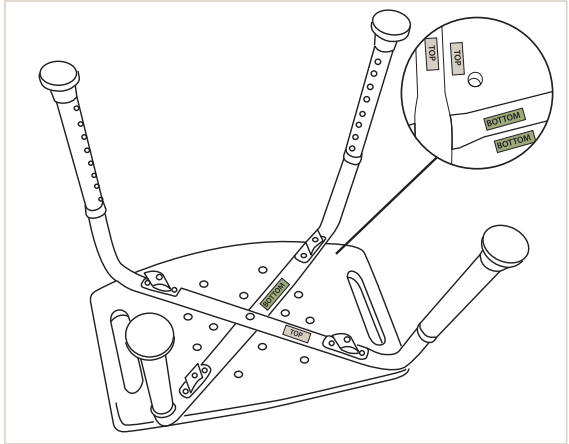
Note. One of the two tubes is "flat" in the middle, right where the tubes cross. This curved tube with the flat center should be on the bottom.

STEP 3

There are nine thumb screws included, as shown to the right.

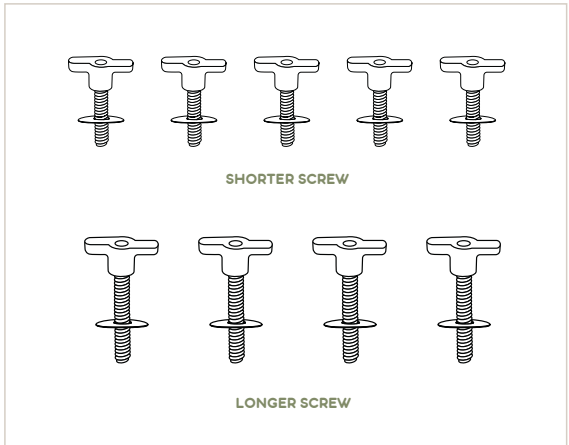
Four long and five short thumbscrews.

STEP 4


Use the short thumbscrew for the center hole, where the curved tubes cross.

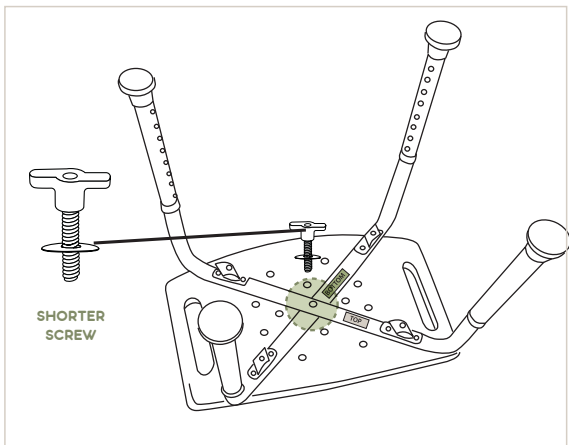Do not fully tighten the thumbscrew.
Do not fully tighten it until step 6.

STEP 5

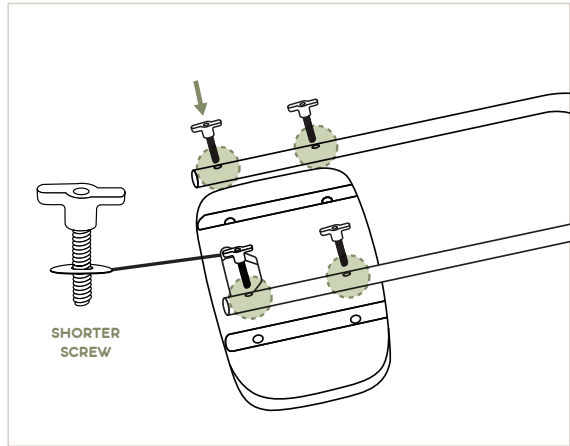
Use the short thumbscrews to attach the L-shaped tubes to the backrest. See image to the right.

STEP 6

Use the long thumbscrews to attach the L-shaped tubes of the backrest to the shower chair. See image to the right.

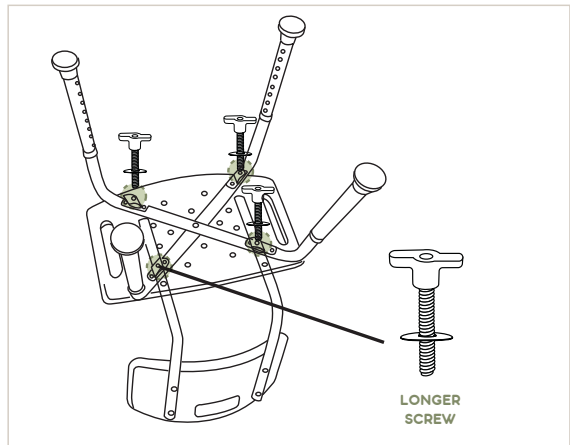Fully tighten the center thumbscrew and all other thumbscrews.

STEP 7

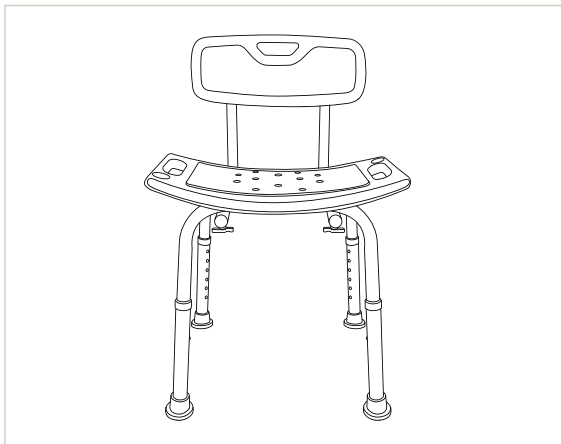
These are handles.
These handles allow you to grab hold of yourself to safely sit down and get up.

STEP 8

Attach the legs to the shower chair and place the blue seating on the shower chair.

This was the last step. Your shower chair is ready to use.

THIS PRODUCT HAS BEEN PRODUCED WITH HIGH CARE AND ACCORDING TO HIGH SAFETY STANDARDS.

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Bicciano B.V.