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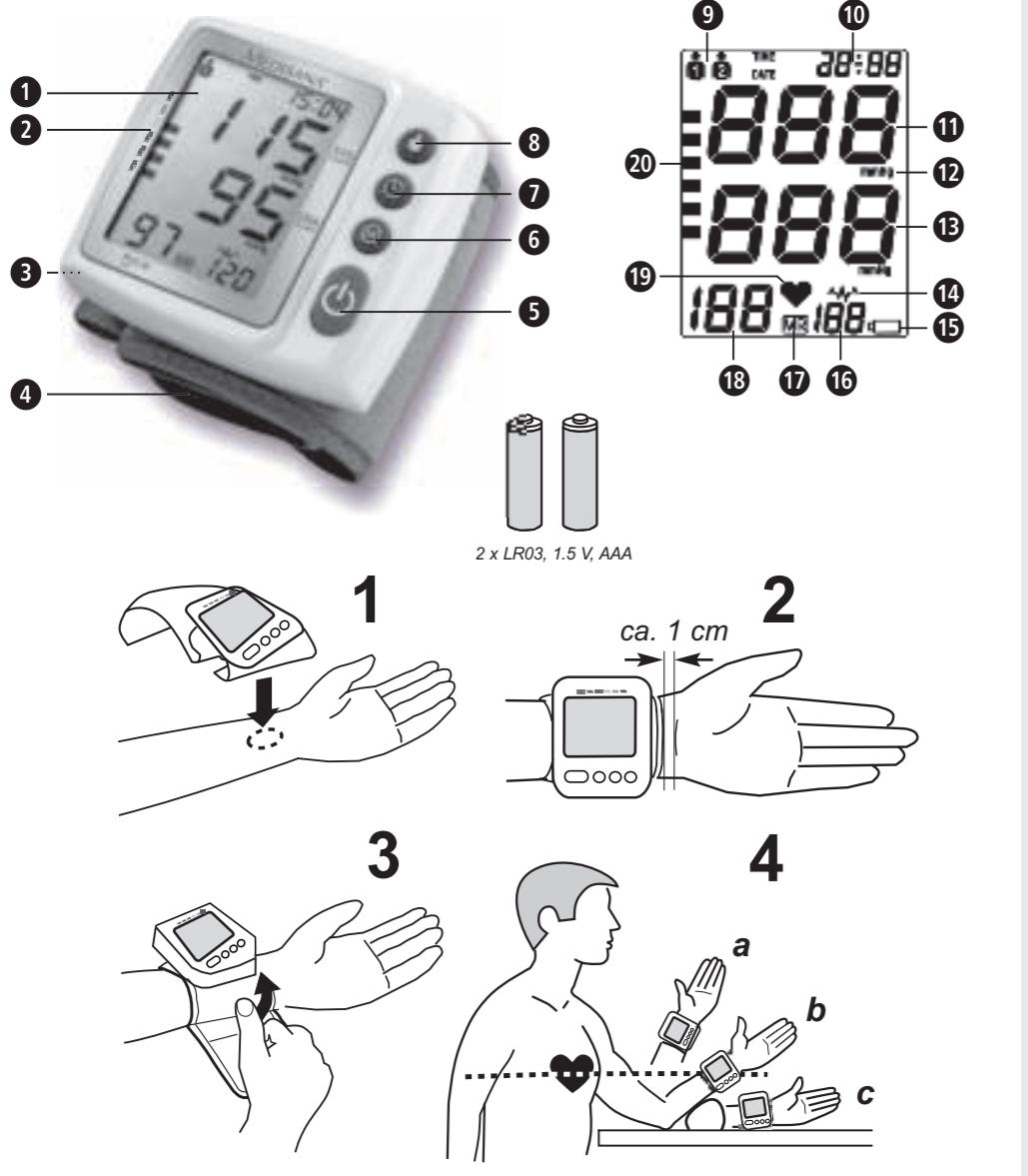
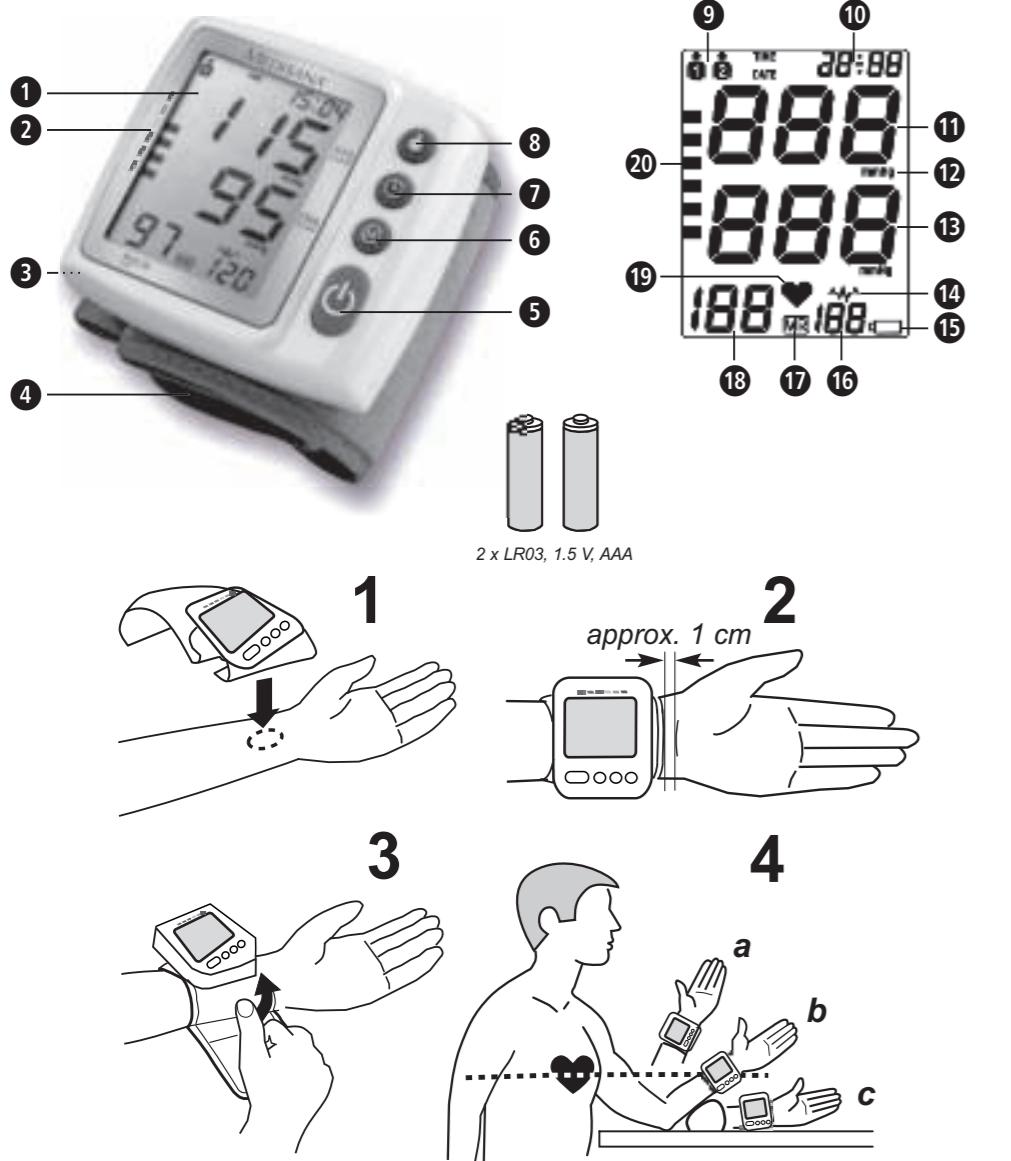
## Blood Pressure Monitor BW 315

Instruction manual - Please read carefully!

### GB Unit and LC-Display

### Proper use

- This fully automatic electronic blood pressure monitor is intended for measuring the blood pressure at home. It is a non-invasive blood pressure measurement system to measure the diastolic and systolic blood pressure and pulse of an adult using a non-invasive technique by means of a cuff, which needs to be fitted on the wrist.

### Contraindications

- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- This item is not suitable for persons who suffer from a strong arrhythmia.

### Explanation of symbols

**IMPORTANT**  
Follow the instructions for use!  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

**Device classification:** type BF applied part

**Not to be used by very young children (0-3 years)!**

**Keep dry**

**LOT**  
LOT number

**Manufacturer**  
Date of manufacture

## GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.

- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- If you experience discomfort during a measurement, such as pain in the wrist or other complaints, press the **O**-button **2** to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.
- Blood pressure measurements in high repetition rates may lead to unwanted side effects, e.g. nerve compression or blood clots.
- Blood pressure measurements - especially if executed frequently - may lead to temporary marks on the skin and / or to blood circulation disorders. If necessary, contact your physician to get more information.
- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons.
- Do not allow children to use the unit. Medical products are not toys!
- Please ensure that the unit is kept away from the reach of children.
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before being used.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves or together with HF surgical equipment, as this could cause functional defects or incorrect measurements.
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
- Do not take measurements, if any other equipment is applied to the same limb for monitoring. This could lead to disturbances or failures.
- Never apply the cuff over hurt skin, in case of an inserted catheter or after a mastectomy.
- In cases of cardiac irregularity (arrhythmia) only use this device after consultation of your doctor.
- No modification of this unit/device is allowed.
- Please do not attempt to repair the unit yourself in the event of malfunctions, since this will void all warranty claims. Only repairs carried out by authorised service centres.
- Only use original additional or spare parts from the manufacturer. Other parts could damage the device or could lead to personal injury.
- Protect the unit and measurement. If liquid does manage to penetrate into the unit, remove the liquid immediately and do not continue to use the unit. In this case contact your specialist dealer or notify us directly.
- The cuff is very sensitive and needs to be handled with care. Only inflate the cuff when it is properly positioned on the wrist.
- Protect the device from dirt, direct sunlight and extreme temperatures.
- Please do not use a detergent agent, alcohol or petrol to clean the unit.
- Please do not hit the device heavily or let it fall down from a high place.
- If the unit is not going to be used for a long period, please remove the batteries.

### SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace batteries of the same type, never use different types of batteries together or use batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

### Unit and LC-Display

- LC-display **2** Blood Pressure Indicator (green - yellow - orange - red)
- Lid of the Battery Compartment **3** Wrist cuff **4** - button (ON/OFF)
- 5** - button (Memory recall) **6** - Taste (Clock/Date) **7** - button (User Setting)
- User memory 1/2 **8** Display of Clock / Date **9** Display of Systolic Pressure
- Unit of measure **10** Display of Diastolic Pressure **11** Arrhythmia display (arrhythmia cordis)
- Change Battery symbol **12** Memory location number
- Memory symbol **13** Display of Pulse Rate **14** Pulse symbol
- Blood Pressure Indicator

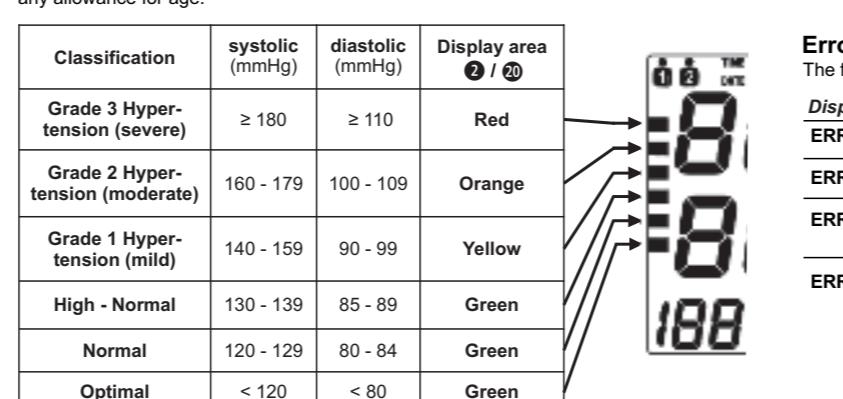
### What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

### Blood pressure classification WHO

These values have been specified by the World Health Organisation (WHO) without making any allowance for age.

Classification	systolic (mmHg)	diastolic (mmHg)	Display area <b>2</b> / <b>10</b>
Grade 3 Hypertension (severe)	≥ 180	≥ 110	Red
Grade 2 Hypertension (moderate)	160 - 179	100 - 109	Orange
Grade 1 Hypertension (mild)	140 - 159	90 - 99	Yellow
High - Normal	130 - 139	85 - 89	Green
Normal	120 - 129	80 - 84	Green
Optimal	< 120	< 80	Green

CE 0297

**WARNING**  
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

### Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

### Starting up

#### Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment **3** is located on the left hand side of the unit. Open it and insert the 2 AAA LR03 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment).

Close the battery compartment. Replace the batteries immediately, if the change battery symbol **14** appears on the display **1** or if nothing appears on the display after the unit has been switched on.

### Setting date and clock

After insertion of the batteries you need to set the date and clock. To enter the setting mode, press and hold the **5** - button **7** for approx. 3 seconds. Firstly, the user memory 1 or 2 **1** is flashing. Now press the **5** - button **7** again to adjust the correct year figure. Press **10** to correct year by pressing **10** - button **6** and press **5** - button **7** to confirm the setting. The device now switches to the month setting mode. Proceed accordingly (in the same way as with the year adjustment) to adjust the correct figures for month, day and hour and minute. Afterwards, the setting procedure is complete. Shortly, the current date appears and then the clock appears. If you want to look at the date later on, press the **5** - button **7** shortly. If the batteries are changed, the set information is lost and needs to be entered again.

### Setting up the user memory

The MEDISANA wrist blood pressure monitor BW 315 gives you the opportunity to assign readings to one of two user memories, each of which provides 120 memory slots. Press **5** - button **1** to select the user memory **1** or **2**. Your selection will be saved by the device and will be considered in next blood pressure measurement respectively when the measurement result is saved. Your selection is valid until you make another selection.

### Fitting the cuff

1. Attach the clean cuff to the left wrist which is free from clothing with the palm of the hand facing upwards (fig.1).
2. There should be a gap of approx. 1 cm between the cuff and the palm of the hand (fig.2).
3. Press the Velcro tape firmly around the wrist ensuring that it is not so tight that it could falsify the measured result (fig.3).

### Correct position for taking readings

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance.
- Raise the wrist so that the cuff of the monitor is at heart level (fig. 4, position a = too high, b = correct, c = too low).
- Keep still during the measurement: Do not move and do not talk as this may alter the measurements.

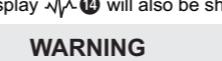
### Taking a blood pressure measurement

After the cuff has been appropriately positioned, the measurement can begin:

1. Press the **4** - button **2** to start the measurement.
2. The device automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display.

3. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol **15** starts to flash on the display.

4. When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display **1**. The blood pressure indicator **20** is shown next to the relevant coloured bar depending on the WHO blood pressure classification. If the unit has detected an irregular heartbeat, the arrhythmia display **11** will also be shown.

**WARNING**  
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

5. The readings are automatically saved in the selected memory (**1** or **2**). Up to 120 measured values with date and time can be stored in each memory.
6. Keep batteries out of children's reach!

6. The measurement readings remain on the display. The unit switches off automatically after approx. 3 minutes if no buttons are pressed. The unit may also be switched off using the **5** - button **2**.

### Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the **4** - button **2** can be pressed at any time. The device immediately decreases the cuff pressure automatically.

### Display stored values

This unit features 2 separate memories, each with a capacity of 120 memory slots. Results are automatically stored in the memory selected by the user. Press the **5** - button **1** to select the user (**1** or **2**). Now press the **4** - button **2** and the average values of the last 3 measurements appear on the display. Press the **5** - button **1** again and the latest stored measurement value will be shown, together with the memory symbol **MR** **17** and the memory location place "1" **18**. Repeatedly pressing the **5** - button **1** displays the respective values measured previously (MR2, MR3 and so on). If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 3 min. Memory recall mode can be exited at any time by pressing the **4** - button **2**. The oldest value is deleted, if 120 measured values have been stored in the memory and a new value is saved - at the same time, "FUL" shortly appears on the display.

### Delete saved values

If you are sure that you want to permanently remove all stored memories of a user, press and hold the **5** - button **2** for approx. 5 seconds, until CL appears in the display. Release the button. Now press the **5** - button **2** again to confirm the deletion.

### Error indicators

The following error codes will appear on the display when a measurement is abnormal:

Display	Cause
ERR1	No pulse was detected.
ERR2	Movement or talking during measurement.
ERR3	Inflation process lasts too long. The cuff probably has not been positioned properly.
ERR5	Systolic and diastolic pressure are too far apart from each other. Repeat the measurement. If you constantly receive unusual measurement results, please contact your doctor.

Power frequency (50/60 Hz)  
magnetic field IEC 61000-4-8

NOTE: U<sub>T</sub> is the a.c. mains voltage prior to application of the test level.

### Troubleshooting Problem

#### No display although the device is switched on and batteries are inserted

Maybe the batteries are weak. Insert new batteries and pay attention to correct polarity. Check, if the device is working well. In the case of doubt, contact the service centre.

#### Measurement can not be proceeded or measurement failed

Put on the cuff correctly. Repeat the measurement after relaxing 30 minutes. Do not speak or move during the measurement.

#### Each measurement result strongly deviates from the one before

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