AVVERTENZA

1. Come pulire:

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
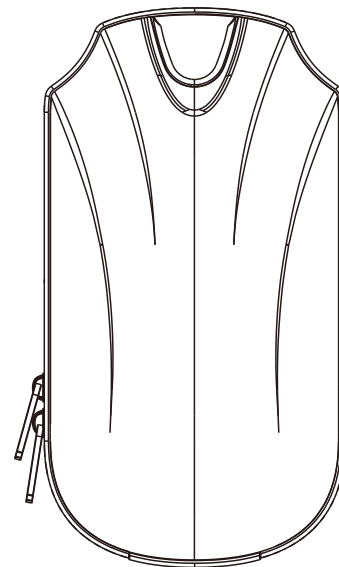
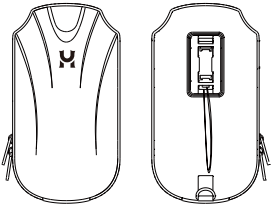
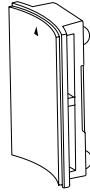
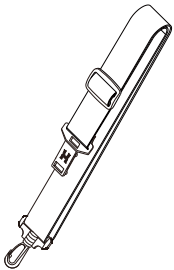
Velcro Straps

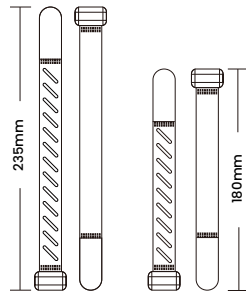SPECIFICATION

Product Name	MOILOC MOBILITY BAG SHELL
Model	Shell
color	Black
Size (L x W x H)	27 x 15 x 7.5 CMs
Weight	670 grams
Material	PU, EVA, and Polyester
Contain	1x Moiloc Mobility Bag Shell 1x Mount Magnetic Bracket 1x Adjustable Strap 2x Velcro Straps (large) 2x Velcro Straps (small)

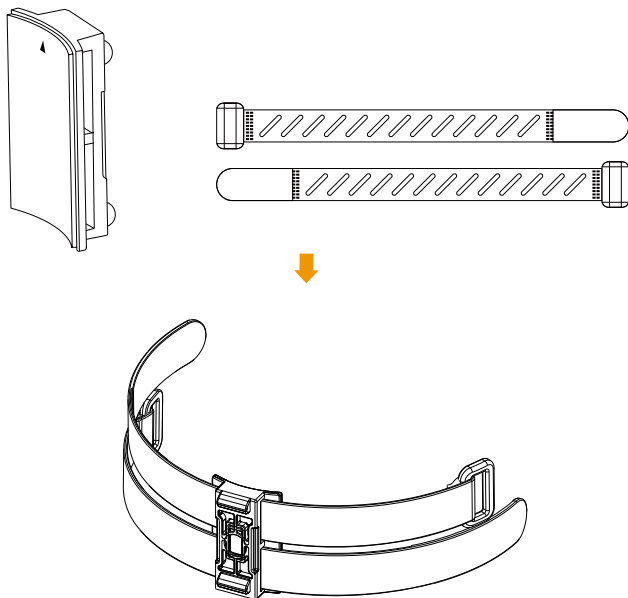
ASSEMBLY AND DISASSEMBLY

Assembly Video


1 Velcro Straps:

1. In the package you will find two sizes of Velcro Straps. Use either one that fits the vehicle of your choice the best.
2. Insert two same size Velcro Straps into the Mounting Magnetic Bracket from different directions.
3. The anti-slip side of the Velcro Straps and that of the Mounting Magnetic Bracket should be on the same face.

2 Mounting Magnetic Bracket:

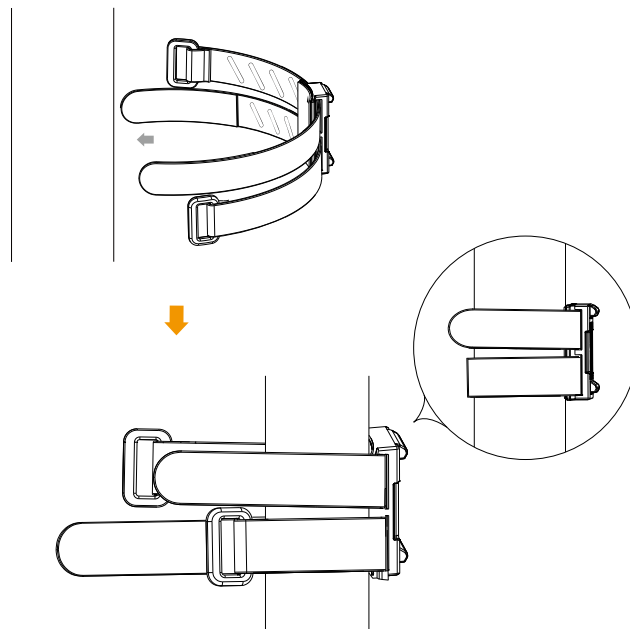1. Place the anti-slip side of the Mounting Magnetic Bracket on the surface of the stem/frame/bar of your vehicle with the arrow pointing upwards.
2. Wrap the Velcro Straps around the stem/frame/bar then feed both Velcro Straps through respective buckle.
3. Tighten the Velcro Straps, then stick the hook side to the rough side and fix it.


ASSEMBLY AND DISASSEMBLY

3 Moiloc Mobility Bag Shell on Vehicle:

To assemble: Adhere the magnetic base of the Moiloc Mobility Bag Shell to the Mounting Magnetic Bracket.

To disassemble:

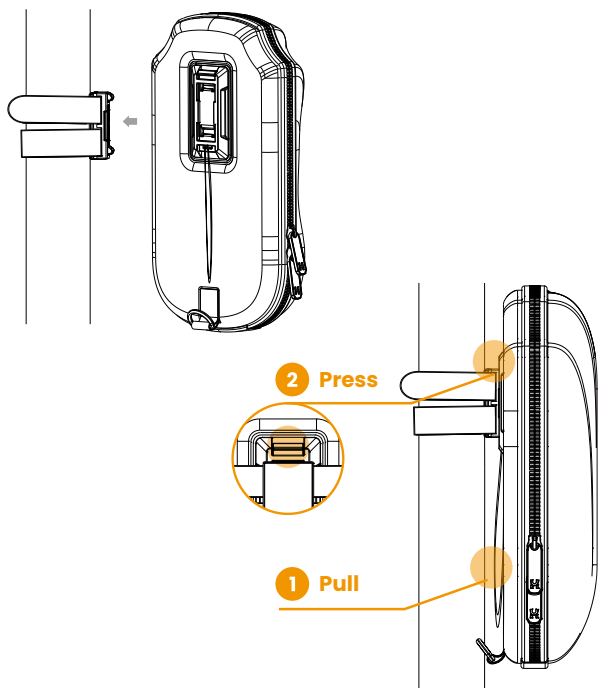1. Hold the Moiloc Mobility Bag Shell with your hand.
2. Pull the string on the bottom of the Moiloc Mobility Bag Shell; or press the button on top of the magnetic base of the Moiloc Mobility Bag Shell.


4 Moiloc Crossbody Function:

To assemble:

1. Adhere the magnetic buckle end (T-shaped) of the Adjustable Strap to the magnetic base of the Moiloc Mobility Bag Shell.
2. Connect the hook end of the Adjustable Strap with D buckle at the bottom of the Moiloc Mobility Bag Shell.

To disassemble:

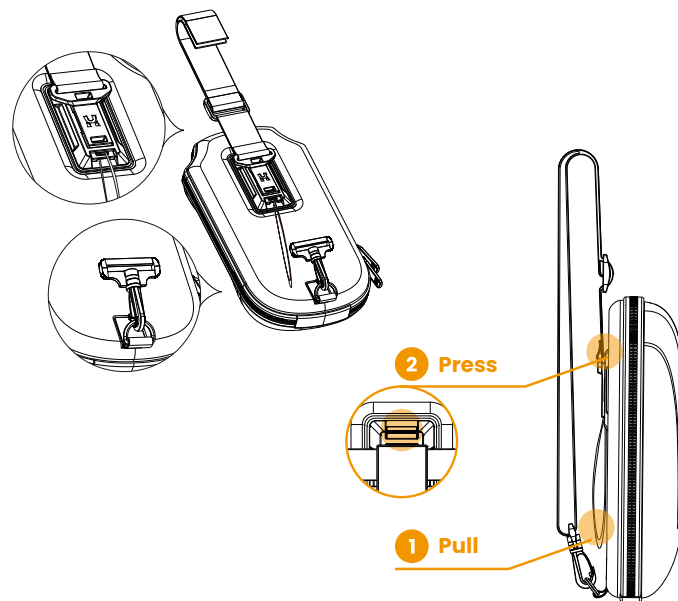1. Pull the string on bottom of the Moiloc Mobility Bag Shell; or press the button on top of the magnetic base of the Moiloc Mobility Bag Shell.
2. Disconnect the hook end of Adjustable Strap with D buckle at the bottom of the Moiloc Mobility Bag Shell.

NOTICE

1. How to clean:

- Clean with a damp cloth and then wipe with a dry cloth.
- Do not wash or use detergents and corrosive solvents for wiping.

2. Warranty

All Moiloc Mobility Bag Shell are sold with 2 years warranty from the date of purchase (proof of purchase required), provided with normal use. The warranty does not cover artificial damages (damages arising from accidents or improper usage) or any alteration, service or repair performed by any party other than Moiloc. Defective product covered by warranty will be repaired for free, and that covered by premium warranty service will be replaced with brand-new one. If it turns out that the product is out of warranty, our repair representative will contact you for repair or swap at your expenses.

More information about the premium warranty service, please visit www.moiloc.com.

3. Conditions:

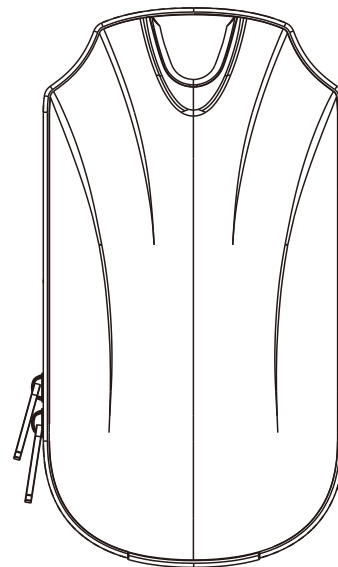
- Maximum carrying weight: 1.5 KG.
- No ripping or rough handling.
- Do not saturate or immerse the Moiloc Mobility Bag Shell into water.
- Avoid prolonged sun exposure and rain.
- Keep away from fire.
- Do not approach magnetic fields.

4. Please visit www.moiloc.com for assembling video, user manual and other information.

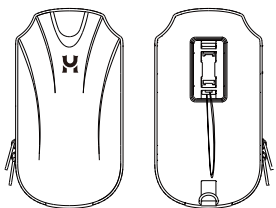
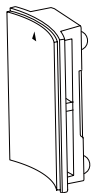
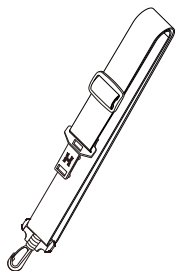
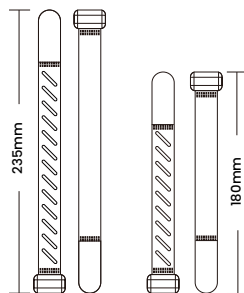
5. Any suggestion/question, you can contact us by our hotline **+31(0)235768059** (from Monday to Friday, from 9:30 to 17:00), or email: info@moiloc.com.

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