



## Shower Chairs



## Fixing and Maintenance Instructions



## INTRODUCTION

Thank you for deciding to purchase Shower Stool from Aidapt, when used correctly they are designed to give many years of reliable trouble-free service.

### IMPORTANT!

**N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.**

### MODELS AND WEIGHT LIMITS

|               |  |                 |
|---------------|--|-----------------|
| VB499/VB500   | Milton Shower Chair                              | 127 kg (20 st.) |
| VB499S/VB500S | Milton Shower Chair                              | 127 kg (20 st.) |
| VB501         | Luton Shower Chair                               | 159 kg (25 st.) |
| VB648/VB506   | Hythe Shower Chair                               | 190 kg (30 st.) |
| VB650/VB506A  | Hythe Adjustable Shower Chair                    | 127 kg (20 st.) |
| VB507         | Charing Shower Chair                             | 190 kg (30 st.) |
| VB510         | Darenth Shower Chair                             | 190 kg (30 st.) |
| VB509         | Rochester Shower Stool                           | 159kg (25 st.)  |
| VB508         | Dartford Shower Stool                            | 190 kg (30 st.) |
| VB635         | Strood Shower Stool                              | 159 kg (25 st.) |
| VB635H        | Strood Shower Stool Aluminium                    | 159kg (25 st)   |
| VB636         | Pembury Corner Shower Chair                      | 159 kg (25 st.) |
| VB638W        | Portland Bariatric Shower Chair                  | 254 kg (40 st.) |
| VB639W        | Portland Bariatric Shower Chair with Padded Back | 254 kg (40 st.) |

**DO NOT exceed the weight limit stated – doing so could put the user at risk.  
DO NOT STAND ON THIS PRODUCT.**

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## ADJUSTMENTS TO EXTENSION LEGS

Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2).

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

## USE OF BATH OILS

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.


## CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair.

Certain products have clip on/clip off backs and/or seats; you can easily remove these if required, clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/or back as this will permanently damage the plastic.

**N.B. Regular daily cleaning will reduce the build up of limescale on your product.**

UNDER NO CIRCUMSTANCES use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.

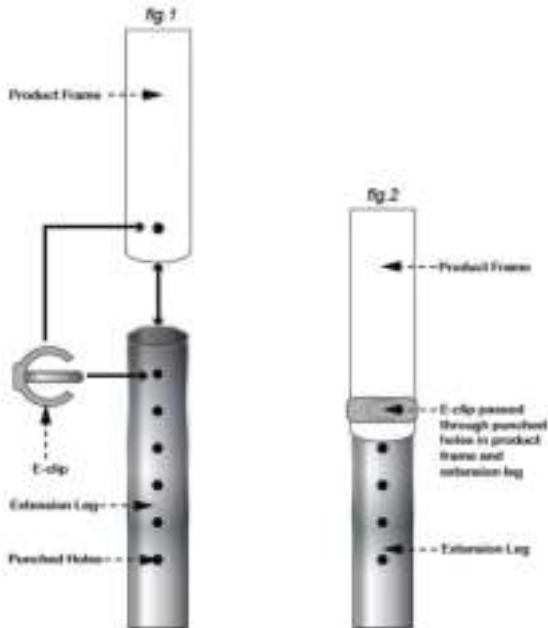

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals (at least every 6 months) or on reissue, or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

## **IMPORTANT INFORMATION**

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9  
3EX**

**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001**

**Web: [www.aidapt.com](http://www.aidapt.com)  
Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**