## Height Adjustable Crutches

**VP145/VP146/VP147/VP148/VP149 (Double Adjustable)**  
**VP149A (Bariatric Double Adjustable)**

**VP148S (Single Adjustable)**

**VP148C (Double Adjustable - Ergonomic Handle)**

Usage and Maintenance Instructions

This file is available to view and download as a PDF at [www.aidapt.co.uk](http://www.aidapt.co.uk). Sight impaired customers can use a free PDF Reader (such as [adobe.com/reader](http://adobe.com/reader)) to zoom in and increase the text size for improved readability.

User Weight Limit:

159kg (25 stone) VP145, VP146, VP147, VP148, VP149, VP148C, VP148S

222kg (35 stone) VP149A

**DO NOT exceed the weight limit stated – doing so could put the user at risk.**

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **HEIGHT ADJUSTMENT**

This range of crutches features double height adjustment for greater fit and comfort. The height can be adjusted at the top and bottom of the crutch. Follow the same steps below to adjust both the top and bottom sections of the crutch:

1. Grip shaft with one hand whilst pushing the metal locking pins down with the other hand so the locking pins retract fully into the shaft.
2. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pins in front of the required hole position. (No adjustment on top section for VP148S)
3. Repeat step 2 for the longer, bottom section of the shaft.
4. **Prior to use always ensure that the locking pins fully protrude from the selected hole position and, by applying downward pressure on the handle of the crutch, test that the crutch has been assembled correctly.**

## **CLEANING**

Clean your Crutches using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors/rivets.
- Other screw in/ bolt in/ push in components.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

Usage and Maintenance Instructions

Ver. 3 08/2012 (1719)

Usage and Maintenance Instructions

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**Altai Germany (Altai Europe Ltd.)**