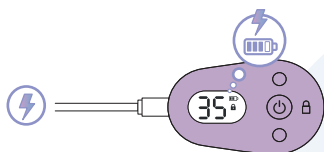
Quick Start Guide

Thank you for choosing Ellune. Quick tip: Before first use, we recommend charging your device for 2 hours. But if you're in a hurry and need relief *right now*, see the page below!

Charging Ellune

Connect the charging cable to your Ellune device. Connect the other end to a suitable USB charger.

Additional gel pads

Keep your Ellune working as effectively as possible by regularly replacing the gel pads. We recommend changing the pads every month.

Visit www.paingone.com for additional gel pads

Questions? Need accessories or spare parts?

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



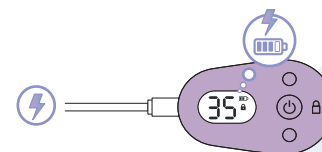
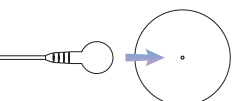
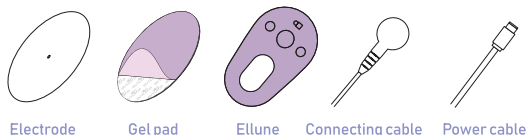
Step 1 Take the 2x electrodes from their packet and attach them to the connecting cable.

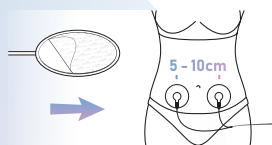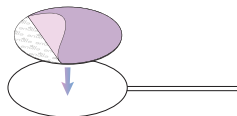Step 2 Take 2x gel pads, remove the plastic from one side (and retain this for later). Apply the sticky side to the reverse of the electrodes.



Step 3 Remove the remaining cover and place the pads on your skin, on either side of the pain, 5 to 10cm apart.

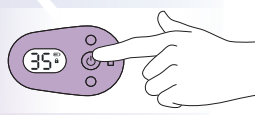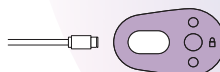Step 4 Connect the cable to the Ellune device.

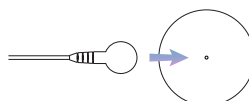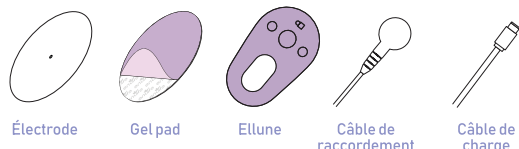Step 5 Power on your Ellune device. Set the intensity to a level you find comfortable. Relax and let Ellune soothe and relieve your period pain.

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