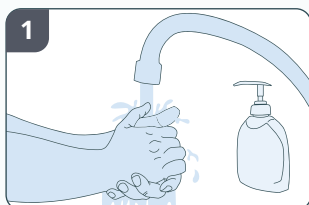
Instructies bloedafname (vingerprik) For English, see backside

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Preparation

1. Wash your hands with warm water and soap until they feel warm. Dry your hands using a clean towel. Fill a cup with hot water for step 3.
2. Choose which hand you will hold the finger pricker with, the other hand will then be the hand in which the blood samples is taken from.
3. Place your hand around the cup for at least 2 minutes to warm your hand and improve the blood flow.
4. Let the hand you will take the blood sample from hang loose for at least 1 minute to improve blood flow even more.
5. Sit at a table. Open the tube by pulling up the yellow cap. (use your nail to assist if required). **CAUTION: Do not turn the tube, this can result in the big tube disconnecting from its sleeve (mini tube) it is import that this remains intact.** Place the tube in the tube holder (of the box) Place the cap upside down next to the tube. Tip: Take the included components (alcohol wipe, gauze and plaster) out of the packaging and place this ready for use.

Taking the test

6. The ring and middle fingers of the hand are the least sensitive. Prepare the ring and middle fingers of your warmed hand by massaging these fingers from palm to fingertip for 20 seconds.
7. Clean your middle and ring finger using the alcohol wipe.
8. Remove the blue cap of the pricker by twisting it upwards. **Note: the finger pricker can only be used once. Do not touch the tip (this activates the pricking mechanism).**
9. Place your hand on the table (palm up). Prick or have yourself pricked in the side of one of your fingertips (either your middle or ring finger). The sides are the least sensitive. Press the pricker firmly against the finger. You will now feel a sting.
10. Hold your hand with fingers downwards above the tube. Massage your fingers, from palm to finger top until a blood droplet forms. Scrape the blood droplet from your finger into the tube by running it across the to edge of the tube repeat this process until the minimum amount has been collected Under the CE-IVD marking (see picture 10) Should you not meet the minimum amount of blood required, repeat the steps 6 to 10 by using a new finger pricker on another finger. **Note: With insufficient amounts of blood the analysis may not be accurate.**
11. Apply the bandage to your finger.
12. Press the cap onto the tube.
13. Swirl the tube up and down a few times to mix the blood well. **Note: this makes the analysis easier.**

Ship the test

14. Place the tube within the hard plastic component of the leak-free bag (Safetybag). Remove the white backing strip and seal the Safetybag closed by applying even pressure across the surface of the blue banding.
15. Place the Safetybag in the return envelope - removing the backing strip and fold down the flap to seal the envelope.
16. Post the envelope as quickly as possible using the regular PostNL postboxes.