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THE NATURAL SOLUTION FOR A SENSITIVE SKIN

SANATURE TAMPONS:

- Made from 100% cotton and therefore extra soft.
- Made without the addition of bleach or perfumes
- Highly absorbent for maximum comfort
- Have a neutral pH level and are therefore highly suitable for the sensitive skin
- For a fresh, clean and protected feeling

Sanature tampons are suitable for all women, also for virgins. Inserting and removing the tampon doesn't harm the hymen. We recommend you to read the instructions for use, also if you're an experienced user.

INSTRUCTIONS FOR USE

Insertion of the Sanature tampon is very simple. Wash your hands thoroughly before use and then remove the plastic packaging by tearing off the top. Position: Either seated on the toilet or standing with one foot on the toilet seat or with legs spread and knees bended.

INSERTION

Firstly, relax well. Place the tampon on the top of your index finger and push this at an angle upwards and backwards. If the tampon feels uncomfortable then gently push it a bit further backwards or remove the tampon and use a new one. The tampon cannot fall out as the muscles in the vagina will keep it in place.

REMOVAL

Relax and use the same position as with the insertion and gently pull on the string. The tampon will come out naturally. Always wash your hands thoroughly before and after this procedure.

CHANGE

Regularly change the tampon. Once every 4 hours during the day and directly on waking after a night's sleep. **IMPORTANT:** never use more than one tampon at a time.

WARNING TSS
(Toxic Shock Syndrome)

This bacterial infection is extremely rare but in extreme cases, and if the doctor is not consulted on time, can have a fatal outcome. The infection is caused by toxins from certain bacteria and can occur by men, women and children. About half of all cases of TSS occur by menstruating girls

Don't flush the tampon through the toilet.

Only use the tampon during the menstrual period. Never use it pre-emptively!

Always remove the last tampon when the menstrual period is over.

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