DETEPO

22 88 SNR SNR 22 SNR 98 FREE DAILY CASE & 2 BONUS PAIRS Noise Reduction Ear Plugs 4+2 22 Thank You We're beary grateful you chose us! **PAIRS** dB Please Read Instructions Inside Before Use sonar silicone ear plugs are designed to make your world more bearable! Simply, roll a silicone marshmallow in your palm until warm. Then press the ear plug over your ear canal for 8 seconds for a flush fit. Do not insert into the ear canal. **Product Care** Our ear plugs are reuseable, but please ensure to wash frequently with warm soapy water to remove harmful bacteria (do not use an oil based soap). Leave to dry naturally. Should they start to crumble or break apart they should be disposed of immediatley and a new pair should be used. Store in a cool, dry place. Protect from light. DETEPO CHOKING HAZARD - KEEP OUT OF REACH OF INFANTS Me ~~ ~~ Reuseable Washable Mouldable Swimming Hibernate Travel **BPA FREE** CRUELTY FREE **PACKAGING Motor Sports** Study Music