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EN Additional Information Condoms: Handle carefully, when removing from package avoid damage by fingernails, jewellery etc. If being damaged replace an individual condom. pharmacist about compatibility of any topical medicines possible contact with condom. If additional lubricant should be added, use only those recommended for condoms and never oily ones like petroleum jelly, baby oil, body lotion, etc. Use condom correctly. Do not use condoms on the penis before and contact between penis and partner body or organs to assist preventing STIs or pregnancy. Stop if you feel condom slipping off or tightening excessively onto penis because of breakage danger. Withdraw penis soon after ejaculation. Do not use condoms if you have a sore condom burst or leak during use seek medical assistance as soon as possible, at least within 72 hours. Dispose used condom to waste, not to toilet. Don't use twice, reuse might increase risk of failure or infection. For vaginal use use condom with spermicide. Condoms are a reliable protection but not 100% safe. Hazy condoms are a reliable protection but not 100% safe.

contraceptive can protect 100 % against pregnancy, AIDS and/or infectious diseases.

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mpt med production and trading GmbH