



## **Padded Raised Toilet Seat**



**VR222P/VR223P/VR209P**

## **Fixing and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Padded Raised Toilet Seat from Aidapt, when used correctly it is designed to give many years of reliable trouble-free service.

**Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.**

**NB. This equipment must be installed by a competent person.**

## BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

## CLEANING

Clean regularly with a mild household cleaner with a soft cloth. Under no circumstances use any abrasive cleaners or materials on any of the components. Always read the label of your cleaning product to check it is suitable for use on this product.

## FIXING INSTRUCTIONS

The Aidapt Padded Raised Toilet Seat is designed to fit the majority of standard sized toilet and commode seats.

The Padded Seat can be secured to your toilet seat with the integral Velcro straps located on the underside of the product. It is simply attached to your existing toilet seat using a hook and loop method commonly found on shoes/trainers. The length of the straps can be adjusted by cutting them with a pair of scissors.

**NB. Please ensure that the padded seat is secured tightly and firmly to your toilet seat before attempting to sit on it.**

**NB. If the padded seat is loose and is able to move then re-adjust the fixing straps so that it is tightly fixed in place.**

**NB. Under no circumstances should you stand on the seat.**

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Ensure Velcro straps are intact and not thread bare
- Vinyl cover has no rips or tears
- Foam filling is in tact

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**

**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**

**Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**