1. Instructions for use Propeaq 3 Light glasses

WARNING! Risk of fire/explosion. The article contains a lithium-ion battery. It must not be dismantled, taken apart, thrown into fire or short-circuited.

WARNING! Possible material damage! Do not drop the light glasses and protect them from shocks. Do not expose the light goggles to extreme temperatures or strong temperature fluctuations. Also protect the light goggles from direct sunlight and dust. Do not use chemicals, aggressive or abrasive cleaning agents to clean.

ATTENTION! Do not throw the device in the regular household waste! If you have any questions, please contact the recycling service in your municipality. Do not throw used batteries and accumulators in the household waste, but collect them separately or return them to a shop that sells batteries. The packaging can be reused or recycled. Dispose of packaging material that is no longer required properly.

CE - Technical changes reserved as part of ongoing further product development.

Disclaimer
The information contained in this document is subject to change without notice and does not represent any obligation on the part of the seller. No warranty or representation, express or implied, is made as to the quality, accuracy or fitness for a particular purpose of this document. All liability of any kind arising from the use of this product is excluded. In no event shall Chrono Eyewear B.V. be liable for any direct, indirect, special, incidental or consequential damages arising out of the use or inability to use this product or its documentation, even if advised of the possibility of such damages. The contents of this document are protected by copyright. All rights are reserved. No part of this manual may be reproduced or transmitted in any manner or for any purpose without express written permission. Product names mentioned in this document are for identification purposes only. All trademarks and product names appearing in this document is registered property.

Propeaq TimeTooler app
Download the Propeaq app from the App Store (iOS) or Google Play (Android) and follow the steps in the app. After downloading, add your personal details according to the app's screens to determine your chrono type.

Make sure the data is correct. The app will use your chrono type for calculations in order to generate a personalised user recommendation.
To use the app, please also refer to Propeaq's support page: www.propeaq.com/support

Contact details:
Chrono Eyewear BV
Koningshoeven 20, 5018 AB Tilburg
The Netherlands
info@propeaq.com
+31 (0)13 631 631 3
www.propeaq.com

PROPEAQ LIGHT GLASSES

Model 3

- This manual will help you use your Propeaq 3 light glasses. You can operate the light glasses manually.
- **Sleep improvement:** For sleep improvement, wear the light glasses for 60 minutes with blue light after waking up and possibly 60 minutes with blue light when it's getting dark.
 - **Stop Winter blues:** To stop winter blues, wear the light glasses with 30 minutes of blue light in the morning, right after waking up.
 - **Energy boost:** For more energy, wear the light glasses with blue light for 30 minutes.
 - **Anti-jetlag:** For detailed instructions on preventing jetlag, refer to the Propeaq TimeTooler app (see later in this document).
 - **Irregular work:** For detailed instructions to prevent or reduce fatigue during irregular shifts, consult the Propeaq TimeTooler app.

Warnings:

- Do not use the light glasses for longer than advised. Refer to the Propeaq TimeTooler app for personalised usage instructions.
- Do not use the light glasses in traffic or other conditions where good visibility is required.
- Do not use the light glasses in damp or wet environments.

Before use:

- Fully charge the battery of the light glasses before use.
1. Plug the micro USB cable into the light glasses.
 2. Connect the other end directly to a PC or adapter and plug it into a wall outlet.
 3. Charge the light glasses until the battery indicator shows that the battery is fully charged. This takes about 2 hours.

Package contents:

- The Propeaq 3 light glasses come with:
- Stylish frame with integrated LEDs
 - Micro USB cable
 - Glasses case
 - Microfibre pouch, to be used as cleaning cloth as well


Inside the frame you will find the power button, battery indicator lights and micro USB port

- The Propeaq 3 is supplied without lenses, as they are not necessary for operation. The optional accessories below can be ordered at www.propeaq.com:
- Dark blue lenses: Turns the Propeaq 3 into sunglasses.
 - Orange lenses: Blocks blue light for better sleep.
 - Prescription lenses: Available via our website for users who need corrective lenses.

The lenses can be easily pushed into the frame and removed again, so you can easily switch between different lenses. A manual with instructions on how to change and use the lenses is included with the order.

Battery:

- To maintain battery life, we recommend:
- switch off the light glasses when not in use; after 30 minutes they automatically switch off.
 - to avoid extreme cold temperatures.
 - to store the glasses at room temperature or place them in a warm place prior to use

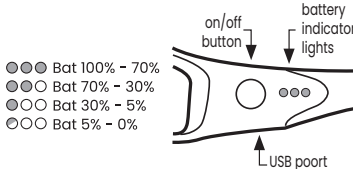Usage:
The diagram shows how to use the Propeaq 3 light glasses:

Function	Action
Battery check	Press the on/off button for 2 seconds
Switch on light glasses	Briefly press the on/off button once
Turning off light glasses	Briefly press the on/off button once

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