# BAPSCARCARE

GEL

silicone therapy  
for scars

Scar treatment

Silicone gel

e 20 g (0.7 oz.)

 **BAPMEDICAL**  
MEDICAL CARE FOR THE SKIN  
www.bap-medical.com

 BAP Medical BV, NL, 7324 BT,  
Laan Van De Maagd 119, Apeldoorn.  
Produced exclusively for BAP Medical  
by M-technologies, France. 

PRODUCT NR: 60-1020 | IFU-BSC-GEL-004

EN

## BAPSCARCARE gel

Medical silicone gel enriched with vitamin E for the treatment and prevention of abnormal scars.

### Product description:

BAPSCARCARE gel is a new generation of silicone gel for scar treatment enriched with vitamin E. The silicone gel is quick-drying, non-greasy and non-sticky, invisible and easy to apply in a single movement. BAPSCARCARE feels soft and silky and can be applied anywhere on the skin. BAPSCARCARE is suitable for treating scars on the face and neck, joints and skin folds. BAPSCARCARE gel is available in a 20 g tube.

### Composition:

Polysiloxanes, vitamin E

### Properties:

BAPSCARCARE gel makes scars flatter, more subtle and thinner and helps reduce symptoms (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints). The gel provides an occlusive environment on top of the skin, resulting in increased hydration of the skin (epidermis). The increased hydration helps the damaged scar tissue to heal more quickly and reduces functional complaints.

### Indications/uses:

BAPSCARCARE gel can be used on both old and new scars:

- Treatment and prevention of abnormal scars (hypertrophic and keloid)
- Treatment of burn scars
- Treatment of surgical scars
- Persistent functional complaints (persistent redness, pulling, itching, restricted mobility at joints, discomfort and cosmetic complaints)

### Instructions for use:

- Before use, clean and dry the skin
- Unscrew the cap and remove the foil across the top of the tube
- Apply the gel thinly in one movement: do not rub in
- After use, replace the cap on the tube
- The recommended application is twice daily, morning and evening
- The gel is for external use only
- Once the gel has dried, cosmetics can be applied over it

### Contraindications. Do not use in the following situations:

On open wounds  
Avoid contact with eyes and mucous membranes  
In case of hypersensitivity to one of the components

### Side-effects:

No side-effects have been reported to date.

### Storage:

See pack for expiry date. After opening use within 6 months. Store at room temperature, but not exceeding 25°C.

**Date of last revision of patient information sheet:** July 2016

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